

RELEASE NOTES TYPING KIT FILE - RUO

File Name: Typingkit_QTYPE_20220825.vda

Kit Lots and Revisions

This typing kit file contains the following lots and revision numbers.

Lots	Expiry Date	Revision	Allele database update	Allele database design
E061	2024/01/01	00	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E060	2023/12/01	00	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E059	2023/11/01	00	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E058	2023/10/01	00	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E057	2023/09/01	02	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E056	2023/08/01	02	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E055	2023/06/07	03	allele marsh HLADB_3.47.0	allele marsh HLADB_3.47.0
E054	2023/04/19	05	allele marsh HLADB_3.47.0	allele marsh HLADB_3.43.0
E053	2023/03/14	04	allele marsh HLADB_3.47.0	allele marsh HLADB_3.43.0
E052	2023/02/24	07	allele marsh HLADB_3.47.0	allele marsh HLADB_3.43.0
E051	2022/12/29	08	allele marsh HLADB_3.47.0	allele marsh HLADB_3.43.0
E050	2022/11/04	04	allele marsh HLADB_3.47.0	allele marsh HLADB_3.43.0
E049	2022/09/02	06	allele marsh HLADB_3.47.0	allele marsh HLADB_3.40.0

Revisions affecting LightCycler 480 II

There are no changes from Typingkit_QTYPE_20220721.

Revisions affecting QuantStudio Dx, 6/7 and ViiA 7

There are no changes from Typingkit_QTYPE_20220721.

Mix-level changes affecting all instruments and kit lots:

Well	Change
H7	Reactivity change from '+' to '-': B*50:01:01:02, B*50:01:01:08, B*50:01:08, B*50:01:18, B*50:51, B*50:57 Reactivity change from '+' to '?': B*50:01:02, B*50:01:03, B*50:01:04, B*50:01:05, B*50:01:06, B*50:01:07, B*50:01:17, B*50:05, B*50:06, B*50:07, B*50:08, B*50:09, B*50:10, B*50:11, B*50:12, B*50:13, B*50:14, B*50:15, B*50:16, B*50:17, B*50:18, B*50:19, B*50:20, B*50:32, B*50:33, B*50:34, B*50:36, B*50:37, B*50:38, B*50:39, B*50:40, B*50:41, B*50:42, B*50:43, B*50:45, B*50:46, B*50:47, B*50:48, B*50:49, B*50:50, B*50:53, B*50:71
F17	Reactivity change from '+' to '?': DRB1*01:09, DRB1*01:10, DRB1*01:16, DRB1*01:29:01, DRB1*03:174N, DRB1*03:76, DRB1*13:192, DRB1*14:109, DRB1*14:19, DRB1*14:21, DRB1*14:81
K19	Reactivity change from '?' to '-': DRB1*11:09:01:01, DRB1*11:09:01:02, DRB1*11:113, DRB1*11:115, DRB1*11:122, DRB1*11:182, DRB1*11:28:02, DRB1*11:40, DRB1*11:58:01, DRB1*11:59, DRB1*11:83, DRB1*11:87, DRB1*13:05:01:01, DRB1*13:05:01:02, DRB1*13:05:01:03, DRB1*13:05:02, DRB1*13:05:03, DRB1*13:111, DRB1*13:144, DRB1*13:158:01, DRB1*13:158:02, DRB1*13:18, DRB1*13:189, DRB1*13:199, DRB1*13:203, DRB1*13:26:01, DRB1*13:26:02, DRB1*13:277, DRB1*13:302, DRB1*13:42:01, DRB1*13:42:02, DRB1*13:50:02, DRB1*13:50:03, DRB1*13:63, DRB1*14:27:01, DRB1*14:27:02, DRB1*16:08

R01: Generation of document

CONTACT INFORMATION

Manufacturer:

CareDx AB, Franzégatan 5, SE-112 51 Stockholm, Sweden.

Tel: +46-8-508 939 00

Fax: +46-8-717 88 18

E-mail: orders-se@caredx.com

Web page: www.caredx.com

Distributed by:

CareDx Lab Solutions Inc., 901 S. Bolmar St., Suite R, West Chester, PA 19382

Tel: +1-877-653-7871

Fax: +1-610-344-7989

E-mail: orders-us@caredx.com

Web page: www.caredx.com

CareDx Pty., 20 Collie St. Fremantle, WA 6160

Tel: + 61-8-9336-42-12

E-mail: orders-aus@caredx.com

Web page: www.caredx.com

For information on worldwide distributors, contact CareDx AB.