

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
HLA-A allele <sup>1,2</sup>	ser <sup>3</sup>																									
*01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-	A1, Null, -	1			4																					
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314	A1, Null, -	1																								
*01:01:56	-	1			4							12														
*01:07, 01:95, 01:289	A1, -	1			4													17								
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																						
*01:13, 01:176	A1, -	1			4						11		13		15											
*01:15N	Null	1			w																					
*01:21, 01:126	A1, -	1		3	4																					
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:290, 11:326 <sup>4</sup>	A1, -	1			4						11															
*01:28, 01:229, 01:299, 11:282 <sup>4</sup>	-	1			4						11				15											
*01:43, 11:271 <sup>4</sup>	-	1			4					10																
*01:51	-	1			4			8		10																
*01:121	-	1			4														19							
*01:194	-	1			4								13		15											
*01:298	-	1			4					9								17								
*01:301Q	-	1			4	5		7																		
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:17-02:02, 02:18-02:22:02, 02:24-01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830-02:841, 02:843-	A2, A203, A210, Null, -	2																								
*02:17:02:01, 02:17:03-02:17:04	A2, -	2						w																		
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -	2											13							20		22				
*02:35:02	A2												13							20		22				
*02:46, 02:70	A2	2																		20		22				
*02:48, 02:129, 02:571	A2, -																			20		22				
*02:55	A2, A28	2						8													21		23			
*02:65, 02:407, 02:449	A2, A19, -	2																		19						
*02:78	-	w			4																20		22			
*02:81, 02:124	A2	2								9																
*02:135	-	2											13												24	
*02:149, 02:842	-	2																			21					
*02:169	-	2			4																					
*02:237, 02:809	-	2												14		16								23		
*02:243:01-02:243:03	-	2																18			21					
*02:309	-	2										12		14							21				24	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

*02:338	-	2	3										
*02:454	-	2						12	14				24
*02:527, 02:582	-	2				8							
*02:576, 02:682	-	2										22	
*02:644	-					8	10					21	
*02:651	-	2									20		
*02:671	-	2										23	
*02:741	-	2				8			13		20	21	22
*02:804	-	2					7						
*02:815	-	2				8						21	
*02:829	-		4				9						
*03:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:82, 03:01:84-03:07:02, 03:09-03:11N, 03:13- 03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44:01-03:49, 03:51:01- 03:56, 03:58, 03:60-03:62:01:02, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:103:01, 03:104- 03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197N, 03:201-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220- 03:224, 03:226-03:230, 03:231:02-03:245, 03:247-03:248, 03:250-03:251, 03:253-03:259, 03:261-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:321, 03:323N-03:329N, 03:331- 03:342N, 03:344-03:346, 03:348-03:374N	A3, Null, -												
*03:01:19, 03:103:02	-	3						12	13				24
*03:01:23, 03:01:83, 03:08:01:01-03:08:01:02, 03:36N, 03:57, 03:59, 03:72, 03:89:01-03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205, 03:211, 03:252, 03:267, 03:330N	A3, Null, -		3										
*03:01:38, 03:123:02, 03:171, 03:231:01	-	3							13	14			
*03:12	A3	3	4										
*03:15, 03:19, 03:30	A3	3	5						13				
*03:18, 03:135	-	1	4						13				
*03:24, 03:50	A3, -	3				8			13				
*03:32, 03:219	-	3									17		
*03:41	-	3											23
*03:43, 03:82, 03:186	-	3							13	15	17		
*03:63	-	3						11	13				23
*03:75, 03:177	-	3							13				23
*03:88	-	3	4					11					23
*03:95, 03:343	-	3							13	14	16		23
*03:152, 03:273	-	3	5								17		
*03:187	-								13				22
*03:215, 03:225, 03:249, 03:322	-								13				
*03:246, 03:282	-	3							13		19		
*03:347	-		5	7					13				23
*11:01:01-11:02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q- 11:281, 11:283-11:289, 11:291-11:305, 11:307-11:324, 11:327- 11:334, 11:336-11:347N, 11:349-11:354	A11, Null, -			4					11				
*11:03, 11:20, 11:25:01-11:25:02, 11:60, 11:158, 11:175, 11:209, 11:348	A11, -	3	4						11				
*11:10, 11:191, 11:269	A11, -		4			8			11				
*11:11, 34:19 <sup>d</sup>	-		4						11	12			



*26:09, 26:91	A26, -								8		10		12												
*26:16	A26								7		10		13												
*26:19	-				4						10		13	14											
*26:22	A26									8		10		13	14									21	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*26:30	A26									8				13										23	24
*26:54	-								8		10		12	13							20				
*26:65	-								8		10			13											24
*26:72, 26:184	-				4						10			13											
*26:78	-								8				11											23	
*26:92	-												11		13									24	
*26:120	-	1							8		10			13											
*26:122	-								8		10			13							20				
*26:152, 26:193	-								8		10	11		13											
*26:177	-												11		13				18			21		23	
*29:01:01:01-29:06, 29:08N-29:12, 29:15-29:27, 29:29-29:38, 29:40-29:47, 29:50-29:66, 29:68-29:79, 29:81-29:97, 29:99-29:104, 29:106-29:140	A29, Null, -														14										
*29:07, 29:49	A29, -					5	6								14										
*29:13, 29:39, 29:98	-														14			17							
*29:14	-														14		16								
*29:28	-								8						14										
*29:48	-														14			18							
*29:67	-				4										14										
*29:80	-														14							21			
*29:105	-										10				14			18							
*30:01:01:01-30:04:03, 30:06-30:07, 30:09-30:20, 30:22-30:54, 30:56-30:88, 30:90-30:124, 30:126-30:139, 30:141-30:149, 30:151-30:161, B*07:260 <sup>4</sup>	A30, Null, -															15									
*30:08:01-30:08:02, 30:150	A30, -				4										15										
*30:55	-													13	15										
*30:89	-															15	17								
*30:125, 30:140	-											11			15										
*31:01:02:01-31:02:02, 31:05-31:07, 31:09-31:20, 31:22-31:23, 31:26-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:108, 31:110-31:122, 31:124-31:167	A31, Null, -																16								
*31:03	A31			w									12	14	16										
*31:04:01:01	A31			w									12	14	16									24	
*31:04:01:02	-			3									12	14	16									24	
*31:08, 33:53 <sup>4</sup>	A31, A24, -					5										16		18							
*31:21	-															16	17								
*31:24-31:25	-													13		16									
*31:29	-					5	6									16									
*31:35	-														15	16	17								
*31:41	-															16					21				
*31:62	-															16						22			
*31:85	-						6									16									
*31:109, 33:125, 33:131 <sup>4</sup>	-															16		18							
*31:123	-												12	14	16										24
*32:01:01:01-32:02, 32:07-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:59, 32:61, 32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:127, 32:129-32:133N	A32, Null, -									9								17							
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																	17							
*32:04	A32, A3			3						9								17							
*32:05, 32:79	A32, -					5										16	17	18							
*32:06	A32									9								17			21				
*32:15	A32								8	9								17	18						
*32:26:01-32:26:02	-								9	9				13				17							
*32:42	-								9					14				17							

*32:62	-								9								17	19							
*32:64	-			4					9								17								
*32:72	-					6			9								17								
*32:93	-								9								17	18							
*32:128	-								9							16	17								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*33:01:01:01-33:01:14, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:178, 33:180-33:183, 33:185-33:186, 33:188-33:189, 33:191-33:193	A33, Null, -																	18							
*33:13, 33:48	-									10								18							
*33:19	-					5		7										18							
*33:22	-																	18		21					
*33:24	-																	18				23			
*33:49, 33:144, 33:179	-			w														18							
*33:51	-								8									18							
*33:61, 33:190	-													13				18							
*33:119	-							7										18							
*33:184	-										11							18					23		
*33:187	-					5	6											18							
*34:01:01:01-34:01:05, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:18, 34:23	A34, A10, -								8		11	12													
*34:02:01:01-34:03, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21	A34, Null, -			3					8		11	12													
*34:04	A34			3					8		11	12		14											
*34:08	A34			3					8		11	12	13												
*34:09	-			3					8			12												24	
*34:22	-			3					8		11	12											22		
*36:01, 36:03, 36:05-36:09	A36, -	1																					22		
*36:02	A36	1		3											14								22	23	
*36:04	A36	1			4																		22		
*43:01	A43									10	12	13													
*66:01:01:01-66:01:05, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38	A66, A26, Null, -								8		11	13													
*66:02-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N	A66, A10, Null, -								8			12												24	
*66:06	A66								8		11	13										21			
*66:09	-								8		11	13	14									21			
*66:10	-								8		11	w													
*66:12	-								8			13												24	
*66:19	-								8		11	12	13												
*68:01:01:01-68:04:02, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:05, 68:15, 68:20, 68:109, 68:136	A68, A28, Null, -								8													20			
*68:05, 68:15, 68:20, 68:109, 68:136	A68, -								8													20		23	
*68:08:02, 68:105, 68:157	A68, -								8													20	21		
*68:11N, 68:41	Null, -								8													20		22	
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8													20			
*68:19, 68:130:02	A68, -								8				14									20			
*68:25	A68								8												19	20			
*68:29	A68								8									18			20	21			
*68:45, 68:117	-							7	8													20			
*68:71	-								8				13									20			
*68:84, 68:227	-								8		10											20			
*68:103:01-68:103:02	-				3				8													20			
*68:130:01	-								8			12										20			

*68:178	-				4															20					
*68:218	-								8																
*69:01:01:01-69:01:03, 69:03-69:06, 69:08	A69, -								8															21	
*69:02	-								8			11												21	
*69:07	-								8						13									21	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*74:01:01:01-74:01:02, 74:01:04-74:03:01:02, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36	A74, Null, -																			19					
*74:01:03	-												12							19				24	
*74:04, 74:21	A74, -																			18	19				
*74:07, 74:33	A74, -																		17	19					
*74:13	-													13		16				19					
*74:19	-				4															19					
*74:23	-			3																19					
*74:30	-											11								19					
*74:35	-																			19		21			
*74:37	-													13						19					
*80:01:01:01-80:01:02, 80:03, 80:05-80:06	A80, -								6															23	
*80:02	-								6				11												
*80:04	-								6										17					23	
<i>B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60</i>	B53, -										9														
<i>B*18:27</i>	-								6																
<i>B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569</i>	-																							22	
<i>C*03:365, C*12:131</i>	-				4																				
<i>C*06:187</i>	-																							23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24