

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ^{1,2}																								
*01:02:01:01:02:17, 01:02:19-01:02:48, 01:02:50-01:02:67, 01:02:69-01:02:70, 01:02:72-01:02:81, 01:05-01:08, 01:11-01:13, 01:15:01-01:20, 01:22-01:23, 01:25-01:33, 01:35-01:42, 01:44-01:47, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:79:01-01:89N, 01:91-01:100, 01:103-01:112, 01:114:01-01:115, 01:117N, 01:119-01:121Q, 01:123-01:124, 01:126-01:130, 01:132-01:135, 01:137N-01:144, 01:145:02N, 01:147-01:159, 01:161-01:167, 01:170-01:175, 01:178-01:185Q, 01:187-01:214, 01:216-01:217:01:02, 01:219-01:224N	73																			92				
*01:02:18	73											84								92				
*01:02:49, 01:136, 01:145:01N, 01:146:01-01:146:02	73							80												92				
*01:02:68, 01:02:71, 01:168-01:169:01	73														87					92				
*01:03:01-01:03:02, 01:24, 01:34, 01:49:01-01:50, 01:55, 01:78, 01:176	73																			92	93			
*01:04	73											84		86						92				
*01:09:01-01:09:02	73													86						92				
*01:10	73	74																		92				
*01:14, 01:59, 01:118, 01:186	73											83								92				
*01:21	73											84	85											
*01:43	73	74										82								92				
*01:48	73		75																	92				
*01:60	73																		91	92				
*01:67, 01:122	73			76													88			92				
*01:90, 01:101-01:102, 01:113	73							80										89		92				
*01:116	73																	89		92				
*01:125, 01:218, 14:58 ³	73																88			92				
*01:131	73			76																92	93			
*01:160	73													86				90		92				
*01:169:02, 03:94 ³	73		75												87					92		94		
*01:177	73															88				92		94		
*01:215	73												85							92			95	
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:02:30-02:02:37, 02:02:39-02:02:43, 02:02:45-02:02:52, 02:02:54-02:04, 02:07-02:11, 02:15-02:16:02:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38:02N, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61:02, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95-02:100, 02:102-02:106, 02:108-02:110, 02:112-02:114, 02:116-02:117, 02:119-02:130, 02:132-02:133, 02:135N-02:151, 02:153-02:163, 02:165N-02:166, 02:168-02:179, 02:181-02:186, 02:188, 02:190, 02:192N-02:196, 02:198-02:210		74	75			78					83				87									
*02:02:13, 02:02:29		74	75									83			87							94		
*02:02:26, 02:21, 02:26:01-02:26:03, 02:39, 02:45, 02:189, 02:191		74	75									83			87									
*02:02:38, 02:02:53		74	75			78						83			87							94		
*02:02:44		74	75			78						83	84		87									
*02:05:01-02:05:03, 02:17		74	75			78						83		86	87									
*02:06:01-02:06:02, 02:23, 02:36:01-02:36:02		74	75			78		80				83		87	87		89							
*02:12, 02:55:01-02:55:02		74	75			78						83	85	87										
*02:13, 02:18, 02:33, 02:75		74	75			78						83		87				90						
*02:14:01-02:14:02, 02:27:01-02:27:02, 02:65, 02:131, 02:164, 02:187		74	75			78						83												
*02:22, 02:62, 02:118		74	75			78						83			87							93		
*02:40:01-02:40:02		74	75			78						83			87				91					
*02:47, 02:111		74	75			78		80				83			87									
*02:49		74	75			78						83	85	87				90						
*02:58			75			78						83		87										
*02:68		74	75			78						83		87			89							
*02:87		74										82												
*02:94		74	75			78	79					83		87										
*02:101		74	75			w						83		87										
*02:107		74										83												
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*02:115		74	75			78						83	85					90						
*02:134, 02:152, 02:167		74										83			87									
*02:180		74	75			78						83			87			90	91					
*02:197			75			78						83			87						93			

Negative Control

*03:04:07, 03:41:01-03:41:02, 03:80:01, 03:113:01-03:113:02, 03:260, 03:286, 03:293, 03:384, 03:435, 03:512			75																		93	94	Negat		
*03:04:15, 03:04:25-03:04:26, 03:118, 03:137, 03:145, 03:156, 03:166, 03:170, 03:172, 03:342, 03:454, 03:459			75	76																		93			
*03:07:01:01-03:07:02, 03:45, 03:163, 03:297			75	76					83					87								93		94	
*03:10, 03:29, 03:209			75	76										87								93		94	
*03:13:01:01-03:13:02, 03:97, 03:132, 03:167, 03:177, 03:271, 03:474				76	77																			94	
*03:15			75	76					83					87										94	
*03:16, 03:536, B*46:77 ³			75																					94	
*03:58	73				77									87							92	93		94	
*03:67, 03:295				76	w																			93	w
*03:81, 03:175				76	77															89				93	94
*03:86	73				77									87								92		94	
*03:130, 03:140:01-01-03:140:01:02			75	76					83															93	94
*03:155			75	76											88									93	94
*03:199, 03:245			75	76											89									94	
*03:231				76		78			83						88									93	
*03:243, 03:308				76	77				83															93	94
*03:248			75	76					83																
*03:262				76	77																			?	94
*03:268, 03:461				76	77				83					87										93	94
*03:278, 03:566			75	76																				93	w
*03:280			75	76					82															93	94
*03:302			75	76										87											94
*03:317, 03:388			75	76																89				93	94
*03:324, 03:479				76	77																			93	
*03:351				76	77										88									93	94
*03:416	73			76	77																			93	94
*03:450				76	77									87										93	94
*03:530				76					82															93	
*03:557				76											88									93	94
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94		95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94		95	96
*04:01:01:01-04:01:01:29, 04:01:01:31-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:60, 04:01:62, 04:01:64, 04:01:66-04:01:73, 04:01:75-04:01:143, 04:04:01:01-04:05, 04:07:01, 04:08-04:09N, 04:12-04:13:01:02, 04:15:01-04:15:03, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:34-04:35:02, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:70, 04:73-04:76, 04:78-04:79, 04:81-04:99, 04:101-04:102, 04:104-04:106, 04:109-04:111, 04:113, 04:115N, 04:117-04:128, 04:131-04:139, 04:141-04:146, 04:148-04:152, 04:154-04:159, 04:161-04:166:01, 04:166:03-04:168, 04:170N, 04:173N-04:177, 04:179-04:189, 04:191N, 04:193, 04:195, 04:197-04:201:01, 04:202-04:209, 04:211, 04:216-04:219, 04:221-04:226, 04:228-04:229, 04:232-04:235, 04:237-04:248, 04:250-04:255N, 04:257-04:262, 04:264, 04:266-04:277, 04:280-04:282, 04:284-04:285, 04:287-04:293, 04:295-04:298, 04:300N-04:303, 04:305N-04:310, 04:312-04:334, 04:336, 04:338Q-04:340, 04:342-04:346, 04:348-04:350N, 04:352-04:356, 04:358-04:359, 04:361-04:362N, 04:364N-04:366, 04:369N-04:380, 04:382Q, 04:384-04:392, 04:394-04:396N, 04:398-04:399, 04:401, 04:403-04:405, 04:407-04:413, 04:415-04:421, 04:423-04:431, 04:433-04:435, 04:437-04:438, 04:440-04:441, 04:443-04:446, 04:448-04:469						78				83		85					90					93			
*04:01:16, 04:54:01-04:54:02, 04:72, 04:341, 04:367, 04:439						78				83		85					90								
*04:01:23, 04:33, 04:116, 04:130, 04:166:02, 04:231, 04:283						78				83		85												93	
*04:01:28, 04:01:61, 04:01:63, 04:01:65, 04:140, 04:196, 04:213, 04:236N, 04:304:01-04:304:02:02, 04:311										83		85					90							93	
*04:01:74						78				83		85					w							93	
*04:03:01:01-04:03:08, 04:06:01-04:06:03, 04:80, 04:147, 04:171, 04:190, 04:256, 04:286, 04:294, 04:337, 04:363, 04:381, 04:393, 04:400, 04:402			75			78				83		85		87			90							93	
*04:03:09			75							83				87										93	94
*04:07:02										83														93	94
*04:10, 04:153, 04:192, 04:194, 04:210, 04:214-04:215N, 04:227, 04:249, 04:278, 04:347						78				83							90							93	
*04:11						78				83					88								93		
*04:14, 04:68						78				83		85					90						93	95	
*04:16, 04:351:01-04:351:02			75							83		85		87			90						93		
*04:23						78		80		83		85					90						93		
*04:29, 04:114						78				83		85		88			90						93		
*04:32, 04:77, 04:265			74			78				83		85					90						93	94	
*04:36, 04:55						78				83				88			90						93		
*04:37	73					78				83		85		87			90		92				93		
*04:42:01-04:42:02, 04:220										83		85	86				90						93	95	
*04:58, 04:368						78				83		85					90	91					93		
*04:71, 04:436N	73					78				83		85					90						93		
*04:100						78		80		83		85					90						93		

Negative Control

