

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*44:127	-				28													41							
*44:131	-																	41							
*44:140	-				28							35						41							
*44:146	-				28													41							
*44:150	-	25			28		30											41							
*45:01, 45:05, 45:07, 45:11-45:14	B45(12), -				28				32			35													48
*45:04	-				28				32																48
*45:06	-				28				32			35													48
*45:08	-				28				32																48
*45:09	-				28				32					36	37										48
*45:10	-				28				32		34														48
*46:01:01-46:01:06, 46:04-46:05, 46:07N, 46:09-46:10, 46:14-46:16, 46:20, 46:22-46:24, 46:27-46:30	B46, Null, -								31				36												
*46:01:07	-								31				36												
*46:02	B46								31				36												
*46:03, 46:08, 46:13:01-46:13:03, 46:19, 46:21:01-46:21:02, 46:25-46:26	B46, -								31				36												
*46:06	-								31				36												
*46:11	-								31																
*46:12	-			27					31				36												
*46:17	-								31				36	37											
*46:18	-								31																
*47:01:01-47:01:02, 47:06-47:08	B47, -				28																				
*47:02	B47				28																				
*47:03	B47				28																				
*47:04-47:05	-				28	29																			
*48:01:01-48:01:03, 48:07, 48:09, 48:11, 48:19-48:20, 48:22, 48:27	B48, -											35													48
*48:02:01-48:02:02, 48:25	B48, -											35	36					41							48
*48:03:01-48:03:02	B48											35													48
*48:04, 48:24	B48, -											35													48
*48:05, 48:08	B48, -	25										35													48
*48:06	-																								48
*48:10, 48:12	B48, -											35													48
*48:13	-											35													48
*48:14	-											35	36												48
*48:15	-	25										35													48
*48:16	-											35													48
*48:17	B48, 40											35													48
*48:18	-											35													48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

	?							58	60		62							70		-	*40:109					
	?										62							70		-	*40:110					
49	?							58	60		62							70		-	*40:117					
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
49	?			53								61	62									71		-	*40:132, 40:178	
	?	51										61	62									71		-	*40:136	
49													62									71	72	-	*40:137	
49												61	62									71		-	*40:138	
	?									59		61	62									71		-	*40:148, 40:161	
	?												62									70		-	*40:157	
	?											61	62									71		-	*40:158	
	?											61	62									71		-	*40:159	
49	?		52									61										71		-	*40:166	
49												61	62									71		-	*40:170	
	?											61	62									71		-	*40:174	
49	?											61	62									71		-	*40:183	
	?											61	62									71		-	*40:184	
49													62									71		-	*40:185	
		51			54							61	62									71		B41, -	*41:01, 41:06-41:07, 41:12, 41:14, 41:16-41:17, 41:20	
		51										61	62									71	72	B41, -	*41:02:01-41:02:04, 41:11, 41:13, 41:19	
		51										61	62									71		B41, -	*41:03:01-41:03:02, 41:15	
		51										61	62									71	72	-	*41:04, 41:10, 41:18	
		51			54							?	62									71		-	*41:05	
		51										61	62									71		-	*41:08	
					54							61	62									71		-	*41:09, 45:02-45:03 ¹⁰	
		51	52																			71	72	B42, -	*42:01:01-42:02, 42:06, 42:10, 42:12, 42:15	
		51	52		54																	71		-	*42:04	
		51	52																			71	72	-	*42:05:01-42:05:02	
			52																			71		-	*42:08	
		51	52																			71	72	-	*42:09	
		51											62									71	72	-	*42:11	
		51	52	53																		71	72	-	*42:13	
		51	52																			71		-	*42:14	
		51	52																			71	72	-	*42:16	
																									B44(12), Null, -	*44:02:01:01-44:02:21, 44:02:23-44:02:24, 44:11, 44:19N, 44:21-44:24, 44:27:01-44:27:02, 44:33-44:34, 44:48, 44:52N, 44:55, 44:59, 44:63, 44:66-44:68, 44:71-44:74, 44:80, 44:84, 44:86-44:87, 44:89, 44:93, 44:101-44:102, 44:104, 44:112-44:113, 44:116, 44:118-44:119, 44:121, 44:126:01, 44:137, 44:142Q, 44:148-44:149N, 44:151-44:152

				53								63								70				B44(12)		*44:02:22, 44:12		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.		
																						ser. ⁵		HLA-B allele ⁴				
				53																				B44(12)	*44:03:01-44:03:06, 44:03:08-44:05:03, 44:13, 44:26, 44:28:01-44:30, 44:32, 44:35-44:36, 44:38-44:39, 44:42, 44:45, 44:51, 44:56N, 44:58N, 44:61N, 44:69-44:70, 44:76-44:79, 44:82, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:105, 44:107-44:109, 44:114-44:115, 44:117, 44:120, 44:122-44:125, 44:128, 44:133, 44:136, 44:139, 44:141, 44:143, 44:147			
				53								62								70				B44(12)	*44:03:07, 44:07, 44:81, 44:103, 44:111, 44:134			
				53	54					60		63							68	70				B44(12)	*44:06			
				53								62	63							70				B44(12)	*44:08			
				53								61	62	63								71		B45(12)	*44:09			
				53								62				65				70				B44(12)	*44:10			
				53								62								70				B44(12)	*44:14			
					54							62				65				70				B12	*44:15			
				53								62	63							70				B47, -	*44:16, 44:91, 44:132			
				53								62								70				B44(12)	*44:17, 44:43:01-44:43:02, 44:144			
					54					58		62				65				70				-	*44:18			
				53	54							62	63							70				-	*44:20, 44:100			
				53						58	60	62								70				-	*44:25, 44:50			
49				53								62								70				B44(12)	*44:31			
												62								70				-	*44:37:01-44:37:02, 44:64:01-44:64:02			
				53								62								70				B44(12)	*44:40, 44:130			
				53								62	63							70				B44(12)	*44:41:01-44:41:02, 44:53, 44:99, 44:138Q			
				53								62	63							70				-	*44:44			
				53								61	62									71		-	*44:46, 44:75, 44:129			
				53								62								70				-	*44:47, 44:65			
				53								62	63							70				-	*44:49			
				53								62								70				B44(12)	*44:54			
				53								62								70				-	*44:57			
				53								62								70				-	*44:60, 44:110			
				53								62								70				-	*44:62			
				53								62	63							70				-	*44:83, 44:145			
					52	53						61	62	63								71		-	*44:90			
						53						58	60			62	63			70				-	*44:95			
					52	53						62	63							70				-	*44:97			
						53						62	63							70				-	*44:106			

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																								ser. ⁵	HLA-B allele ⁴	
				53								62	63											-	*44:127	
				53								61	62	63								71		-	*44:131	
				53								62				65					70			-	*44:140	
				53								62									69	70		-	*44:146	
												62									70	72		-	*44:150	
					54							61	62			65						71		B45(12)	*45:01, 45:05, 45:07, 45:11-45:14	
					54							61	62			65						71		-	*45:04	
			52		54							61	62			65						71		-	*45:06	
					54							61	62	64								71		-	*45:08	
					54							61	62									71		-	*45:09	
					54							61	62									71		-	*45:10	
			52			55															69	71		B46, Null, -	*46:01:01-46:01:06, 46:04-46:05, 46:07N, 46:09-46:10, 46:14-46:16, 46:20, 46:22-46:24, 46:27-46:30	
	50		52			55															69	71		-	*46:01:07	
			w			55															69	71		B46	*46:02	
				52		55																71		B46, -	*46:03, 46:08, 46:13:01-46:13:03, 46:19, 46:21:01-46:21:02, 46:25-46:26	
49				52		55																71		-	*46:06	
				52		55										65						71		-	*46:11	
				52		55															69	71		-	*46:12	
				52		55															69	71		-	*46:17	
				52		55									64							71		-	*46:18	
	50					56						62										70		B47, -	*47:01:01:01-47:01:02, 47:06-47:08	
	50					56						61	62									71		B47	*47:02	
	50					56						w	62									70	w	B47	*47:03	
	50											62										70		-	*47:04-47:05	
								57				62											71	72	B48, -	*48:01:01-48:01:03, 48:07, 48:09, 48:11, 48:19-48:20, 48:22, 48:27
												62											71		B48, -	*48:02:01-48:02:02, 48:25
49												62										71		B48	*48:03:01-48:03:02	
								57				62											71		B48, -	*48:04, 48:24
												62											71	72	B48, -	*48:05, 48:08
								57															71	72	-	*48:06
												62											71	72	B48, -	*48:10, 48:12
												62											71		-	*48:13
												62											71	72	-	*48:14
								57				62											71	72	-	*48:15
								57															71	72	-	*48:16
49												62											71	72	B48, 40	*48:17
								57				62										70	72	-	*48:18	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	

																								ser. ⁵	HLA-B allele ⁴	
												62												71	-	*48:21, 48:26
49												62												71	-	*48:23
												58 59	62	65	70	B49(21), Null, -	*49:01:01, 49:01:03, 49:06, 49:08-49:10, 49:12-49:17, 49:19N									
												58	62	65	70	B49(21), -	*49:01:02, 49:11									
												59	62	65	70	B49(21)	*49:02									
												59	62	65	70	-	*49:03									
												58 59	62	70	B49(21), -	*49:04-49:05										
												58 59	62	70	-	*49:07										
												58 59	62	70	-	*49:18										
												58 59	62	65	70	-	*49:20									
												59	61 62	65	71	B50(21), -	*50:01:01-50:01:03, 50:04-50:05, 50:08, 50:10-50:11, 50:13, 50:16									
												59	61 62	65	71	B45(12)	*50:02									
												59	62	65	71	-	*50:06									
												59	61 62	65	71	-	*50:07									
												59	61 62	71	-	*50:09										
												61 62	65	71	-	*50:12										
												59	61 62	71	-	*50:14										
												59	61	65	71	-	*50:15									
54												59 60	68	70	B51(5), B5102, B5103, Null, -	*51:01:01, 51:01:03, 51:01:05-51:01:08, 51:01:10-51:01:20, 51:01:22-51:01:24, 51:01:26-51:02:01, 51:02:03, 51:02:05-51:03, 51:07:01, 51:09:01, 51:11N-51:12, 51:13:02-51:14, 51:16-51:19, 51:21-51:23, 51:26-51:33, 51:38-51:41N, 51:43, 51:48-51:53, 51:55, 51:57-51:58, 51:60, 51:65-51:67, 51:69-51:77, 51:79-51:80, 51:82-51:88, 51:90-51:91, 51:94-51:96, 51:98N, 51:100, 51:102, 51:105, 51:107-51:111, 51:113-51:115, 51:117, 51:120-51:130										
												59 60	68	70	B51(5), B5102, -	*51:01:02, 51:02:02, 51:05, 51:07:02, 51:09:02, 51:89, 51:103, 51:116, 51:119										
54												59 60	68	70	B51(5), B5102, -	*51:01:04, 51:02:04, 51:13:01, 51:24:01, 51:34-51:35, 51:92, 51:99, 51:131										
												59 60	68	70	B51(5)	*51:01:09										
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.		
																								ser. ⁵	HLA-B allele ⁴	

			52				56											69	71	B22	*56:03				
			52				56		59				64						71	B56(22)	*56:05:01				
			52				56		59										71	B56(22)	*56:05:02				
			52				56		59				64			w			71	B78	*56:06				
			52				56						65					70	B56(22)	*56:07					
			52				56												71	-	*56:09				
			52				56						64			67			71	B55(22)	*56:10				
			52				56												71	B56(22)	*56:11-56:12				
			52		54		56							65					71	B56(22)	*56:13				
														65					71	-	*56:17, 56:33				
			52				56							65					71	-	*56:20:02				
			52				56		58 59									70	-	-	*56:21				
													64						71	-	*56:23				
			52				56		59					65					71	-	*56:25				
									60										70	B57(17)	*57:01:01-57:01:04, 57:01:06-57:01:13, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:22-57:23, 57:25-57:27, 57:29-57:30, 57:33-57:38, 57:40-57:41, 57:43-57:44, 57:48-57:50, 57:52-57:56				
									60										70	B57(17)	*57:01:05, 57:11, 57:13, 57:21, 57:31, 57:47				
									60										70	B57(17)	*57:02:01-57:03:02, 57:17, 57:28N, 57:32, 57:39, 57:42, 57:46, 57:57				
									60										70	B57(17)	*57:04				
			50						60										70	-	*57:05				
					53				60										70	-	*57:07				
					53				60										70	-	*57:09				
																			71	-	*57:12				
							55		60										70	-	*57:14				
					53				60										70	-	*57:24				
								58	60										70	-	*57:45, 57:51				
									60										70	B58(17)	*58:01:01-58:01:02, 58:01:04-58:01:11, 58:04-58:05, 58:10N-58:15, 58:19, 58:21-58:24, 58:29-58:33, 58:35				
									60										70	B58(17)	*58:01:03, 58:02, 58:06, 58:16, 58:25-58:26				
									60										70	-	*58:07				
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
																								ser. ⁵	HLA-B allele ⁴
					54				59	60									67	70	B17	*58:08:01			
									59	60									67	70	-	*58:08:02			
					55				60										67	70	-	*58:09			
																			67	70	Null	*58:17N			
									60										67	70	-	*58:18			
									60										67	70	-	*58:20			
									60										67	70	-	*58:27			

