

INTERPRETATION TABLE

HLA-B low resolution SSP

Amplification patterns of the B*07:02 to B*83:01 alleles

		Well																														
		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48							
Length of spec.		110		215		140		130		185		180		290		165		150		140		60		210		170		110		395		160
PCR product(s)					235		265		235						435	395	105				400	245								425		
Length of int. pos. control ¹		800		1070		1070		800		800		800		1070		1070		800		1070		800		800		1070		1070		1070		
5'-primer(s) ²		5'-TgA ³ 527	5'-TCg ³ 97	5'-ggC ³ 209	5'-CCg ³ 103	5'-CCT ³ 103	5'-CCg ³ 103	5'-ggA ³ 45	5'-ggA ³ 45	5'-ggA ³ 45	5'-TTA ³ 420	5'-AgA ³ 206	5'-gTC ³ 419	5'-TCT ³ 142	5'-ggA ³ 45	5'-Cgg ³ 161	5'-gCT ³ 167	5'-TCA ³ 355	5'-gAC ³ 206	5'-ggA ³ 45	5'-TCT ³ 142	5'-ggC ³ 409	5'-gAA ³ 246	5'-ggC ³ 44	5'-ggC ³ 44	5'-TCA ³ 355	5'-gAC ³ 206	5'-ggA ³ 45	5'-TCT ³ 142	5'-ggC ³ 409	5'-gAA ³ 246	5'-ggC ³ 44
3'-primer(s) ³		5'-gCT ³ 605	5'-TgA ³ 272	5'-gCC ³ 312	5'-CgT ³ 193	5'-TAT ³ 246	5'-gCC ³ 572	5'-TgG ³ 165	5'-gCT ³ 206	5'-TCC ³ 266	5'-CTC ³ 559	5'-gCg ³ 272	5'-TAT ³ 3rd I	5'-gTC ³ 301	5'-ATC ³ 272	5'-TCT ³ 234	5'-TCT ³ 204	5'-gCT ³ 463	5'-gGC ³ 302	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	5'-gTT ³ 259	
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48							

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
			52																				71 72	B7, B7/blank, B7/B42, Null, -	*07:02:01-07:02:31, 07:02:33-07:02:38, 07:05:01-07:06, 07:10, 07:12-07:15, 07:18:01-07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34-07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55-07:59, 07:61-07:64, 07:66-07:68:03, 07:70-07:72, 07:74-07:76, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:121, 07:124-07:130, 07:132, 07:135N-07:138, 07:140-07:145, 07:147-07:148, 07:150-07:154, 07:156-07:161N, 07:163, 07:166-07:173, 07:175-07:178, 07:181N, 07:183-07:190	
			52																				71 72	-	*07:02:32	
																							71 72	B703/B7, B7/blank	*07:03, 07:08, 07:16, 07:32, 07:37	
		51	52																				71 72	B7, -	*07:04, 07:25, 07:146	
			52																				71	B7, -	*07:07, 07:77	
			52																				71 72	B7, -	*07:09, 07:11, 07:17, 07:162	
			52																				71	B7/42, -	*07:19, 07:33, 07:40, 07:53, 07:122, 07:139, 07:165	
			52																				71 72	B7/blank, -	*07:20, 07:24, 07:131	
																					70	72	-	-	*07:27	
			52																				71 72	B7	*07:28	
			52																			70	72	B7, -	*07:36, 07:81, 07:149	
																						70	72	-	*07:38	
			w																				71 72	B7	*07:47	
			52																				71 72	B7, -	*07:48, 07:51, 07:155, 07:164	
																							71 72	-	*07:50	
			52									62											71 72	B7, -	*07:54, 07:123	
			52																				71	-	*07:60	
			52				56						64						w				71 72	-	*07:65	
																							71 72	-	*07:69, 07:85	
			52																				71 72	-	*07:73	
			52							59													71	-	*07:78	
			52							59													71	-	*07:84	
			52																		69	71	-	-	*07:100	
			52									61											71 72	-	*07:133	
			52																w				71 72	-	*07:134	
			52					57															71 72	-	*07:174	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	Well No.
																										HLA-B allele ⁴

			52																71	72	-		*07:179		
																			70	72	-		*07:180		
			52																	71	72	Null		*07:182N	
			51																	71	72	B8, Null, -		*08:01:01-08:01:20, 08:05, 08:08N, 08:10-08:11, 08:15, 08:18-08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41-08:48, 08:51, 08:53:01-08:53:02, 08:56:01-08:59:02, 08:61, 08:63-08:64, 08:66-08:69, 08:71-08:73, 08:75, 08:80-08:83, 08:86N, 08:90-08:93, 08:95-08:100, 08:104-08:106	
			51																70	72	B8, -		*08:02-08:03, 08:52, 08:78		
			51																	71	72	B8/blank		*08:04	
			51																	71	72	B8, -		*08:07, 08:14, 08:74, 08:77, 08:108	
			51				54													71		B8, -		*08:09, 08:84	
			51																	71		B8		*08:12:01-08:12:03, 08:16	
																				71	72	B8, -		*08:13, 08:20, 08:40, 08:70	
			51										62							71	72	-		*08:17, 08:54	
			51																	71	72	B8		*08:21	
																				71	72	B8		*08:25	
			51																	71	72	B8, -		*08:26, 08:50, 08:62, 08:85, 08:94, 42:07 ⁶	
			51																	71	72	B8, -		*08:28, 08:37	
			51																	71	72	B8		*08:32	
			51																	71	72	-		*08:35	
			51																	71	72	-		*08:36	
			51										62							71	72	-		*08:38, 08:101	
			51				53													71	72	-		*08:49	
																				71	72	-		*08:55	
			51				53													71	72	-		*08:60, 08:76	
			51																	71	72	-		*08:65	
																				71	72	-		*08:79	
			51				53						63							71	72	-		*08:87	
			51																	71	72	-		*08:88, 08:102	
			51																	71		-		*08:89, 08:107	
																				71		-		*08:103	
													62												
																				70		B13, Null, -		*13:01:01-13:01:06, 13:17, 13:20, 13:22:01-13:23, 13:25, 13:28-13:29, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
																								ser. ⁵	HLA-B allele ⁴

										w		62				69	71	B62	*15:04							
											62					69	71	B62, -	*15:07:01-15:07:02, 15:126, 15:207							
								57								69	71	B75/B62, B75/62, Null, -	*15:08, 15:11:01-15:11:03, 15:11:05, 15:15, 15:148, 15:191, 15:209N, 15:215							
																	71	B70	*15:09							
																	71	B71, B70, -	*15:10:01, 15:37, 15:90, 15:99, 15:133, 15:229, 15:252							
																	71	B71	*15:10:02							
								57								69	71	B75	*15:11:04							
											62					69	71	B76, -	*15:12, 15:14, 15:19, 15:270							
								57									70	B77, -	*15:13:01-15:13:02							
									59								70	B63, -	*15:16:01-15:16:02, 15:67, 15:95, 15:254							
50									59								70	-	*15:16:03							
																	70	B63, -	*15:17:01:01-15:17:02, 15:162, 15:168, 15:177, 15:196, 15:208, 15:216, 15:273							
																		71	B71, B70, Null, -	*15:18:01-15:18:02, 15:18:04-15:18:05, 15:51-15:52, 15:72, 15:93, 15:108, 15:114, 15:119, 15:134, 15:153, 15:176, 15:197-15:198, 15:200, 15:221, 15:226N, 15:238, 15:263, 15:275						
														67			71	B71	*15:18:03							
50											62						71	B62	*15:20							
								57									71	B75	*15:21, 15:44							
																	70	B70/B5/b lank	*15:23, 15:115							
											62					69	70	B62, B15, -	*15:24:01-15:24:02, 15:87, 15:157							
											62						71	B62, -	*15:25:01-15:25:03, 15:204, 15:240, 15:250, 15:271							
																	71	B70	*15:29							
											62						69	71	B62, -	*15:30, 15:58, 15:63, 15:269						
								57									71	B75	*15:31							
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
												62										70			-	*15:36
									59			62	64										71		B15	*15:42
												62									69	70			B15, -	*15:43, 15:256
												62									69	71			B62	*15:45
												61	62								69	71			-	*15:46
												62										71			B62, -	*15:48, 15:150, 15:188, 15:248
												61	62								69	71			-	*15:53, 15:212
												62									69	71			B72	*15:54
								57														71			B15	*15:55
				w								62									69	71			B62	*15:57

			52				56											w		71	B35/22	*35:76			
							55											68		71	-	*35:81, 35:96, 35:109, 35:157			
		51																68		71 72	-	*35:87			
			53															68		71	-	*35:115			
										61										71	-	*35:154			
																		68 69		71	-	*35:163, 35:213			
																		68		71	-	*35:164			
													64					68 69		71	-	*35:185			
		52					56													71	-	*35:186			
																				71	-	*35:191			
																				71	-	*35:198			
		50									62									71	-	*35:207			
																			68		71	-	*35:218		
																			68			-	*35:222		
		52					56												68		71	-	*35:229		
				53																	70	B37, Null, -	*37:01:01-37:01:07, 37:01:09, 37:03N, 37:06, 37:13, 37:15-37:21, 37:23-37:27, 37:30N-37:33N, 37:36		
											62										70	B37, -	*37:01:08, 37:09		
											62										70	B37, -	*37:01:08, 37:09		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
																								ser. ⁵	HLA-B allele ⁴
													62								70	B37/B27		*37:02	
				53		55							62								70	B37, -		*37:04:01-37:04:02	
				53									62								71	Blank		*37:05	
													62								70 72	-		*37:07	
				53														68		70		B37		*37:08	
				53									62								70	B37, -		*37:10, 37:22, 37:29	
				53									62								71	-		*37:11, 37:37	
				53									62								70	B37		*37:12	
				53									62								70 w	-		*37:14	
				53									62								70	-		*37:28	
				53							60		62								70	-		*37:34	
				53									62								70	-		*37:35	
																					70	B38, Null, -		*38:01:01-38:02:02, 38:02:04-38:02:05, 38:08-38:09, 38:11-38:16, 38:18-38:19, 38:21, 38:23-38:24, 38:27-38:29, 38:31-38:32, 38:34N, 38:38-38:43	
																					70	-		*38:02:03	
												62									70	B38		*38:03	
												62									70	B38, -		*38:04, 38:25	
																					70	B38, -		*38:05, 38:33	
																					68	B38		*38:06	
																					68	B38		*38:07	
																					70	B38, -		*38:10, 38:36-38:37	
																					70	-		*38:17	
																					70	-		*38:20	

49	?							58	60		62						70			-		*40:117				
49	?		53								61	62						71		-		*40:132, 40:178				
	?	51									61	62						71		-		*40:136				
49												62						71	72	-		*40:137				
49											61	62						71		-		*40:138				
	?								59		61	62						71		-		*40:148, 40:161				
	?											62					70			-		*40:157, 40:201				
	?										61	62						71		-		*40:158				
	?										61	62						71		-		*40:159				
49	?		52								61							71		-		*40:166				
49											61	62						71		-		*40:170				
	?										61	62						71		-		*40:174				
49	?										61	62						71		-		*40:183, 40:198				
	?										61	62						71		-		*40:184				
49												62						71		-		*40:185				
49	?											62						71		-		*40:208				
												62						71		-		*40:209				
49											61	62						71		-		*40:215				
49	?	51		53							61	62						71		-		*40:231				
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
		51		54								61	62									71		B41, -	*41:01, 41:06-41:07, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25	
		51										61	62									71	72	B41, -	*41:02:01-41:02:05, 41:11, 41:13, 41:19, 41:23, 41:27	
		51										61	62									71		B41, -	*41:03:01-41:03:02, 41:15	
		51										61	62									71	72	B41, -	*41:04, 41:10, 41:18	
		51		54								?	62									71		B41	*41:05	
		51										61	62									71		B41	*41:08	
				54								61	62									71		B45, -	*41:09, 45:02-45:03 ¹⁰	
		51										61	62									71	72	-	*41:24	
		51		54									62											-	*41:26	
		51	52																				71	72	B42, -	*42:01:01-42:02, 42:06, 42:10, 42:12, 42:15, 42:17-42:18
		51	52		54																		71		B42	*42:04
		51	52																				71	72	B42	*42:05:01-42:05:02
			52																				71		B42	*42:08
		51	52																				71	72	B42	*42:09
		51											62										71	72	-	*42:11
		51	52	53																			71	72	-	*42:13
		51	52																				71		-	*42:14
		51	52																				71	72	-	*42:16, 42:20
			52																				71	72	-	*42:19

			52			55										69	71	-		*46:17						
			52			55						64					71	-		*46:18						
			52			55			w							69	71	-		*46:32						
	50		52			55											71	-		*46:33						
	50						56					62					70	B47, -		*47:01:01:01-47:01:02, 47:06-47:08						
	50						56				61	62					71	B47		*47:02						
	50						56				w	62					70	w	B47	*47:03						
	50											62					70		B17, -	*47:04-47:05						
								57				62					71	72	B48, -	*48:01:01-48:01:04, 48:07, 48:09, 48:11, 48:19-48:20, 48:22, 48:27, 48:29-48:30						
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.
																									ser. ⁵	HLA-B allele ⁴
													62									71	B48/B70, -		*48:02:01-48:02:03, 48:25	
49													62									71	B48		*48:03:01-48:03:02	
								57					62									71	B48, -		*48:04, 48:24	
													62									71	72	B48, -		*48:05, 48:08
								57														71	72	-		*48:06
													62									71	72	B48		*48:10, 48:12
													62									71		-		*48:13
													62									71	72	B48		*48:14
								57					62									71	72	B48		*48:15
								57														71	72	B48		*48:16
49													62									71	72	-		*48:17
								57					62								70	72	B48			*48:18
													62									71		-		*48:21, 48:26
49													62									71		-		*48:23
								57					62									71	72	-		*48:28
									58	59			62		65							70		B49, Null, -		*49:01:01, 49:01:03-49:01:04, 49:06, 49:08-49:10, 49:12-49:17, 49:19N, 49:22-49:24
									58				62		65							70	B49, -		*49:01:02, 49:11	
										59			62		65							70	B49			*49:02
										59			62		65							70	B49			*49:03
									58	59			62									70	B49			*49:04-49:05
									58	59			62									70	-			*49:07
									58	59			62									70	-			*49:18
									58	59			62		65							70	-			*49:20
									58	59			62		65								-			*49:21
									58	59			62		65							70	-			*49:25
										59					65							70	-			*49:26
										59		61	62		65							71	B50, -		*50:01:01-50:01:03, 50:04-50:05, 50:08, 50:10-50:11, 50:13, 50:16, 50:18-50:19, 50:32	
										59		61	62		65							71	B45			*50:02
										59			62		65							71	-			*50:06
										59		61	62		65							71	-			*50:07

										59	61 62									71	-	*50:09, 50:20			
											61 62			65							71	-	*50:12		
										59	61 62										71	-	*50:14		
										59	61			65							71	-	*50:15		
											61 62			65							71	-	*50:17		
										59	61 62										71	-	*50:31		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
																								ser. ⁵	HLA-B allele ⁴
					54					59	60								68		70			B51, B5102/B51, B5103/B51, B51/blank, Null -	*51:01:01, 51:01:03, 51:01:05-51:01:08, 51:01:10-51:01:20, 51:01:22-51:01:24, 51:01:26-51:01:39, 51:01:41-51:02:01, 51:02:03, 51:02:05-51:03, 51:07:01, 51:09:01, 51:11N-51:12, 51:13:02-51:14, 51:16-51:19, 51:21-51:23, 51:26-51:33, 51:38-51:41N, 51:43, 51:48-51:53, 51:55, 51:57-51:58, 51:60, 51:65-51:67, 51:69-51:77, 51:79-51:80, 51:82-51:88, 51:90-51:91, 51:94-51:96, 51:98N, 51:100, 51:102, 51:105, 51:107-51:111, 51:113-51:115, 51:117, 51:120-51:130, 51:133-51:134, 51:136-51:138, 51:140, 51:142, 51:145-51:146, 51:149N-51:152
										59	60								68		70			B51, B5102/B51, -	*51:01:02, 51:02:02, 51:05, 51:07:02, 51:09:02, 51:89, 51:103, 51:116, 51:119, 51:132
					54					59	60								68		70			B51, B5102/B51, -	*51:01:04, 51:02:04, 51:13:01, 51:24:01, 51:34-51:35, 51:92, 51:99, 51:131, 51:143
										59	60								68		70			B51	*51:01:09
					54					59	60								68		70			B51, -	*51:01:21, 51:01:25, 51:24:03-51:24:04
										59									68		70			-	*51:01:40, 51:54, 51:78:01-51:78:02
					54	55					60								68		70			B51, -	*51:04, 51:46, 51:56:01-51:56:02, 51:139
					54						60								68		70			B51, -	*51:06:01-51:06:02, 51:59, 51:135
					54					59	60								68		70			B51, Null, -	*51:08, 51:20, 51:44N, 51:141
					54						60								68		70			B51	*51:10
											60				65				68		70			-	*51:15
					54					59	60								68		70			B51	*51:24:02
					54					59	60								68		70			B51	*51:36
					54					59	60								68		70			B51, -	*51:37, 51:63
				53	54						60								68		70			-	*51:42

52				56				63 64								71				B54, Null, -				*54:01:01, 54:01:03-54:02, 54:05N, 54:07-54:08N, 54:10, 54:13, 54:16-54:19, 54:21-54:25, 54:27-54:28							
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.							
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.							
																ser. ⁵								HLA-B allele ⁴							
52				56				64				71				B55, B54, -				*54:01:02, 55:01:07, 55:02:01-55:02:06, 55:07, 55:10, 55:12, 55:16, 55:19, 55:26, 55:30, 55:35, 55:37, 55:39, 55:41-55:43, 55:47-55:48, 55:50, 55:57 ¹¹											
52				56				63				65				71				B22				*54:03							
52				56				63								71				B54, -				*54:04, 54:15							
52				56				63								69				71				-				*54:06			
52				56				63												71				-				*54:09, 54:14			
52				56				63												71				B54				*54:11			
52				56				58				63 64								70				-				*54:12			
52				56				59				63 64								71				-				*54:20			
52				56				63												71				-				*54:26			
52				56				59				64				71				B55, Null, -				*55:01:01-55:01:06, 55:01:08-55:01:10, 55:03, 55:05, 55:11, 55:15, 55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:44-55:45, 55:52-55:55N							
52				56												71				B55, B56, Null, -				*55:04, 55:08, 55:13, 55:27, 55:46, 55:49, 56:01:05, 56:15, 56:19N, 56:22 ¹²							
52				56				59								71				B22, B55				*55:09, 55:24							
52				56								66				71				-				*55:14							
52				56				59								71				B55, -				*55:17, 55:28, 55:58							
								62				64				71				-				*55:18							
51		52		54		56						71				B22				*55:20											
52				56				w				64				69				71				B55				*55:21			
52				56												71				B22				*55:22							
52				56												71				B55, -				*55:23, 55:32, 56:18, 56:31-56:32 ¹³							
52				56				62				64				71				-				*55:34							
52				56				59				62				64				71				-				*55:40			
52				56												71				-				*55:51							
52		54		56						64				71				-				*55:56									
52				56								65				71				B56, Null, -				*56:01:01-56:01:04, 56:08, 56:14, 56:16, 56:20:01, 56:24, 56:26-56:30, 56:34-56:36, 56:38N							
52				56								65				67				71				B56				*56:02, 56:04			
52				56												69				71				B22/15				*56:03			
52				56				59				64				71				B56				*56:05:01							

			52				56			59										71	B56	*56:05:02			
			52				56			59			64			w				71	B78	*56:06			
			52				56							65					70		B56	*56:07			
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	Well No.	
																								ser. ⁵	HLA-B allele ⁴
			52				56														71	B56	*56:09		
			52				56						64				67				71	B55	*56:10		
			52				56														71	B56, B22	*56:11-56:12		
			52		54		56									65					71	B56	*56:13		
																65					71	B56, -	*56:17, 56:33		
			52				56									65					71	-	*56:20:02		
			52				56			58	59										70		-	*56:21	
															64						71		-	*56:23	
			52				56			59						65					71		-	*56:25, 56:37	
			52				56	w								65					71		-	*56:39	
											60						66				70	B57, -	*57:01:01-57:01:04, 57:01:06-57:01:14, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:22-57:23, 57:25-57:27, 57:29-57:30, 57:33-57:38, 57:40-57:41, 57:43-57:44, 57:48-57:50, 57:52-57:56, 57:58, 57:60-57:62, 57:64		
											60						66				70	B57, -	*57:01:05, 57:11, 57:13, 57:21, 57:31, 57:47		
											60						66				70	B57, Null, -	*57:02:01-57:03:02, 57:17, 57:28N, 57:32, 57:39, 57:42, 57:46, 57:57, 57:63		
											60						66				70	B57	*57:04		
		50									60						66				70	B57	*57:05		
				53							60						66				70	B57	*57:07		
				53							60						66				70	B57	*57:09		
																	66				71	B57	*57:12		
						55					60						66				70	-	*57:14		
				53							60						66				70	-	*57:24		
49											60						66				70	-	*57:59		
											60							67			70	B58, Null, -	*58:01:01-58:01:02, 58:01:04-58:01:12, 58:04-58:05, 58:10N-58:15, 58:19, 58:21-58:24, 58:29, 58:31N-58:33, 58:35, 58:37, 58:39N-58:40, 58:42		
											60							67			70	B58, -	*58:01:03, 58:02, 58:06, 58:16, 58:25-58:26, 58:38		
											60							67			70	B58	*58:07		
				54						59	60							67			70	B17/B5	*58:08:01		
										59	60							67			70	-	*58:08:02		
					55						60							67			70	B17	*58:09		
																		67			70	Null	*58:17N		
											60							67			70	-	*58:18		

