

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01-01:02:12, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:72	DR1, Null, -	73																							
*01:03, 01:39N	DR103/DR1, Null		74																						
*01:42, 01:61	-	73	74																						
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:113-03:114, 03:116-03:118, 03:121-03:124, 03:127	DR17, DR3, Null, -					77	78											89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92	DR17, -					77	78																		
*03:01:24, 03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					77													89						
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119	DR18, DR3, -					77	79												89						
*03:08, 03:65	DR3, -					77	78					83		85											
*03:10	DR3					77	78													91	92				
*03:42, 03:69, 03:87, 03:111	-					77																			
*03:76	-					77	78					84				87		89							
*03:112	-					77	78					83						89							
*03:125	-					77	78					84				87	88	89							
*03:126, 13:193	-					78	79											89	91						
*04:01:01-04:05:11, 04:05:13-04:14, 04:16-04:35, 04:37-04:46, 04:48-04:52, 04:54-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:81N, 04:83-04:104, 04:106-04:121, 04:123-04:135, 04:137-04:145, 04:147-04:205, 04:207	DR4, DR4/DR3, Null, -								80																
*04:15, 04:36, 04:47, 04:53:01-04:53:02, 04:82, 04:136, 04:206	DR4, -								80					85											
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								80										90						
*07:01:01-01:07:01:18, 07:03-07:68N	DR7, Null, -									81															
*08:01:01-08:01:05, 08:02:01-08:02:04, 08:04:01-08:04:07, 08:06-08:07, 08:11, 08:16-08:17, 08:22, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79	DR8, Null, -										82						88								
*08:01:06, 08:03:02-08:03:06, 08:10, 08:12-08:15, 08:19, 08:23, 08:27, 08:30:01-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:46, 08:48-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:69, 08:71, 08:74, 08:76	DR8, Null, -										82														
*08:05, 08:24, 08:31, 08:41, 08:75, 11:67, 11:193	DR8, DR8/DR11, -										82			85			88								
*08:08	DR8										82						88		90						
*08:09	DR8										82						88			92					
*08:18, 08:25, 08:29, 08:47, 08:63	DR8, -										82			85											
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164 ⁴	DR13, -																87	88	89		92				
*08:21	DR8										82						87	88			92				
*08:32, 08:53	-										82				86						92				
*08:35, 08:36:02	-										82										92				
*08:40	-						78				82														
*08:57	-																	88							
*08:67	-									81							88								
*08:68	-										82										w				
*08:70	-										82						88				w				
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:22, 09:24-09:27	DR9, -											83							91						
*09:01:06, 09:03, 09:23	DR9, -											83													
*10:01:01-10:01:09, 10:03-10:10, 10:13-10:16	DR10, -										82		84												
*10:02, 10:12	-												84												
*10:11	-									81	82														

Negative Control

