

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
DRB1 allele ^{1,2}	ser ³																									
*01:01:01-01:01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62-01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:110	DR1, Null, -	73																								
*01:03:01-01:03:04, 01:39N, 01:99, 01:102	DR103/DR1, Null, -	74																								
*01:42, 01:61, 01:89	-	73	74																							
*03:01:01-01:03-01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:29, 03:04:01-03:06, 03:09, 03:11-01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:149, 03:151-03:153, 03:158-03:161, 03:163-03:166, 03:168-03:170	DR17, DR3, Null, -					77	78										89									
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130, 03:156N	DR17, Null, -					77	78																			
*03:01:24, 03:07:01-03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:155	DR3, -					77												89								
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131, 03:154, 03:167	DR18, DR3, -					77	79											89								
*03:08, 03:65, 03:140	DR3, -					77	78				83	85														
*03:10	DR3					77	78												91	92						
*03:42, 03:69, 03:87, 03:111, 03:162	-					77																				
*03:76	-					77	78				84				87		89									
*03:125	-					77	78				84				87	88	89									
*03:126, 14:179 ⁴	-					78	79				84						89	91								
*03:157	-					77	78										89	92								
*04:01:01-01:04-05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:250, 04:252-04:264N, 04:266N-04:283, 04:285-04:290, 04:292-04:301	DR4, DR4/DR3, Null, -								80																	
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:251, 04:284	DR4, -								80									90								
*04:211	-								80										91							
*04:265, 04:291	-								80				85													
*07:01:01-01:07-01:23, 07:03-07:05, 07:07-07:111	DR7, Null, -									81																
*07:06	DR7									81	82															
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86, 08:88-08:90, 08:93-08:94, 08:96, 08:99-08:100	DR8, Null, -										82						88									
*08:01:06, 08:03:02:01-08:03:10, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97	DR8, Null, -										82															
*08:03:11, 08:68:01	-										82									w						
*08:08	DR8										82						88	90								
*08:09:01	DR8										82						88			92						
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
*08:09:02, 14:15 ⁴	-, DR8										82						88	89		92						
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164, 13:232, 13:286 ⁴	DR13, -															87	88	89		92						
*08:21	DR8										82						87	88		92						
*08:31, 11:05, 11:209 ⁴	DR8/DR11, -												85				88									
*08:32, 08:53	-										82			86						92						
*08:35, 08:36:02	-										82									92						
*08:40	-						78				82															
*08:41, 08:75, 11:67, 11:193:01-11:193:02 ⁴	-										82			85			88									
*08:57, 08:80, 08:98, 13:248 ⁴	-													85			88									
*08:63	-										82			85												
*08:67	-									81							88									
*08:70	-										82						88			w						
*08:87	-										82			86			88			w						
*08:92	-						78			81	82						88									
*09:01:02:01-09:01:05, 09:01:07-09:02:02, 09:04, 09:06-09:22, 09:24-09:28, 09:30-09:44	DR9, Null, -											83							91							
*09:01:06, 09:03, 09:23, 09:29	DR9, -											83														
*09:05	DR9											83	84						91							
*10:01:01-01-10:10, 10:12-10:18, 10:20-10:36	DR10, -												84													
*10:11	-									81			84													
*10:19	-											84											94			

Negative Control

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*13:01:01-13:02:01:05, 13:02:03-13:02:17, 13:04, 13:08, 13:16, 13:20, 13:22:01-13:24, 13:28:01-13:29, 13:31-13:32, 13:34-13:36, 13:38-13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:02-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143, 13:145, 13:147-13:149, 13:153, 13:155, 13:162, 13:166, 13:168, 13:171:02, 13:173, 13:177, 13:182, 13:184-13:187, 13:190, 13:200N-13:202, 13:204-13:205, 13:207-13:209, 13:212-13:213, 13:215, 13:218, 13:221, 13:223, 13:225-13:226, 13:230, 13:233-13:234, 13:236-13:245, 13:247, 13:249N-13:252N, 13:254-13:276, 13:278Q-13:283, 13:285, 13:287-13:288, 13:290-13:294	DR13, DR11, DR14, Null, -						78									87	88	89							
*13:02:02	DR13						78											89							
*13:03:01-13:03:09, 13:10, 13:33:02-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94:02-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N, 13:151-13:152, 13:167, 13:170, 13:172, 13:174, 13:178, 13:180, 13:188, 13:194, 13:196, 13:216, 13:219, 13:227, 13:253	DR13, DR13, Null, -						78										87		89						
*13:06, 13:12:01-13:12:04, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:110, 13:118, 13:134, 13:157, 13:161, 13:163, 13:181, 13:220, 13:224, 13:231, 13:295N	DR13, DR11, DR6, Null, -															87		89							
*13:09, 13:183, 13:192, 13:210-13:211, 13:222	DR13, -																88	89							
*13:13, 13:119, 13:154, 13:156, 14:84, 14:116, 14:144 ⁴	DR13, -															87		89			92				
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*13:15, 13:19, 13:53, 13:57, 13:104, 13:198	DR13, -						78	79								87	88	89							
*13:17, 13:116, 13:175, 13:214	DR13, -						78			81							88								
*13:26:01-13:26:02	DR14, -							79									87	88	89						
*13:27:01-13:27:02, 13:41, 13:71, 13:277	DR13, -						78						84			87	88	89							
*13:33:01, 13:94:01	DR13, -					77	78									87		89							
*13:43, 13:159, 13:179, 13:191, 13:217	DR13, -						78									87	88			91	92				
*13:44, 13:86, 13:206, 14:183 ⁴	-							79										89							
*13:45	DR13						78									87	88		90	92					
*13:61:02, 13:96:01	-					77	78									87	88	89							
*13:67, 13:103, 13:160, 13:195	DR13, -															87	88								
*13:85	-						78	79								87		89							
*13:122	-						78									87									
*13:129	-						78						84			88	89								
*13:144	-						78									87	88	89			92				
*13:146	-															88	89			92					
*13:165	-						78									87	88								
*13:171:01	-						78				82					87	88			91	92				
*13:176	-						78						84			87		89							
*13:193	-						78	79										89		91					
*13:197	-										82					87	88			90	92				
*13:228	-															87									
*13:235, 14:98 ⁴	-							79								87		89							
*13:289N	Null						?									?		89							
*14:01:01, 14:01:05, 14:04:01:01, 14:04:04-14:04:06, 14:07:01, 14:10, 14:54:01:01-14:54:01:04, 14:54:03, 14:54:05-14:54:06, 14:54:08, 14:57, 14:137N, 14:152N, 14:191-14:192, 14:196-14:197N, 14:199, 14:201-14:202, 14:204-14:205, 14:207, 14:210Q, 14:213-14:216, 14:219	DR14, DR1404/DR14, DR4/DR6, Null, -																	89	90	91	92				
*14:01:02, 14:01:04, 14:04:01:02-14:04:03, 14:07:02, 14:26, 14:28, 14:31, 14:32:03, 14:35, 14:38:01, 14:39, 14:54:01:05, 14:54:09-14:55, 14:60-14:62, 14:70-14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99, 14:101, 14:107, 14:110, 14:112-14:114, 14:117-14:118, 14:124-14:125, 14:129, 14:131, 14:138-14:140, 14:142-14:143, 14:145, 14:147, 14:149-14:151, 14:153, 14:157-14:158, 14:163-14:164, 14:166N, 14:172, 14:180, 14:182, 14:184, 14:189-14:190, 14:203	DR14, Null, -																	?	90	91	92				
*14:01:03, 14:23:02, 14:34, 14:72, 14:92N, 14:97, 14:126:01-14:126:02, 14:130, 14:133, 14:148, 14:162, 14:175, 14:185	DR14, Null, -																	?		91	92				
*14:02:01:01-14:02:07, 14:06:01-14:06:04, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94, 14:106, 14:108, 14:121, 14:159, 14:165, 14:170, 14:176-14:177, 14:194-14:195N, 14:198, 14:209, 14:212	DR14, DR6, Null, -							79										89		91					
*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85, 14:102, 14:115, 14:135, 14:156, 14:174, 14:181, 14:218	DR1403/DR14, DR6, -						79									87		89			92				
*14:04:07, 14:54:07	-																		90	91	92				
*14:05:01:01-14:05:05, 14:08, 14:11, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36, 14:43-14:44:03, 14:54:02, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100, 14:103, 14:123, 14:127:01-14:127:02, 14:134, 14:155, 14:167, 14:169, 14:187-14:188N, 14:193, 14:206, 14:211	DR14, Null, -																	89		91	92				
*14:16	DR6						78									87	88	?	90	91	92				
*14:18, 14:81, 14:154, 14:160	DR14, -							79										89		91	92				

Negative Control

Ne

