

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52-01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:402, 01:404-01:408	A1, Null, -																								
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:391, 01:396Q, 01:403	A1, Null, -	1																							
*01:01:56, 01:361N	Null, -	1			4								12												
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4							11		13		15									
*01:15N	Null	1			w																				
*01:21, 01:126, 01:360	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 01:380:01-01:01:380:01:02, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4							11													
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4							11				15									
*01:43, 11:271 ⁴	-	1			4							10													
*01:51	-	1			4				8			10													
*01:121	-	1			4															19					
*01:194	-	1			4																				
*01:298	-	1			4					9															
*01:301Q	-	1			4	5		7																	
*01:344, 01:381	-	1			4				8																
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24-01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:1039	A2, A203, A210, Low A2, Null, -				2																				
*02:17:02:01-02:17:04	A2, -		2					w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907	A2, -		2											13							20		22		
*02:35:02	A2													13							20		22		
*02:46, 02:70	A2		2																		20		22		
*02:48, 02:129, 02:571	A2, -																				20		22		
*02:55	A2, A28		2						8													21		23	
*02:65, 02:407, 02:449, 02:932	A2, A19, -		2																	19					
*02:78	-		w		4																20		22		
*02:81, 02:124	A2		2							9															
*02:135	-		2											13											24
*02:149:01-02:149:02, 02:842, 02:947	-		2																			21			
*02:169	-		2		4																				
*02:237, 02:809, 02:927	-		2													14		16							23
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:243:01-02:243:03	-		2																		18		21		
*02:309	-		2										12		14							21			24
*02:338, 02:952	-		2	3																					
*02:454	-		2										12		14										24
*02:527, 02:582	-		2						8																
*02:576, 02:682	-		2																				22		
*02:644	-								8		10												21		
*02:651	-		2																			20			
*02:671	-		2																						23
*02:741	-		2						8					13								20	21	22	

*36:01:01:01-36:01:02, 36:03, 36:05-36:12	A36, -	1																		22						
*36:02	A36	1	3									14								22 23						
*36:04	A36	1		4																22						
*43:01-43:03	A43, Null, -									10	12	13														
*66:01:01:01-66:01:06, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41	A66, A26, Null, -							8			11	13														
*66:02:01:01-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 66:43	A66, A10, Null, -							8				12								24						
*66:06	A66							8			11	13								21						
*66:09	-							8			11	13 14								21						
*66:10	-							8			11	w														
*66:12	-							8				13								24						
*66:19	-							8			11 12	13														
*68:01:01:01-68:04:02, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:226, 68:228-68:229, 68:231-68:248, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:285	A68, A28, Null, -							8												20						
*68:05, 68:15, 68:20, 68:109, 68:136, 68:255	A68, -							8												20 23						
*68:08:02, 68:105:01:01-68:105:01:02, 68:157, 68:276Q	A68, -							8												20 21						
*68:11N, 68:41, 68:230	Null, -							8												20 22						
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8												20					
*68:19, 68:130:02	A68, -								8					14							20					
*68:25, 68:262	A68, -								8												19 20					
*68:29	A68								8												18	20 21				
*68:45, 68:117	-							7	8													20				
*68:71	-								8					13								20				
*68:84, 68:227	-								8		10											20				
*68:103:01-68:103:02	-				3				8													20				
*68:130:01	-								8				12									20				
*68:178	-				4																	20				
*68:218, 68:258	-								8																	
*68:249	-					5			8													20				
*68:250	-																				18	20 21				
*68:271	-								8			11										20				
*69:01:01:01-69:01:04, 69:03-69:06, 69:08	A69, -								8														21			
*69:02	-								8			11											21			
*69:07	-								8					13									21			
*74:01:01:01-74:01:02, 74:01:04-74:03:02, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38-74:41	A74, Null, -																									
*74:01:03	-												12												24	
*74:04, 74:21	A74, -																					18 19				
*74:07, 74:33	A74, -																					17	19			
*74:13	-													13		16						19				
*74:19	-				4																	19				
*74:23	-				3																	19				
*74:30	-											11										19				
*74:35	-																					19	21			
*74:37	-													13								19				
*80:01:01:01-80:01:02, 80:03, 80:05-80:09N	A80, Null, -						6																		23	
*80:02	-						6					11														
*80:04	-						6															17			23	
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -									9																
B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569, C*07:990, C*07:994	-																								22	
B*18:27	-							6																		
C*03:365, C*12:131	-				4																					
C*06:187	-																								23	
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	