

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:126	DR1, Null, -	73																							
*01:03:01-01:03:05, 01:39N, 01:99, 01:102, 01:112	DR1, DR103, Null, -		74																						
*01:42, 01:61, 01:89	-	73	74																						
*03:01:01:01-03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:37, 03:04:01-03:06, 03:09, 03:11:01, 03:11:03-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:149, 03:151-03:153, 03:158-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180-03:184, 03:186-03:187, 03:191-03:192	DR3, DR17, Null, -						77	78										89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:111, 03:130, 03:156N, 03:162, 03:189N	DR17, Null, -						77	78																	
*03:01:24, 03:07:01-01-03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:155, 03:171, 03:178, 03:185, 03:188	DR3, -						77												89						
*03:02:01:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131, 03:154, 03:167, 03:176, 03:179	DR3, DR18, -						77	79											89						
*03:08, 03:65, 03:140	DR3, -						77	78			83	85													
*03:10	DR3						77	78												91	92				
*03:42, 03:69, 03:87, 03:173, 03:190	-						77																		
*03:76, 03:174N	Null, -						77	78			84				87			89							
*03:125	-						77	78			84				87	88		89							
*03:126, 13:193 ⁴	-						78	79										89		91					
*03:157	-						77	78										89			92				
*03:177	-						77	78	79									89		91					
*04:01:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:161, 04:163-04:210, 04:212N-04:250, 04:252-04:283, 04:285-04:336	DR4, DR3, Null, -								80																
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:162, 04:251, 04:284:01-04:284:02	DR4, -								80											90					
*04:211	-								80											91					
*07:01:01:01-07:01:01:02, 07:01:01-04-07:01:28, 07:03-07:05, 07:07-07:127, 07:129N-07:133	DR7, Null, -									81															
*07:06	DR7									81	82														
*07:128	-									81										90					
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:04:01-01-08:07, 08:11:01-01-08:11:01:02, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86, 08:88-08:90, 08:93-08:94, 08:96, 08:99-08:101, 08:103, 08:105-08:107, 08:109, 08:111	DR8, Null, -										82						88								
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96

Negative control

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*08:01:06, 08:03:02:01-08:03:10, 08:03:12, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97, 08:102, 08:104, 08:110	DR8, Null, -										82														
*08:03:11, 08:68:01	-										82										w				
*08:08	DR8										82						88		90						
*08:09:01	DR8										82						88				92				
*08:09:02, 14:15 ⁴	DR8, -										82						88	89			92				
*08:20, 13:18, 13:47, 13:55, 13:158:01-13:158:02, 13:164, 13:232, 13:286 ⁴	DR13, -															87	88	89			92				
*08:21	DR8										82						87	88			92				
*08:31, 11:05, 11:209 ⁴	DR8, DR11, -													85			88								
*08:32, 08:53	-										82				86						92				
*08:35, 08:36:02	-										82										92				
*08:40	-							78			82														
*08:41, 08:75, 11:67, 11:193:01-11:193:02 ⁴	-										82			85			88								
*08:57, 08:80, 08:98	-																88								
*08:63	-										82			85											
*08:67	-									81							88								
*08:70	-										82						88				w				
*08:87	-										82				86		88				w				
*08:92	-							78		81	82						88								
*08:108	-									81	82						88								
*09:01:02:01-09:01:05, 09:01:07-09:02:02, 09:04-09:22, 09:24-09:28, 09:30-09:51	DR9, Null, -											83								91					
*09:01:06, 09:03, 09:23, 09:29	DR9, -											83													
*10:01:01:01, 10:01:01:03-10:10, 10:12-10:18, 10:20-10:43	DR10, -												84												
*10:11	-									81		84													
*10:19	-											84											94		
*11:01:01:01, 11:01:01:03-11:01:17, 11:01:20-11:01:27, 11:01:29-11:01:45, 11:04:01:01-11:04:16, 11:04:18-11:04:20, 11:04:22, 11:06:01-11:06:03, 11:09:01:01-11:10:02, 11:12:01-11:12:03, 11:15:01:01-11:15:02, 11:24:01-11:24:02, 11:27:01, 11:28:01:01-11:29:02, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43:01:01-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62:02, 11:66:01-11:66:02, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117:02, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N-11:170, 11:172, 11:174-11:175, 11:177-11:181, 11:183, 11:185-11:190, 11:192, 11:194:01-11:201, 11:203-11:208, 11:210-11:212, 11:214-11:228, 11:230, 11:232, 11:234-11:236, 11:238-11:240, 11:243-11:247, 11:249-11:250N, 11:252-11:255, 11:257-11:259, 11:262-11:263, 11:265, 11:269, 11:271-11:275, 11:277-11:279, 11:281-11:282, 11:284-11:285, 11:287N, 11:289, 11:291-11:292, 11:294N-11:298	DR11, Null, -													85		87	88								
*11:01:18-11:01:19, 11:04:21, 11:08:01-11:08:03, 11:18-11:19:03, 11:42:01-11:42:02, 11:57, 11:130, 11:191, 11:248Q, 11:268, 11:280, 11:286	DR11, -													85		87									
*11:01:28	-													85		?	88								
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*11:02:01:01-11:03:03, 11:11:01, 11:11:03, 11:14:01:01-11:14:02, 11:16:01:01-11:16:01:02, 11:20-11:21, 11:36, 11:40-11:41, 11:48, 11:63:01-11:63:02, 11:65:01-11:65:02, 11:68, 11:70, 11:76, 11:85-11:86, 11:93, 11:118:01-11:118:02, 11:122, 11:124:01, 11:127, 11:138-11:139, 11:151, 11:161, 11:168, 11:171, 11:176, 11:213, 11:231, 11:233, 11:237, 11:241-11:242, 11:251, 11:256, 11:260, 11:276, 11:290	DR11, DR13, -						78							85		87	88								
*11:03:04	-						78							85		87	88	89							
*11:04:17, 11:149	-													85		87	88	89							
*11:07:01-11:07:02, 11:103:01-11:103:02, 11:105, 11:107, 11:125, 11:173	DR11, -					77					83			85											
*11:13:01-11:13:02	DR11						w							85				89		91	92				
*11:17, 11:202	DR14, -													85				89		91	92				
*11:22, 11:53, 11:98, 11:104, 11:143, 11:261	DR11, -													85											
*11:23:01-11:23:02, 11:25, 11:96, 11:148	DR11, -													85		87	88				92				
*11:26, 11:34, 11:267	DR11, -							79						85											
*11:27:02-11:27:03, 11:84:02-11:84:03	DR11, -					77								85		87	88								
*11:30, 11:288	DR11, -				76									85		87	88								

Negative control

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*14:120, 14:122, 14:178	-						78											?	90	91	92				
*14:132	-						78											?		91	92				
*14:141	-																	89				93			
*14:168	-																	?	w	91	92				
*14:179	-						78	79				84						89		91					
*14:200	-							79													91	92			
*14:208	-																			90		92			
*14:217	-																	89	90	91	92		94		
*14:222N	Null						79												?			92			
*14:235	-																	87	89	90	91	92			
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:157, 15:159N-15:166, 15:168-15:186, 15:188-15:198	DR15, -, DR2, Null			75																					
**15:21	DR2			75																		w			
*15:25, 15:37:01, 15:100, 15:104:01	-			75		77																			
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-			75																	91				
*15:141	-			75																			94		
*15:158, 15:187	-			75																	91	w			
*15:167	-			75																90					
*16:01:01:16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:65, 16:67-16:71	DR16, Null, -				76																				
*16:04:01-16:04:02, 16:18	DR16, -				76																	w			
*16:66	-			75	76																				
DRB3*01:01:02:01-01:41, DRB3*01:43-01:99, DRB3*02:01-02:26, DRB3*02:28-02:110, DRB3*02:112-02:130, DRB3*02:132-02:159, DRB3*02:161-02:175, DRB3*03:01:01:01-03:21, DRB3*03:23-03:55	DR52, Null, -																						93		
DRB3*01:42	-											83											93		
DRB3*02:27	-																				87		93		
DRB3*02:131	-																				87		92		
DRB3*02:160	-																				87	88		93	
DRB4*01:01:01:01, DRB4*01:01:01:03-01:03:01:01, DRB4*01:03:01:03-01:03:01:12, DRB4*01:03:01:14-01:13, DRB4*01:15-01:30, DRB4*01:32-01:154, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																							94	
DRB4*01:03:01:02N, DRB4*01:03:01:13N, DRB4*01:14N	Null																				90			94	
DRB4*01:31	-																					92		94	
DRB5*01:01:01:01-01:117, DRB5*02:02:01-02:30	DR51, Null, -																								95

Negative control