

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:01:156, 01:01:158-01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:402, 01:404-01:417N, 01:419-01:426, 01:428-01:438, 01:440-01:463, 01:465, 01:467-01:478	A1, Null, -		1		4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:01:157, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:391, 01:396Q, 01:403, 01:418N, 01:439	A1, Null,-		1																						
*01:01:56, 01:361N	Null, -	1			4							12													
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	Undefined,	1		3																					
*01:13, 01:176	A1, -	1			4							11		13		15									
*01:15N	Null	1			w																				
*01:21, 01:126, 01:360	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 01:380:01:01-01:380:01:02, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4							11													
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4							11			15										
*01:43, 11:271 ⁴	-	1			4							10													
*01:51	-	1			4				8			10													
*01:121	-	1			4															19					
*01:194	-	1			4									13		15									
*01:298	-	1			4					9								17							
*01:301Q	-	1			4	5		7																	
*01:344, 01:381	-	1			4				8																
*01:427, 36:02 ⁴	A36, -	1		3											14								22	23	
*01:464, 36:04 ⁴	A36, -	1			4																		22		
*01:466	-	1		3																				23	
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:1049, 02:1051-02:1094, 02:1096-02:1170, 02:1172-02:1206	A2, Low A2, A203, A2, A210, A2, Null, A2, blank, -		2																						
*02:17:02:01-02:17:04	A2, -		2					w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907, 02:1171	A2, -		2											13							20		22		
*02:35:02	A2													13							20		22		
*02:46, 02:70, 02:1050, 02:1095	A2, -		2																		20		22		
*02:48, 02:129, 02:571	A2, -																				20		22		
*02:55	A2, A28		2						8													21		23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:81, 02:124	A2		2							9															
*02:135	-		2											13											24
*02:149:01-02:149:02, 02:842, 02:947:01-02:947:02	-		2																			21			
*02:169	-		2		4																				
*02:237, 02:809, 02:927	-		2												14		16							23	
*02:243:01-02:243:03	-		2																18			21			
*02:309	-		2										12		14							21			24
*02:454	-		2										12		14										24
*02:527, 02:582	-		2						8																
*02:576, 02:682	-		2																				22		
*02:644	-								8		10											21			
*02:651	-		2																			20			
*02:671	-		2																					23	
*02:741	-		2						8					13							20	21	22		
*02:804	-		2					7																	
*02:815	-		2						8														21		
*02:829	-				4					9															
*02:929	-		2																18						

*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:42, 03:01:44-03:01:46, 03:01:48-03:01:50, 03:01:52-03:01:54, 03:01:56-03:01:58, 03:01:60-03:01:62, 03:01:64-03:01:66, 03:01:68-03:01:70, 03:01:72-03:01:74, 03:01:76-03:01:78, 03:01:80-03:01:82, 03:01:84-03:01:86, 03:01:88-03:01:90, 03:01:92-03:01:94, 03:01:96-03:01:98, 03:02:00-03:02:02, 03:02:04-03:02:06, 03:02:08-03:02:10, 03:02:12-03:02:14, 03:02:16-03:02:18, 03:02:20-03:02:22, 03:02:24-03:02:26, 03:02:28-03:02:30, 03:02:32-03:02:34, 03:02:36-03:02:38, 03:02:40-03:02:42, 03:02:44-03:02:46, 03:02:48-03:02:50, 03:02:52-03:02:54, 03:02:56-03:02:58, 03:02:60-03:02:62, 03:02:64-03:02:66, 03:02:68-03:02:70, 03:02:72-03:02:74, 03:02:76-03:02:78, 03:02:80-03:02:82, 03:02:84-03:02:86, 03:02:88-03:02:90, 03:02:92-03:02:94, 03:02:96-03:02:98, 03:03:00-03:03:02, 03:03:04-03:03:06, 03:03:08-03:03:10, 03:03:12-03:03:14, 03:03:16-03:03:18, 03:03:20-03:03:22, 03:03:24-03:03:26, 03:03:28-03:03:30, 03:03:32-03:03:34, 03:03:36-03:03:38, 03:03:40-03:03:42, 03:03:44-03:03:46, 03:03:48-03:03:50, 03:03:52-03:03:54, 03:03:56-03:03:58, 03:03:60-03:03:62, 03:03:64-03:03:66, 03:03:68-03:03:70, 03:03:72-03:03:74, 03:03:76-03:03:78, 03:03:80-03:03:82, 03:03:84-03:03:86, 03:03:88-03:03:90, 03:03:92-03:03:94, 03:03:96-03:03:98, 03:04:00-03:04:02, 03:04:04-03:04:06, 03:04:08-03:04:10, 03:04:12-03:04:14, 03:04:16-03:04:18, 03:04:20-03:04:22, 03:04:24-03:04:26, 03:04:28-03:04:30, 03:04:32-03:04:34, 03:04:36-03:04:38, 03:04:40-03:04:42, 03:04:44-03:04:46, 03:04:48-03:04:50, 03:04:52-03:04:54, 03:04:56-03:04:58, 03:04:60-03:04:62, 03:04:64-03:04:66, 03:04:68-03:04:70, 03:04:72-03:04:74, 03:04:76-03:04:78, 03:04:80-03:04:82, 03:04:84-03:04:86, 03:04:88-03:04:90, 03:04:92-03:04:94, 03:04:96-03:04:98, 03:05:00-03:05:02, 03:05:04-03:05:06, 03:05:08-03:05:10, 03:05:12-03:05:14, 03:05:16-03:05:18, 03:05:20-03:05:22, 03:05:24-03:05:26, 03:05:28-03:05:30, 03:05:32-03:05:34, 03:05:36-03:05:38, 03:05:40-03:05:42, 03:05:44-03:05:46, 03:05:48-03:05:50, 03:05:52-03:05:54, 03:05:56-03:05:58, 03:05:60-03:05:62, 03:05:64-03:05:66, 03:05:68-03:05:70, 03:05:72-03:05:74, 03:05:76-03:05:78, 03:05:80-03:05:82, 03:05:84-03:05:86, 03:05:88-03:05:90, 03:05:92-03:05:94, 03:05:96-03:05:98, 03:06:00-03:06:02, 03:06:04-03:06:06, 03:06:08-03:06:10, 03:06:12-03:06:14, 03:06:16-03:06:18, 03:06:20-03:06:22, 03:06:24-03:06:26, 03:06:28-03:06:30, 03:06:32-03:06:34, 03:06:36-03:06:38, 03:06:40-03:06:42, 03:06:44-03:06:46, 03:06:48-03:06:50, 03:06:52-03:06:54, 03:06:56-03:06:58, 03:06:60-03:06:62, 03:06:64-03:06:66, 03:06:68-03:06:70, 03:06:72-03:06:74, 03:06:76-03:06:78, 03:06:80-03:06:82, 03:06:84-03:06:86, 03:06:88-03:06:90, 03:06:92-03:06:94, 03:06:96-03:06:98, 03:07:00-03:07:02, 03:07:04-03:07:06, 03:07:08-03:07:10, 03:07:12-03:07:14, 03:07:16-03:07:18, 03:07:20-03:07:22, 03:07:24-03:07:26, 03:07:28-03:07:30, 03:07:32-03:07:34, 03:07:36-03:07:38, 03:07:40-03:07:42, 03:07:44-03:07:46, 03:07:48-03:07:50, 03:07:52-03:07:54, 03:07:56-03:07:58, 03:07:60-03:07:62, 03:07:64-03:07:66, 03:07:68-03:07:70, 03:07:72-03:07:74, 03:07:76-03:07:78, 03:07:80-03:07:82, 03:07:84-03:07:86, 03:07:88-03:07:90, 03:07:92-03:07:94, 03:07:96-03:07:98, 03:08:00-03:08:02, 03:08:04-03:08:06, 03:08:08-03:08:10, 03:08:12-03:08:14, 03:08:16-03:08:18, 03:08:20-03:08:22, 03:08:24-03:08:26, 03:08:28-03:08:30, 03:08:32-03:08:34, 03:08:36-03:08:38, 03:08:40-03:08:42, 03:08:44-03:08:46, 03:08:48-03:08:50, 03:08:52-03:08:54, 03:08:56-03:08:58, 03:08:60-03:08:62, 03:08:64-03:08:66, 03:08:68-03:08:70, 03:08:72-03:08:74, 03:08:76-03:08:78, 03:08:80-03:08:82, 03:08:84-03:08:86, 03:08:88-03:08:90, 03:08:92-03:08:94, 03:08:96-03:08:98, 03:09:00-03:09:02, 03:09:04-03:09:06, 03:09:08-03:09:10, 03:09:12-03:09:14, 03:09:16-03:09:18, 03:09:20-03:09:22, 03:09:24-03:09:26, 03:09:28-03:09:30, 03:09:32-03:09:34, 03:09:36-03:09:38, 03:09:40-03:09:42, 03:09:44-03:09:46, 03:09:48-03:09:50, 03:09:52-03:09:54, 03:09:56-03:09:58, 03:10:00-03:10:02, 03:10:04-03:10:06, 03:10:08-03:10:10, 03:10:12-03:10:14, 03:10:16-03:10:18, 03:10:20-03:10:22, 03:10:24-03:10:2
--

[illegible]

[illegible]

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*34:02:01:01-34:02:07, 34:03, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24-34:26N, 34:28-34:30N, 34:32-34:33	A34, Null, -			3					8			11	12												
*34:02:08	-			3					8			11													
*34:04	A34			3					8			11	12		14										
*34:08	A34			3					8			11	12	13											
*34:09	-			3					8				12											24	
*34:22	-			3					8			11	12									22			
*34:31	-					5			8			11	12												
*34:34, 66:02:01:01-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 66:43, 66:48 ⁴	A66, A10, Null, -								8				12											24	
*36:01:01:01-36:01:02, 36:03, 36:05-36:15	A36, -	1																				22			
*43:01-43:03	A43, Null, -										10		12	13											
*66:01:01:01-66:01:07, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41, 66:44-66:47, 66:49	A66, A26, Null, -								8			11		13											
*66:06	A66								8			11		13								21			
*66:09	-								8			11		13	14							21			
*66:10	-								8			11		w											
*66:12	-								8					13											24
*66:19	-								8			11	12	13											
*68:01:01:01-68:01:66, 68:01:68-68:04:02, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:226, 68:228-68:229, 68:231-68:248, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:293Q, 68:295-68:321, 68:323-68:326	A68, A28, Null, -								8												20				
*68:01:67, 68:218, 68:258	-								8																
*68:05, 68:15, 68:20, 68:109, 68:136, 68:255	A68, -								8												20			23	
*68:08:02, 68:105:01:01-68:105:01:02, 68:157, 68:276Q	A68, -								8												20	21			
*68:11N, 68:41, 68:230	Null, -								8												20		22		
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8												20				
*68:19, 68:130:02	A68, -								8					14							20				
*68:25, 68:262	A68, -								8										19	20					
*68:29	A68								8										18	20	21				
*68:45, 68:117, 68:322	-						7	8													20				
*68:71	-								8					13							20				
*68:84, 68:227	-								8		10										20				
*68:103:01-68:103:02	-			3					8												20				
*68:130:01	-								8				12								20				
*68:178	-				4																20				
*68:249	-					5			8												20				
*68:250	-																		18	20	21				
*68:271	-								8			11									20				
*68:294	-								8				12								20				24
*69:01:01:01-69:01:06, 69:03-69:06, 69:08-69:09	A69, -								8													21			
*69:02	-								8			11										21			
*69:07	-								8					13								21			
*74:01:01:01-74:01:02, 74:01:04-74:03:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38-74:42, 74:45-74:47N, 74:49-74:52	A74, Undefined, Null, -																			19					
*74:01:03	-												12								19				24
*74:04, 74:21, 74:43	A74, -																			18	19				
*74:07, 74:33, 74:48	A74, -																			17	19				
*74:13	-													13		16					19				
*74:19	-				4																19				
*74:23	-				3																19				
*74:30	-											11									19				
*74:35	-																				19	21			
*74:37	-													13							19				
*74:44	-															16					19				
*80:01:01:01-80:01:02, 80:03, 80:05-80:10	A80, -, Null					6																		23	
*80:02	-					6						11													
*80:04	-					6													17					23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
B*07:81, B*08:52, B*18:67, B*38:41, 38:67, B*49:67, B*51:01:110, 51:185, B*53:05:01-53:05:02, 53:16, 53:33, B*57:01:38, 57:60	-, B53									9															
B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569, 07:990, 07:994	-																					22			
B*07:507, C*06:187	-																						23		
B*15:01:83, B*35:01:60, B*58:01:24	-			w																					
B*18:27	-					6																			
C*12:131	-				4																				
Allele	Sero.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24