

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ^{1,2}	ser ³																							
*01:01:01-01:02:08, 01:04-01:38, 01:40N-01:41, 01:43-01:60	DR1, Null, -	65																						
*01:03, 01:39N	DR103/DR1, Null	66																						
*01:42	-	65	66																					
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:22, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93	DR17, DR3, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92	DR17, -					69	70																	
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90	DR18, DR3, -					69	71												81					
*03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41, 03:49	DR3, -					69													81					
*03:08, 03:65	DR3, -					69	70				75		77											
*03:10	DR3					69	70													83	84			
*03:42, 03:69, 03:87	-					69																		
*03:76	-					70						76			79			81						
*04:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:164	DR4, DR4/DR3, Null, -								72															
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								72											82				
*07:01:01-01:07:01:07, 07:03-07:28	DR7, Null, -									73														
*08:01:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54	DR8, -										74						80							
*08:03:02-08:03:03, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:49, 08:51	DR8, -										74													
*08:08	DR8										74						80		82					
*08:09	DR8										74						80				84			
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164 ⁴	DR13, -															79	80	81			84			
*08:21	DR8										74					79	80				84			
*08:31, 08:41, 11:67 ⁴	DR8/DR11, -										74		77			80								
*08:32, 08:53	-										74			78							84			
*08:35, 08:36:02	-										74										84			
*08:40	-					70					74													
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:20, 09:22	DR9, -											75								83				
*09:01:06, 09:03	DR9											75												
*10:01:01-10:06	DR10, -												76											
*11:01:01-11:01:17, 11:01:20, 11:04:01-11:04:11, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:30, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01-11:84:03, 11:88, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147	DR11, -													77		79	80							
*11:01:18-11:01:19, 11:08:01-11:08:02, 11:18-11:19:03, 11:42, 11:57, 11:130	DR11, -													77		79								
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*11:02:01-11:03, 11:11:01-11:11:02, 11:14:01-11:14:02, 11:16, 11:20-11:21, 11:36, 11:40-11:41, 11:48, 11:63, 11:65:01-11:65:02, 11:68, 11:70, 11:76, 11:85-11:86, 11:93, 11:118, 11:122, 11:124, 11:127, 11:138-11:139	DR11, DR11/DR13, -					70								77		79	80							
*11:05	DR11													77		80								
*11:07, 11:53, 11:103, 11:105, 11:107, 11:125	DR11, -					69						75		77										
*11:13:01-11:13:02	DR11						w							77				81		83	84			
*11:17, 11:52	DR14, DR11													77						83	84			
*11:22, 11:98, 11:104, 11:136, 11:143	-													77										
*11:23:01-11:23:02, 11:25, 11:96, 11:148	DR11, -													77		79	80					84		
*11:26, 11:34	DR11							71						77										
*11:31, 11:45, 11:64, 11:119	DR11, -													77		79						84		
*11:55	DR11													77		80						84		
*11:59, 11:80, 11:83, 11:87, 11:135, 11:142	DR11, -					70						76		77		79	80							
*11:69, 11:82	-													77		80				82				
*11:73, 11:79, 11:131-11:132	-					70								77		79								
*11:89	-													77										84
*11:149	-													77		79	80	81						

*14:50	DR14							73						?	82	83	84							
*14:52	DR14													?	82	83	84							
*14:58	DR14													?	82	83	84							
*14:65	DR6					w								81		83	84							
*14:68, 14:93	DR14, -							74						?	82	83	84							
*14:69	-												79	?	82	83	84							
*14:73	-												80	?	82	83	84							
*14:74	-												79	?	82	83	84							
*14:82	-					70								81	82	83	84							
*14:89	-					71								81		83	84							
*14:95	-					70								81		83	84							
*14:98	-					71							79	81		83	84							
*14:132	-					70								?	82	83	84							
*14:137N	Null					70							79	81	82	83	84							
*14:141	-													81				85						
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:65, 15:67-15:100	DR15, DR2, Null, -				67																			
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*15:21	DR2				67																w			
*15:25	-				67	69																		
*15:27, 15:34, 15:66	-				67															83				
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:24	DR16, Null, -				68																			
*16:04, 16:18	DR16, -				68																w			
DRB3*01:01:02:01-01:15, DRB3*02:01-02:26, 02:28-02:29N, DRB3*03:01:01-03:03	DR52, Null, -																						85	
DRB3*02:27	-													79									85	
DRB4*01:01:01:01-01:03:01:01, DRB4*01:03:01:03-01:08	DR53, -																							86
DRB4*01:03:01:02N	Null																		82					86
DRB5*01:01:01:01-14, DRB5*02:02-02:06	DR51, Null, -																							87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87