



*02:644	-							7	9											20		
*02:671	-		2																	21		
*02:741	-		2					7			12								19	20		
*02:804	-		2					6														
*02:815	-		2					7												20		
*02:829	-				4				8													
*02:929	-		2																17			
Well No.	ser. <sup>3</sup>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	ser. <sup>3</sup>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*03:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:82, 03:01:84-03:01:93, 03:01:95-03:07:02, 03:09-03:11N, 03:13-03:17:02, 03:19-03:23:02, 03:25-03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-01-03:49, 03:51-01-03:56, 03:58, 03:60-03:62:01:02, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:103:01, 03:104-03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197:02N, 03:201-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:231-02-03:245, 03:247-03:248, 03:250-03:251, 03:253-03:259, 03:261-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:321, 03:323N-03:329N, 03:331-03:342N, 03:344-03:346, 03:348-03:376, 03:378-03:381N, 03:383-03:394, 03:396-03:397, 03:399-03:400N	A3, Null, -			3									12									
*03:01:19, 03:103:02	-			3								11	12									
*03:01:23, 03:01:83, 03:01:94, 03:08:01:01-03:08:01:02, 03:36N, 03:57, 03:59, 03:72, 03:89:01-03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205, 03:211, 03:252, 03:267, 03:330N, 03:398, 24:92, 24:458 <sup>5</sup>	A3, Null, -			3																		
*03:01:38, 03:123:02, 03:171, 03:231:01	-			3									12	13								
*03:12	A3			3	4																	
*03:18, 03:135	-	1		4									12									
*03:24, 03:50	A3, -			3				7					12									
*03:32, 03:152, 03:219, 03:273	-			3												16						
*03:41, 03:382	-			3																		21
*03:43, 03:82, 03:186	-			3									12	14	16							
*03:63	-			3							10	12										21
*03:75, 03:177, 03:377	-			3								12										21
*03:88	-			3	4						10											21
*03:95, 03:343	-												12	13	15							21
*03:187, 03:215, 03:225, 03:249, 03:322, 25:46 <sup>5</sup>	-												12									
*03:246, 03:282	-			3									12							18		
*03:347	-							6					12									21
*03:395	-	1		3									12	14								
*11:01:01-01-11:02:06, 11:04-11:09, 11:12:01-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q-11:281, 11:283-11:289, 11:291-11:305, 11:307-11:325, 11:327-11:347N, 11:349-11:365N, 11:367-11:371, 11:373-11:380, 11:382N-11:385	A11, Null, -			4									10									
*11:03, 11:20, 11:25:01-11:25:02, 11:60, 11:158, 11:175, 11:209, 11:348, 11:366, 11:381	A11, -			3	4								10									
*11:10, 11:191, 11:269	A11, -			4				7					10									
*11:11, 34:19 <sup>5</sup>	-			4									10	11								
*11:17, 11:40, 11:223	-			4						9			10									
*11:113, 11:162	-			4									10		14							
*11:116, 11:140, 11:199:01-11:199:02, 11:222, 66:23 <sup>5</sup>	-			4									10	12								
*11:130	-			3	4								10		13							21
*11:139	-			4				6					10									
*11:166	-			4		5							10									
*11:183	-			3	4			7					10									
*11:199:03	-			4									10	12							19	
*11:306	-			4									10			15						
*11:372	-			4									10								18	
*23:01:01-01-23:13, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:98, 23:100-23:103N, 23:106N-23:110, B*18:27 <sup>5</sup>	A23, Null, -							5														
*23:14:01-23:14:02, 23:105, 24:24, 24:71, 24:315, 24:392 <sup>5</sup>	A23, A9, A24, -							5	6													
*23:57	-																					20
*23:64	-							5								16						
*23:104, 24:02:01-01-24:11N, 24:13:01-24:13:02, 24:17:01:01-24:17:01:02, 24:20:01:01-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56:02, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:103, 24:105-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:290, 24:292-24:295, 24:297-24:303N, 24:305-24:314, 24:317-24:323N, 24:325-24:391, 24:393-24:411, 24:413-24:457, 24:459-24:470, 24:472-24:480, 24:482-24:514N, 24:516-24:519 <sup>5</sup>	A24, A2403, A9, Null, -							6														
Well No.	ser. <sup>3</sup>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	ser. <sup>3</sup>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*24:18, 24:204, 24:213	A24, A3, -							6														21
*24:19, 24:44	A9							6			10											
*24:82	-							6												17		20
*24:104, 24:243	-							6								16						









