

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ^{1,2}	ser ³																							
*01:01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:120	DR1, Null, -	65																						
*01:03:01-01:03:05, 01:39N, 01:99, 01:102, 01:112	DR103, DR1, Null, -		66																					
*01:42, 01:61, 01:89	-	65	66																					
*03:01:01:01-03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:33, 03:04:01-03:06, 03:09, 03:11:01, 03:11:03-03:16, 03:18-03:20, 03:22-03:23, 03:25-01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:149, 03:151-03:153, 03:158-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180-03:181	DR17, DR3, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:111, 03:130, 03:156N, 03:162	DR17, Null, -					69	70																	
*03:01:24, 03:07:01-01:03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:155, 03:171, 03:178	DR3, -					69													81					
*03:02:01:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131, 03:154, 03:167, 03:176, 03:179	DR18, DR3, -					69	71												81					
*03:08, 03:65, 03:140	DR3, -					69	70				75		77											
*03:10	DR3					69	70														83	84		
*03:42, 03:69, 03:87, 03:173	-					69																		
*03:76, 03:174N	Null, -					69	70					76			79			81						
*03:125	-					69	70					76			79	80		81						
*03:126, 13:193 ⁴	-					70	71											81		83				
*03:157	-					69	70											81			84			
*03:177	-					69	70	71										81		83				
*04:01:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:161, 04:163-04:210, 04:212N-04:250, 04:252-04:283, 04:285-04:317	DR4, DR3, Null, -								72															
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:162, 04:251, 04:284	DR4, -								72										82					
*04:211	-								72											83				
*07:01:01:01-07:01:25, 07:03-07:05, 07:07-07:120	DR7, Null, -									73														
*07:06	DR7									73	74													
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:04:01-08:07, 08:11:01-08:11:01:02, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86, 08:88-08:90, 08:93-08:94, 08:96, 08:99-08:101, 08:103, 08:105-08:106	DR8, Null, -										74							80						
*08:01:06, 08:03:02:01-08:03:10, 08:03:12, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97, 08:102, 08:104	DR8, Null, -										74													
Well No.	ser. ³	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	ser. ³	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*08:03:11, 08:68:01	-										74													
*08:08	DR8										74						80		82			w		
*08:09:01	DR8										74						80				84			
*08:09:02, 14:15 ⁴	DR8, -										74						80	81			84			
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164, 13:232, 13:286 ⁴	DR13, -															79	80	81			84			
*08:21	DR8										74					79	80				84			
*08:31, 11:05, 11:209 ⁴	DR8, DR11, -													77			80							
*08:32, 08:53	-										74				78						84			
*08:35, 08:36:02	-										74										84			
*08:40	-						70				74													
*08:41, 08:75, 11:67, 11:193:01-11:193:02 ⁴	-										74			77			80							
*08:57, 08:80, 08:98	-																80							
*08:63	-										74			77										
*08:67	-									73							80							
*08:70	-										74						80				w			
*08:87	-										74				78		80				w			
*08:92	-						70			73	74						80							

