

*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:82, 03:01:84-03:01:93, 03:01:95-03:07:02, 03:09-03:11N, 03:13-03:17:02, 03:19-03:23:02, 03:25-03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-01-03:49, 03:51-01-03:56:01:02, 03:58, 03:60-03:62:01:02, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:103:01, 03:104-03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197:02N, 03:201-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:231-02-03:245, 03:247-03:248, 03:250-03:251, 03:253-03:259, 03:261-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:321, 03:323N-03:329N, 03:331-03:342N, 03:344-03:346, 03:348-03:376, 03:378-03:381N, 03:383-03:394, 03:396-03:397, 03:399-03:412, 03:415-03:427, 03:430-03:448N	A3, Null, -																					
*03:01:19, 03:103:02	-			3									11	12								
*03:01:23, 03:01:83, 03:01:94, 03:08:01:01-03:08:01:02, 03:36N, 03:57, 03:59, 03:72, 03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205, 03:211, 03:252, 03:267, 03:330N, 03:398, 03:413, 24:92, 24:458, 24:551, 24:557 ⁵	A3, Null, -			3																		
*03:01:38, 03:123:02, 03:171, 03:231:01, 03:428	-			3									12	13								
*03:12	A3				3	4																
*03:18, 03:135	-		1			4								12								
*03:24, 03:50	A3, -			3					7					12								
*03:32, 03:152, 03:219, 03:273, 03:429	-			3													16					
*03:41, 03:382	-			3																21		
*03:43, 03:82, 03:186	-			3								12	14	16								
*03:63	-			3						10		12								21		
*03:75, 03:177, 03:377	-										12									21		
*03:88	-			3	4					10										21		
*03:89:01	-				2	3																
*03:95, 03:343	-											12	13	15						21		
*03:187, 03:215, 03:225, 03:249, 03:322, 03:414, 25:46, 25:77, 25:81-25:82 ⁵	-											12										
*03:246, 03:282	-			3								12					18					
*03:347	-								6			12								21		
*03:395	-	1		3								12	14									
*11:01:01:01-11:02:07, 11:04-11:07:01, 11:08-11:09, 11:12:01-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208:02N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q-11:281, 11:283-11:289, 11:291-11:305, 11:307-11:325, 11:327-11:347N, 11:349-11:365N, 11:367-11:371, 11:373-11:380, 11:382N-11:428, 24:555 ⁵	A11, Null, -					4						10										
*11:03:01:01, 11:20, 11:25:01-11:25:02, 11:60, 11:158, 11:175, 11:209, 11:348, 11:366, 11:381	A11, -			3	4							10										
*11:07:02, 11:372	-				4							10						18				
*11:10, 11:191, 11:269	A11, -				4				7			10										
*11:11, 34:19 ⁵	-				4							10	11									
*11:17, 11:40, 11:223	-				4					9	10											
*11:113, 11:162	-				4							10			14							
*11:116, 11:140, 11:199:01-11:199:02, 11:222, 66:23 ⁵	-				4						10	12										
*11:130	-			3	4							10		13						21		
*11:139	-				4				6			10										
*11:166	-				4	5						10										
*11:183	-			3	4				7			10										
*11:199:03	-				4							10	12						19			
*11:306	-				4							10			15							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*23:01:01:01-23:10:02, 23:12-23:13, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:98, 23:100-23:103N, 23:106N-23:112, 23:114-23:124, B*18:27 ⁵	A23, Null, -					5																
*23:11N, 31:29, 31:85 ⁵	Null, -					5										15						
*23:14:01-23:14:02, 23:104-23:105, 24:24, 24:71, 24:315, 24:392, 24:527, 24:569 ⁵	A23, A24, A9, -					5	6															
*23:57	-																				20	
*23:64	-					5											16					
*23:113N	Null					6										15						
*24:02:01:01-24:11N, 24:13:01-24:13:02, 24:17:01:01-24:17:01:02, 24:20:01:01-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56:02, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:103, 24:105-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:290, 24:292-24:295, 24:297-24:303N, 24:305-24:314, 24:317-24:323N, 24:325-24:391, 24:393-24:411, 24:413-24:457, 24:459-24:470, 24:472-24:480, 24:482-24:514N, 24:516-24:526, 24:528-24:550, 24:552-24:554, 24:556N, 24:558-24:568N, 24:570-24:573, 24:575-24:578	A24, A2403, A9, Low A24, Null, -					6																
*24:18, 24:204, 24:213	A24, A3, -					6																21
*24:19, 24:44	A9					6					10											
*24:82	-					6												17			20	
*24:104, 24:243	-					6										16						

*25:01:01:01-25:02, 25:04, 25:07-25:16, 25:18-25:29, 25:31-25:33, 25:35, 25:37-25:42N, 25:44-25:45, 25:47-25:49N, 25:51, 25:53, 25:55-25:58, 25:60-25:64, 25:66-25:70, 25:72-25:76, 25:79-25:80	A25, Null, -						7	8			12											
*25:03	A25						7	8			12			16								
*25:05	A25						7	8			11	12							19			
*25:06	A25						7	8			11											
*25:17, 25:34, 25:54, 25:71	-							8				12										
*25:30	-						7	8				12			16					20		
*25:36, 25:50, 25:52, 25:65	-						7	8														
*25:43	-				4		7	8														
*25:59	-	1			4				9										19			
*25:78, 26:57, 66:12, 66:15 ⁵	-						7					12										
*26:01:01:01-26:01:47, 26:01:49-26:01:75, 26:08:01:01-26:08:03, 26:10-26:15, 26:17-26:18, 26:20:01-26:20:02, 26:23-26:28, 26:31-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:56, 26:58-26:63, 26:65-26:71N, 26:73-26:74, 26:76-26:77, 26:79-26:88, 26:90, 26:93-26:110, 26:112:01-26:119, 26:121, 26:123-26:151, 26:153-26:176, 26:178-26:183, 26:185-26:192, 26:194-26:197, 26:199-26:203, 26:206:01N-26:206:02N, 26:208-26:216, 26:218-26:223	A26, A10, Null, -						7			9		12										
*26:01:48, 26:07:01-26:07:02, 26:64, 26:89	A26, -								9			12										
*26:02:01-26:02:02, 26:29, 26:49	A26, -						7		9		w											
*26:03:01:01-26:03:01:02, 26:06, 26:21, 26:111	A26, -						7			10		12								21		
*26:04, 26:75, 26:205	A26, -						7		9													
*26:05, 26:30	A26						7					12								21		
*26:09, 26:91	A26, -						7		9		11											
*26:16	A26					6			9			12										
*26:19	-				4				9			12	13									
*26:22	A26						7		9			12	13							20		
*26:54	-						7		9		11	12								19		
*26:72, 26:184	-				4				9			12										
*26:78	-						7			10										21		
*26:92, 66:42 ⁵	-									10		12										
*26:120	-	1					7		9			12										
*26:122	-						7		9			12							19			
*26:152, 26:193	-						7		9	10		12							17	20		
*26:177	-									10		12								21		
*26:198	-						7		9			12	13									
*26:204:01:01-26:204:01:02	-				4		7		9													
*26:207	-								9											17		
*26:217	-						7		9		11	12										
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*29:01:01:01-29:06, 29:08N-29:12, 29:15-29:27, 29:29-29:38, 29:40-29:47, 29:50-29:66, 29:68-29:79, 29:81-29:97, 29:99-29:104, 29:106-29:154, 29:157-29:163, 29:165-29:171	A29, Null, -													13								
*29:07, 29:49	A29, -						5							13								
*29:13, 29:39, 29:98	-													13		16						
*29:14, 29:172	-													13		15						
*29:28	-							7						13								
*29:48	-													13					17			
*29:67, 29:156	-				4									13								
*29:80, 29:155	-													13							20	
*29:105, 29:164	-									9				13						17		
*30:01:01:01-30:04:04, 30:06-30:07, 30:09:01:01-30:12:02, 30:14L-30:15, 30:17-30:20, 30:23-30:43, 30:45, 30:47-30:54, 30:56-30:88, 30:90-30:124, 30:126-30:139, 30:141-30:149, 30:151-30:162, 30:164-30:176, 30:178N-30:185, 30:187-30:203, B*07:260, C*12:328 ⁵	A30, Null, -																				14	
*30:08:01-30:08:02, 30:150	A30, -					4															14	
*30:13, 30:16, 30:22, 30:44, 30:46	A30, -		2																		14	
*30:55, 30:186	-												12								14	
*30:89, 30:177	-																				14	
*30:125, 30:140	-										10										14	
*30:163	-						6														14	
*31:01:02:01-31:02:02, 31:05-31:07, 31:09-31:20, 31:22-31:23, 31:26-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:84, 31:86-31:108, 31:110-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:199, 31:201N-31:212	A31, Null, -																				15	
*31:03-31:04:02	A31, -			w								11		13		15						
*31:08, 31:109, 33:53, 33:125, 33:131 ⁵	A31, A24, -																				15	
*31:21	-													15		16						
*31:24-31:25	-												12								15	
*31:35	-													14		15	16					
*31:41	-													15							20	
*31:123	-										11		13		15							
*31:176, 31:179	-				3																15	
*31:200	-	1																			15	
*32:01:01:01-32:02, 32:07-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:59, 32:61-32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:127, 32:129-32:141, 32:143-32:151, 32:153-32:155, 32:157-32:168N	A32, Null, -								8												16	
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																				16	
*32:04, 32:142	A32, A3, -				3				8												16	
*32:05, 32:79	A32, -														15	16	17					
*32:06	A32								8												20	
*32:15	A32							7	8												16	
*32:26:01-32:26:02, 32:152	-								8				12								16	
*32:42	-								8					13							16	

*32:64	-			4				8								16							
*32:72	-						5	8								16							
*32:93	-							8								16	17						
*32:128	-							8							15	16							
*32:156	-			1				8								16							
*33:01:01:01-33:01:16, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:178, 33:180-33:183, 33:185-33:186, 33:188-33:189, 33:191-33:200, 33:202-33:230N	A33, Null, -																		17				
*33:13, 33:48	-								9											17			
*33:19, 33:119	-						6													17			
*33:22	-																			17	20		
*33:24	-																			17	21		
*33:49, 33:144, 33:179, 33:201	-			w																17			
*33:51	-							7												17			
*33:61, 33:190	-											12								17			
*33:184	-										10									17	21		
*33:187	-						5													17			
*34:01:01:01-34:01:07, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 34:27, 66:40 ⁵	A34, A10, -							7		10	11												
*34:02:01:01-34:03, 34:07, 34:10N, 34:13, 34:15, 34:20-34:22, 34:24-34:26N, 34:28-34:30N	A34, Null, -			3				7		10	11												
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
*34:04	A34			3				7		10	11		13										
*34:08	A34			3				7		10	11	12											
*34:09	-			3				7		11													
*36:02	A36	1		3										13								21	
*43:01-43:03	A43, Null, -									9	11	12											
*66:01:01:01-66:01:07, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41, 66:44-66:46	A66, A26, Null, -							7		10	12												
*66:02:01:01-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 66:43	A66, A10, Null, -							7		11													
*66:06	A66							7		10	12										20		
*66:09	-							7		10	12	13									20		
*66:10	-							7		10	w												
*66:19	-							7		10	11	12											
*68:01:01:01-68:04:02, 68:06-68:08:01:02, 68:09-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:226, 68:228-68:249, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:293Q	A68, A28, Null, -							7													19		
*68:05, 68:15, 68:20, 68:109, 68:136, 68:255	A68, -							7													19	21	
*68:08:02, 68:105:01:01-68:105:01:02, 68:157, 68:276Q	A68, -							7													19	20	
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -			4				7													19		
*68:19, 68:130:02	A68, -							7				13									19		
*68:25, 68:262	A68, -							7													18	19	
*68:29	A68							7													17	19	20
*68:45, 68:117	-						6	7													19		
*68:71	-							7				12									19		
*68:84, 68:227	-							7		9											19		
*68:103:01-68:103:02	-			3				7													19		
*68:130:01, 68:294	-							7			11										19		
*68:178	-				4																19		
*68:218, 68:258	-							7															
*68:250	-																				17	19	20
*68:271	-							7		10											19		
*69:01:01:01-69:01:04, 69:03-69:06, 69:08-69:09	A69, -							7														20	
*69:02	-							7		10												20	
*69:07	-							7			12											20	
*74:01:01:01-74:01:02, 74:01:04-74:03:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38-74:42	A74, Null, -																					18	
*74:01:03	-										11											18	
*74:04, 74:21	A74, -																				17	18	
*74:07, 74:33	A74, -																16					18	
*74:13	-										12			15								18	
*74:19	-				4																	18	
*74:23	-				3																	18	
*74:30	-										10											18	
*74:35	-																					18	20
*74:37	-											12										18	
*80:01:01:01-80:01:02, 80:03, 80:05-80:09N	A80, Null, -						5															21	
*80:02	-						5			10													
*80:04	-						5									16						21	
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -								8														
B*35:01:60, B*58:01:24	-			w																			
C*03:365, C*12:131	-				4																		
C*06:187	-																					21	
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	