





Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*11:264, 13:05:01:01-13:05:03, 13:07:01-13:07:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:21:01:01-13:21:02, 13:42:01-13:42:02, 13:46, 13:49:01-13:50:03, 13:62, 13:100, 13:108, 13:132, 13:136, 13:150, 13:169, 13:189, 13:199, 13:203, 13:229, 13:246, 13:284, 13:302 <sup>4</sup>	DR11, DR13, DR6, -															79	80	81						
*11:293, 13:45 <sup>4</sup>	DR13, -					70										79	80		82	84				
*12:01:01:01-12:01:11, 12:03:02-12:04, 12:06-12:12, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36, 12:38, 12:40-12:41, 12:46-12:48, 12:53-12:54, 12:58-12:59, 12:62-12:63, 12:68, 12:70, 12:72N, 12:77, 12:79, 12:82-12:84, 12:86N-12:87, 12:89-12:91, 12:95	DR12, Null, -															78								
*12:02:01:01-12:02:12, 12:13, 12:18-12:19:02, 12:21, 12:23, 12:26-12:27, 12:31N-12:33, 12:37, 12:42-12:45, 12:49-12:52, 12:55-12:56, 12:60N-12:61, 12:64-12:65, 12:67, 12:69, 12:71, 12:74N-12:76, 12:78, 12:80-12:81, 12:85, 12:92-12:94, 12:97-12:98N	DR12, Null, -															78		80						
*12:05	DR12															78								85
*12:15	DR12															78	80							85
*12:16:01-12:16:03	-									74						78	80							
*12:20	-															78	80	81						
*12:22	-									73	74					78								
*12:39:01-12:39:02	-									74						78								
*12:57, DRB3*02:111 <sup>4</sup>	-				68												79	80						85
*12:66	-															78				83				
*12:73	-					70				73						78	80							
*12:88	-									73						78								
*12:96	-															78								83 84
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*13:01:01:01-13:02:01:12, 13:02:03-13:02:21, 13:04, 13:08, 13:16:01:01-13:16:01:02, 13:20:01:01-13:20:01:03, 13:22:01:01-13:24:01:03, 13:28:01-13:29:01:02, 13:31-13:32, 13:34-13:36, 13:38-13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:02-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117:01:01-13:117:01:02, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143, 13:145, 13:147-13:149, 13:153, 13:155, 13:162, 13:166, 13:168, 13:171:02, 13:173, 13:177, 13:182, 13:184-13:187, 13:190, 13:200N-13:202, 13:204-13:205, 13:207-13:209, 13:212-13:213, 13:215, 13:218, 13:221, 13:223, 13:225-13:226, 13:230, 13:233-13:234, 13:236-13:245, 13:247, 13:249N-13:252N, 13:254-13:276, 13:278Q-13:283, 13:285, 13:287-13:288, 13:290-13:294, 13:296-13:298N, 13:300-13:301, 13:303-13:313, 13:315-13:316, 13:318, 13:320, 13:322N-13:325, 13:327-13:328	DR11, DR13, DR14, Null, -						70									79	80	81						
*13:02:02, 13:289N	DR13, Null							?										81						
*13:03:01:01-13:03:12, 13:10, 13:33:02-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94:02-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N, 13:151-13:152, 13:170, 13:172, 13:174, 13:178, 13:180, 13:188, 13:194, 13:196, 13:216, 13:219, 13:227, 13:253, 13:317, 13:319N, 13:329N	DR13, Null, -						70									79		81						
*13:06, 13:12:01-13:12:05, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:118, 13:134, 13:157, 13:161, 13:163, 13:181, 13:220, 13:224, 13:231, 13:295N, 13:321	DR6, DR11, DR13, Null, -															79		81						
*13:09, 13:192, 13:210-13:211, 13:222	DR13, -																80	81						
*13:13, 13:119, 13:154, 13:156, 14:84, 14:116, 14:144 <sup>4</sup>	DR13, -															79		81						84
*13:15, 13:19, 13:53, 13:57, 13:104, 13:198	DR13, -						70	71								79	80	81						
*13:17, 13:116, 13:175, 13:214	DR13, -						70			73							80							
*13:26:01-13:26:02	DR14, -							71								79	80	81						
*13:27:01-13:27:02, 13:41, 13:71, 13:277	DR13, -						70					76				79	80	81						
*13:33:01, 13:94:01	DR13, -					69	70									79		81						
*13:43, 13:159, 13:179, 13:191, 13:217	DR13, -						70									79	80				83	84		
*13:44, 13:86, 13:206, 14:183 <sup>4</sup>	-							71										81						
*13:61:02, 13:96:01	-					69	70									79	80	81						
*13:67, 13:103, 13:160, 13:195	DR13, -															79	80							
*13:85	-						70	71								79		81						
*13:110, 13:167	-						?									79		81						
*13:122	-						70									79								
*13:129, 13:326	-						70					76					80	81						
*13:144	-						70									79	80	81						84

*13:146	-																		80	81			84					
*13:165	-							70												79	80							
*13:171:01	-							70			74									79	80			83	84			
*13:176	-							70						76						79								
*13:183	-							70													80	81						
*13:197	-										74									79	80			82	84			
*13:228	-																			79								
*13:235, 14:98 <sup>4</sup>	-							71												79			81					
*13:248	-							70													80							
*13:299	-							70																				
*13:314, 14:37 <sup>4</sup>	DR14, -																						80	81	82			
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87				
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87				
*14:01:01, 14:01:05, 14:04:01:01-14:04:01:03, 14:04:04-14:04:06, 14:04:09, 14:07:01-14:07:02, 14:10:01:01-14:10:01:02, 14:54:01:01-14:54:01:14, 14:54:03, 14:54:05-14:54:06, 14:54:08-14:54:12, 14:57, 14:70, 14:137N, 14:152N, 14:191-14:192, 14:196-14:197N, 14:199, 14:201-14:202, 14:204-14:205:01, 14:207, 14:210Q, 14:213-14:216, 14:219, 14:224, 14:227-14:228, 14:230-14:234, 14:237-14:238, 14:240-14:241, 14:243-14:245	DR4, DR6, DR14, DR1404, Null, -																											
*14:01:02, 14:01:04, 14:04:02-14:04:03, 14:26, 14:28, 14:31, 14:32:03, 14:35, 14:38:01, 14:39, 14:50:02, 14:55, 14:60-14:62, 14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99, 14:101, 14:107, 14:110, 14:112-14:114, 14:117-14:118, 14:124-14:125, 14:129, 14:131, 14:138-14:140, 14:142-14:143, 14:145, 14:147, 14:149-14:151, 14:153, 14:157-14:158, 14:163-14:164, 14:166N, 14:172, 14:180, 14:182, 14:184, 14:189-14:190, 14:203, 14:220, 14:225, 14:229	DR14, Null, -																											
*14:01:03, 14:23:02, 14:34, 14:72, 14:92N, 14:97, 14:126:01-14:126:02, 14:130, 14:133, 14:148, 14:175, 14:185	DR14, Null, -																											
*14:02:01:01-14:02:09, 14:06:01-14:06:06, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94, 14:106, 14:108, 14:121, 14:159, 14:165, 14:170, 14:176-14:177, 14:194-14:195N, 14:198, 14:209, 14:212, 14:221, 14:242	DR6, DR14, Null, -							71																				
*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85, 14:102, 14:115, 14:135, 14:156, 14:174, 14:181, 14:218, 14:223, 14:239	DR6, DR14, DR1403, -							71								79												
*14:04:07-14:04:08, 14:54:07	-																											
*14:05:01:01-14:05:05, 14:08, 14:11:01:01-14:11:01:02, 14:14, 14:23:01:01-14:23:01:03, 14:23:03-14:23:04, 14:36, 14:43-14:44:03, 14:54:02, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100, 14:103, 14:123, 14:127:01-14:127:02, 14:134, 14:155, 14:162, 14:167, 14:169, 14:187-14:188N, 14:193, 14:205:02-14:206, 14:211, 14:236	DR14, Null, -																											
*14:16	DR6						70										79	80	81	82	83	84						
*14:18, 14:81, 14:154, 14:160	DR14, -							71													81							
*14:19, 14:21, 14:109	DR14, -						70	71									79				81							
*14:22, 14:105	DR14, -																79	80			?	82	83	84				
*14:24, 14:226	DR14, -							71													80	81						
*14:25:01-14:25:02, 14:128	DR6, -																79	80			?	82		84				
*14:27:01-14:27:02	DR14, -							71									79	80			81							
*14:32:01-14:32:02	DR14							w													?	82	83	84				
*14:38:02, 14:104	-					69															?	82	83	84				
*14:42, 14:136, 14:161	-																					81						
*14:45, 14:95	DR14, -						70														81							
*14:46	DR14																											
*14:49, 14:119, 14:146	DR14, -							71													?	82	83	84				
*14:50:01	DR14										73										?	82	83	84				
*14:52	DR14															78					?		83					
*14:53	DR13, 14																79	80										
*14:54:04	-																				81	82	w	84				
*14:58, 14:186	DR14, -																				?	82		84				
*14:65	DR6							w													81							
*14:68:01-14:68:02, 14:93	DR14, -										74										?	82	83	84				
*14:69	-																79				?	82	84					
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87				
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87				
*14:73	-																	80			?	82	83	84				
*14:74, 14:173	-																79				?	82	83	84				
*14:82	-						70														81	82	83	84				
*14:89	-							71													81							
*14:111, 14:171	-					69															81	82	83	84				

*14:120, 14:122, 14:178	-					70												?	82	83	84				
*14:132	-					70													?		83	84			
*14:141	-																		81				85		
*14:168	-																		?	w	83	84			
*14:179	-					70	71				76								81		83				
*14:200	-						71															83	84		
*14:208	-																				82		84		
*14:217	-																		81	82	83	84		86	
*14:222N	Null						71																84		
*14:235	-																		79						
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:157, 15:159N-15:166, 15:168-15:186, 15:188:01:01-15:209N	DR2, DR15, Null, -						67																		
*15:21	DR2						67																w		
*15:25, 15:37:01, 15:100, 15:104:01	-						67	69																	
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-						67															83			
*15:141	-						67																	86	
*15:158, 15:187	-						67															83	w		
*15:167	-						67															82			
*16:01:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:65, 16:67-16:73	DR16, Null, -						68																		
*16:04:01-16:04:02, 16:18	DR16, -						68																w		
*16:66	-						67	68																	
DRB3*01:01:02:01-01:41, DRB3*01:43:01:105N, DRB3*02:01:02:26, DRB3*02:28-02:110, DRB3*02:112-02:130, DRB3*02:132-02:159, DRB3*02:161-02:183, DRB3*03:01:01:01-03:21, DRB3*03:23-03:58	DR52, Null, -																								85
DRB3*01:42	-										75														85
DRB3*02:27	-																				79				85
DRB3*02:131	-																					79		84	
DRB3*02:160	-																					79	80		85
DRB4*01:01:01:01, DRB4*01:01:01:03-01:03:01:01, DRB4*01:03:01:03-01:03:01:12, DRB4*01:03:01:14-01:13, DRB4*01:15-01:30, DRB4*01:32-01:162N, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																								86
DRB4*01:03:01:02N, DRB4*01:03:01:13N, DRB4*01:14N	Null																						82		86
DRB4*01:31	-																						84		86
DRB5*01:01:01:01-126, DRB5*02:02:01-02:30, DRB5*02:32-02:33	DR51, Null, -																								87
DRB5*02:31	-																								87
Well No.	ser. <sup>3</sup>	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	