

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-A allele ^{1,2,6}	ser ³																					
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:01:156, 01:01:158-01:04:01:02N, 01:06, 01:09:01:01-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:402, 01:404-01:417N, 01:419-01:426, 01:428-01:438, 01:440-01:465, 01:467-01:478, 36:04 ⁵	A1, A36, Null, -	1			4																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:01:157, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:391, 01:396Q, 01:403, 01:418N, 01:439, 36:01:01:01-36:01:02, 36:03, 36:05-36:15 ⁵	A1, A36, Null, -	1																				
*01:01:56, 01:361N	Null, -	1			4						11											
*01:07, 01:95, 01:289	A1, -	1			4											16						
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																		
*01:13, 01:176	A1, -	1			4						10	12		14								
*01:15N	Null	1			w																	
*01:21, 01:126, 01:360	A1, -	1		3	4																	
*01:26, 01:136, 01:192, 01:380:01:01-01:380:01:02, 11:94, 11:112, 11:211, 11:226, 11:290, 11:326 ⁴	A1, -	1			4						10											
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4						10			14								
*01:43, 11:271 ⁴	-	1			4					9												
*01:51	-	1			4			7		9												
*01:121	-	1			4														18			
*01:194	-	1			4							12		14								
*01:298	-	1			4				8							16						
*01:301Q	-	1			4			6														
*01:344, 01:381	-	1			4			7														
*01:427, 36:02 ⁵	A36, -	1		3										13								21
*01:466	-	1		3																		21
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-01-02:570:02, 02:572-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:1049, 02:1051-02:1094, 02:1096-02:1170, 02:1172-02:1206	A2, A203, A210, Low A2, Null, -	2																				
*02:17:02:01-02:17:04	A2, -		2			w																
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907, 02:1171	A2, -		2									12							19			
*02:35:02	A2											12							19			
*02:46, 02:70, 02:651, 02:1050, 02:1095	A2, -		2																19			
*02:48, 02:129, 02:571	A2, -																		19			
*02:55	A2, A28	2						7													20	21
*02:65, 02:407, 02:449, 02:932	A2, A19, -	2																18				
*02:78	-	w		4															19			
*02:81, 02:124	A2	2						8														
*02:135	-	2										12										
*02:149:01-02:149:02, 02:842, 02:947:01-02:947:02	-	2																			20	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*02:169	-		2		4										13		15					
*02:237, 02:809, 02:927	-		2											13		15						21
*02:243:01-02:243:03	-		2															17			20	
*02:309	-		2									11		13							20	
*02:454	-		2									11		13								
*02:527, 02:582	-		2					7														
*02:644	-							7		9											20	
*02:671	-		2																			21
*02:741	-		2					7				12							19		20	
*02:804	-		2					6														
*02:815	-		2					7													20	
*02:829	-				4				8													
*02:929	-		2															17				

[illegible]

[illegible]

[illegible]

*80:04	-					5										16					21	
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:01:110, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -							8														
B*07:507, C*06:187	-																				21	
B*15:01:83, B*35:01:60, B*58:01:24	-		w																			
C*12:131	-				4																	
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21