

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ^{1,2,5}	ser ³																							
*01:01:01:01:01:02:14, 01:04:01:01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:126, 01:128-01:138, 01:140-01:154	DR1, Null, -	65																						
*01:03:01:01:01:03:05, 01:39N, 01:99, 01:102, 01:112, 01:127, 01:155	DR1, DR103, Null, -	66																						
*01:42, 01:61, 01:89	-	65	66																					
*01:139	-	65																					86	
*03:01:01:01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:41, 03:04:01-03:06, 03:09, 03:11:01, 03:11:03-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:149, 03:151-03:153, 03:158-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180-03:184, 03:186-03:187, 03:191-03:205, 03:207-03:210, 03:214, 03:216-03:217	DR3, DR17, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130, 03:156N, 03:189N	DR17, Null, -					69	70																	
*03:01:24, 03:07:01:01:03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:155, 03:171, 03:178, 03:185, 03:188, 03:213	DR3, -					69												81						
*03:02:01:01:03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131, 03:154, 03:167, 03:176, 03:179, 03:206, 03:212, 03:215	DR3, DR18, -					69	71											81						
*03:08, 03:65, 03:140	DR3, -					69	70				75		77											
*03:10	DR3					69	70												83	84				
*03:42, 03:69, 03:87, 03:111, 03:162, 03:173, 03:190	-					69																		
*03:76, 03:174N	Null, -					69	70				76			79				81						
*03:125	-					69	70				76			79	80			81						
*03:126, 13:193 ⁴	-					70	71											81		83				
*03:157	-					69	70											81			84			
*03:177	-					69	70	71										81		83				
*03:211	-					69	71											81					86	
*03:218N, 13:176 ⁴	Null, -					70					76			79				81						
*04:01:01:01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:161, 04:163-04:210, 04:212N-04:250, 04:252-04:283, 04:285-04:381, 04:383-04:391	DR4, DR3, Null, -								72															
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:162, 04:251, 04:284:01-04:284:02	DR4, -								72										82					
*04:211	-								72											83				
*07:01:01:01:07:01:01:02, 07:01:01:04-07:01:35, 07:03-07:05, 07:07-07:127, 07:129N-07:166	DR7, Null, -									73														
*07:06	DR7									73	74													
*07:128	-									73									82					
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*08:01:01:01:08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:07, 08:04:01:01:08:07, 08:11:01:01:08:11:01:02, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86, 08:88-08:90, 08:93-08:94, 08:96, 08:99-08:101, 08:103, 08:105-08:107, 08:109, 08:111-08:114, 08:116-08:118, 08:120, 08:122-08:127, 08:130, 08:132-08:134	DR8, Null, -										74						80							
*08:01:06, 08:03:02:01-08:03:10, 08:03:12-08:03:16, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97, 08:102, 08:104, 08:110, 08:115, 08:119, 08:121, 08:128, 08:131	DR8, Null, -										74													
*08:03:11, 08:68:01	-										74										w			
*08:08	DR8										74							80		82				
*08:09:01	DR8										74							80				84		
*08:09:02, 14:15 ⁴	DR8, -										74							80	81			84		

[illegible]

*11:69, 11:82	-															77		80		82				
*11:73, 11:79, 11:131-11:132, 11:153, 11:184	-							70								77	79							
*11:88	-															77	79	80		82				
*11:89, 11:159, 11:266, 11:283	-															77					84			
*11:124:02	-							69	70							77		79	80					
*11:136	-							69								77								
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*11:229, 11:270	-						70			73				77			80							
*11:264, 13:05:01-01-13:05:03, 13:07:01-13:07:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:21:01-01-13:21:02, 13:42:01-13:42:02, 13:46, 13:49:01-13:50:03, 13:62, 13:100, 13:108, 13:132, 13:136, 13:150, 13:169, 13:189, 13:199, 13:203, 13:229, 13:246, 13:284, 13:302, 13:349, 13:352 ^d	DR13, DR11, DR6, -															79	80	81						
*11:293, 13:45 ^d	DR13, -						70									79	80		82		84			
*11:339, 13:67, 13:103, 13:160, 13:195 ^d	DR13, -															79	80							
*12:01:01-01-12:01:13, 12:03:02-12:04, 12:06-12:12, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36, 12:38, 12:40-12:41, 12:46-12:48, 12:53-12:54, 12:58-12:59, 12:62-12:63, 12:68, 12:70, 12:72N, 12:77, 12:79, 12:82-12:84, 12:86N-12:87, 12:89-12:91, 12:95, 12:99, 12:102, 12:105-12:107, 12:110-12:114	DR12, Null, -														78									
*12:02:01-01-12:02:15, 12:13, 12:18-12:19:02, 12:21, 12:23, 12:26-12:27, 12:31N-12:33, 12:37, 12:42-12:45, 12:49-12:52, 12:55-12:56, 12:60N-12:61, 12:64-12:65, 12:67, 12:69, 12:71, 12:74N-12:76, 12:78, 12:80-12:81, 12:85, 12:92-12:94, 12:97-12:98N, 12:100-12:101, 12:103-12:104, 12:108-12:109	DR12, Null, -														78		80							
*12:05	DR12														78								85	
*12:15	DR12														78		80						85	
*12:16:01-12:16:03	-									74					78		80							
*12:20	-														78		80	81						
*12:22	-									73	74				78									
*12:39:01-12:39:02	-										74				78									
*12:57, DRB3*02:111 ^d	-				68											79	80						85	
*12:66	-														78					83				
*12:73, 13:366 ^d	-						70			73					78		80							
*12:88	-									73					78									
*12:96	-														78					83	84			
*13:01:01-01-13:01:40, 13:01:42-13:02:01:14, 13:02:03-13:02:22, 13:04, 13:08, 13:16:01-01-13:16:01:02, 13:20:01-01-13:20:01:03, 13:22:01-01-13:24:01:03, 13:28:01-13:29:01:02, 13:31-13:32, 13:34-13:36, 13:38-13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:02-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117:01-01-13:117:01:02, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143, 13:145, 13:147-13:149, 13:153, 13:155, 13:162, 13:166, 13:168, 13:171:02, 13:173, 13:177, 13:182, 13:184-13:187, 13:190, 13:200N-13:202, 13:204-13:205, 13:207-13:209, 13:212-13:213, 13:215, 13:218, 13:221, 13:223, 13:225-13:226, 13:230, 13:233-13:234, 13:236-13:245, 13:247, 13:249N-13:252N, 13:254-13:276, 13:278Q-13:283, 13:285, 13:287-13:288, 13:290-13:294, 13:296-13:298N, 13:300-13:301, 13:303-13:313, 13:315-13:316, 13:318, 13:320, 13:322N-13:325, 13:327-13:328, 13:332, 13:334-13:336, 13:338-13:339, 13:342-13:343, 13:345, 13:348, 13:350-13:351, 13:354, 13:356-13:361, 13:363-13:365, 13:367N	DR13, DR11, DR14, Null, -						70									79	80	81						
*13:01:41, 13:165	-						70									79	80							
*13:02:02, 13:289N	DR13, Null						?											81						
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*13:03:01-01-13:03:13, 13:10, 13:33:02-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94:02-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N, 13:151-13:152, 13:170, 13:172, 13:174, 13:178, 13:180, 13:188, 13:194, 13:196, 13:216, 13:219, 13:227, 13:253, 13:317, 13:319N, 13:329N, 13:333, 13:337, 13:346-13:347, 13:362	DR13, Null, -						70									79		81						
*13:06, 13:12:01-13:12:06, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:118, 13:134, 13:157, 13:161, 13:163, 13:181, 13:220, 13:224, 13:231, 13:295N, 13:321, 13:330	DR13, DR11, DR6, Null, -															79		81						
*13:09, 13:183, 13:192, 13:210-13:211, 13:222, 13:341	DR13, -																80	81						

*14:53	DR13, 14																	79	80			82		84		
*14:54:04	-																				81	82	w	84		
*14:58, 14:186	DR14, -																				?	82		84		
*14:65	DR6								w												81		83	84		
*14:68:01-14:68:02, 14:93	DR14, -											74									?	82	83	84		
*14:69	-																	79			?	82		84		
*14:73	-																		80		?	82	83	84		
*14:74	-																	79			?	82	83	84		
*14:82	-							70													81	82	83	84		
*14:89, 14:272	-							71													81			84		
*14:111, 14:171	-					69															81	82	83	84		
*14:120, 14:122, 14:178	-							70													?	82	83	84		
*14:132	-							70													?		83	84		
*14:141	-																				81				85	
*14:168	-																				?	w	83	84		
*14:173, 14:235	-																	79			81	82	83	84		
*14:179	-							70	71				76								81					
*14:200	-								71														83	84		
*14:208	-																					82		84		
*14:217	-																				81	82	83	84		86
*14:222N	Null							71										79			?			84		
*14:250	-																				?	82				
*14:257, 14:262	-																				?				85	
*14:264	-																				81					
*14:265	-																						83	84		
*14:273	-							70			73							80			?	82	83	84		
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87		
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:57:01, 15:58-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:157, 15:159N-15:166, 15:168-15:172:01, 15:173-15:186, 15:188:01:01-15:213, 15:215-15:235	DR15, DR2, Null, -				67																					
*15:21	DR2			67																			w			
*15:25, 15:37:01, 15:57:02, 15:100, 15:104:01	-			67		69																				
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-			67																			83			
*15:141	-			67																				86		
*15:158, 15:187	-			67																			83	w		
*15:167, 15:172:02, 15:214	-			67																			82			
*16:01:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:65, 16:67-16:79	DR16, Null, -				68																					
*16:04:01-16:04:02, 16:18	DR16, -				68																		w			
*16:66	-			67	68																					
DRB3*01:01:02:01-01:41, DRB3*01:43-01:133N, DRB3*02:01-02:26, DRB3*02:28-02:110, DRB3*02:112-02:130, DRB3*02:132-02:159, DRB3*02:161-02:189, DRB3*02:191-02:215, DRB3*03:01:01:01-03:21, DRB3*03:23-03:66, DRB3*03:68-03:78	DR52, Null, -																							85		
DRB3*01:42	-											75												85		
DRB3*02:27	-																79							85		
DRB3*02:131	-																79						84	85		
DRB3*02:160	-																79	80						85		
DRB3*02:190	-								72															85		
DRB3*03:67	-																					82		85		
DRB4*01:01:01:01, DRB4*01:01:01:03-01:03:01:01, DRB4*01:03:01:03-01:03:01:12, DRB4*01:03:01:14-01:03:01:24, DRB4*01:03:01:26-01:13, DRB4*01:15-01:30, DRB4*01:32-01:190, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																							86		
DRB4*01:03:01:02N, DRB4*01:03:01:13N, DRB4*01:14N	Null																					82		86		
DRB4*01:31	-																						84	86		
DRB5*01:01:01:01-01:01:01:09, DRB5*01:01:01:11-01:130, DRB5*01:132-01:143, DRB5*02:02:01-02:30, DRB5*02:32-02:33, DRB5*02:35-02:39	DR51, Null, -																							87		
DRB5*02:31, DRB5*02:34	-											75													87	
Well No.	ser. ³	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87		