

INTERPRETATION TABLE			
HLA-B*27 SSP typing			
Amplification pattern of the B*27:01 to 27: 73 alleles ¹			
	Well		
	1	2	
Length of spec.	145	95	Length of spec.
PCR product			PCR product
Length of int.	430	515	Length of int.
pos. control ²			pos. control
5'-primer ³	167	363	5'-primer ³
	5' -gCT 3'	5' -AAT 3'	
3'-primer ⁴	272	418	3'-primer ⁴
	5' -TgC 3'	5' -gTC 3'	
Well No.	1	2	Well No.
HLA-B allele ⁵			HLA-B allele ⁵
*27:01-27:05:08, 27:05:10-27:05:15, 27:08, 27:10, 27:13, 27:15, 27:17, 27:25, 27:28, 27:36-27:40, 27:42, 27:44-27:45, 27:47-27:69, 27:71-27:73	1	2	*27:01-27:05:08, 27:05:10-27:05:15, 27:08, 27:10, 27:13, 27:15, 27:17, 27:25, 27:28, 27:36-27:40, 27:42, 27:44-27:45, 27:47-27:69, 27:71-27:73
*27:05:09, 27:12, 27:16, 27:18, 27:23, 27:26, 27:29, 27:31, 37:02, 47:04-47:05		2	*27:05:09, 27:12, 27:16, 27:18, 27:23, 27:26, 27:29, 27:31, 37:02, 47:04-47:05
*27:06-27:07, 27:09, 27:11, 27:14, 27:19-27:21, 27:24, 27:27, 27:30, 27:32-27:35, 27:41, 27:43, 27:46, 27:70, 44:97	1		*27:06-27:07, 27:09, 27:11, 27:14, 27:19-27:21, 27:24, 27:27, 27:30, 27:32-27:35, 27:41, 27:43, 27:46, 27:70, 44:97
HLA-B allele ⁵			HLA-B allele ⁵
Well No.	1	2	Well No.