

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
DPA1 allele <sup>1,2,3</sup>																							
*01:03:01:01-01:03:02, 01:03:04-01:03:09, 01:03:11-01:03:13, 01:03:15-01:03:26, 01:03:28-01:03:31, 01:03:33-01:03:57, 01:18-01:26, 01:28-01:32N, 01:34-01:38, 01:41-01-01:45, 01:48-01:50, 01:54-01:57, 01:59-01:61, 01:63-01:65, 01:67, 01:69-01:94, 01:96-01:110, 01:112-01:117, 01:119N-01:122, 01:124-01:132, 01:134-01:150, 01:152-01:153, 01:155:01:01-01:177, 01:179-01:180, 01:182-01:201	1	2	3																				
*01:03:03		2	3						10					15									
*01:03:10, 01:33:01-01:33:02, 01:95	1	2	3														17						
*01:03:14, 01:39, 01:181	1		3																				
*01:03:27	1	2	3		5																		
*01:03:32, 01:27, 01:118, 01:133, 01:178		2	3																				
*01:04:01, 01:04:03-01:04:04, 01:151	1	2		4																			
*01:04:02	1	2		4													?						
*01:05, 04:05 <sup>4</sup>	1				5																		
*01:06:01-01:06:02, 01:06:05		2	3			6	7																
*01:06:03-01:06:04		2	3			6	7								15								
*01:07	1	2	3							11				14									
*01:08	1	2		4										14									
*01:09	1	2	3										13										
*01:10	1	2	3			6																	
*01:11:01-01:11:02	1	2	3											14									
*01:11:03		2	3						10					14	15								
*01:12:01	1	2	3							11								17					
*01:12:02		2	3						10	11					15		17						
*01:12:03	1	2	3							11													
*01:13	1		3			6																	
*01:14, 01:68	1	2	3																19				
*01:15	1	2	3															18					
*01:16	1	2	3				7																
*01:17	1	2	3																	20			
*01:40, 01:46-01:47, 01:51-01:53, 01:66N, 01:123N	1	2	3														?						
*01:58:01:01-01:58:01:02, 02:03:03-02:03:05	1				5												17						
*01:62					5												17	18					
*01:111, 01:154N		2	3														?						
*02:01:01:01-02:01:01:04, 02:01:01:06-02:01:02:03, 02:01:04, 02:01:07-02:01:31, 02:13N, 02:18-02:19, 02:24, 02:26:01:01-02:26:01:02, 02:29, 02:31-02:32N, 02:34:01:01-02:34:01:02, 02:36-02:37, 02:39-02:40, 02:45:01-02:46, 02:49, 02:55, 02:57, 02:61, 02:66:01N-02:66:02N, 02:69:01-02:70, 02:74N-02:75, 02:77-02:80N, 02:82, 02:84, 02:86-02:87, 02:91, 02:93-02:94N, 02:97-02:99, 02:101:01-02:102Q, 02:104, 02:106-02:111, 02:114-02:116, 02:118-02:123, 02:125, 02:129-02:130					5		7										17						
*02:01:03, 02:01:05-02:01:06, 02:53					5		7										?						
*02:02:02:01-02:02:02:05, 02:02:02:07-02:02:02:17Q, 02:02:07-02:02:15, 02:07:04, 02:14, 02:17, 02:35, 02:38Q, 02:42, 02:44, 02:48, 02:51, 02:54, 02:56Q, 02:58, 02:62-02:64Q, 02:71-02:72, 02:76, 02:81, 02:83, 02:88-02:89, 02:95-02:96, 02:100, 02:103, 02:105, 02:112-02:113, 02:124, 02:126-02:128					5			8	9								17						
*02:02:03-02:02:06, 02:25, 02:117					5			8	9								?						
*02:03:01					5					10							?						
*02:03:02, 03:07:02, 03:15 <sup>4</sup>					5					10							17						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
*02:04					5	6		8	9								?						
*02:05					5			8	9					14			?						
*02:06, 02:15, 02:30, 02:67-02:68, 02:92					5			8	9				13				17						
*02:07:01:01-02:07:03, 02:52N, 02:90					5			8	9							16	17						
*02:08					5		7										17		19				
*02:09:01:01-02:09:02					5		7										17	18					

Negative Control

*02:10					5		8	9								17				21			
*02:11			3		5		7									?							
*02:12:01					5		8	9						16	17					22			
*02:12:02					5		8	9								17				22			
*02:16					5		7					13				17							
*02:20								8	9							17							
*02:21:01, 02:21:04		2	3		5	6	7						14	15		17							
*02:21:02		2	3			6	7						14			?							
*02:21:03		2	3			6	7						14	15		17							
*02:22:01-02:22:02					5				9	10						17							
*02:23:01-02:23:02, 02:28:01-02:28:02					5											17							
*02:27:01			3		5	6		8	9				14	15		17							
*02:27:02			3			6		8	9				14	15	16	?							
*02:27:03			3			6		8	9				14	15		17							
*02:33, 02:85					5			8	9														
*02:41N					5			8	9			?				?							
*02:43							7									17							
*02:47:01					5			8	9							?	18						
*02:47:02					5			8	9							17	18						
*02:50:01		2	3			6	7						14	15									
*02:50:02		2	3			6	7						14										
*02:59:01					5		7																
*02:59:02							7																
*02:60					5		7								16	17		19		22			
*02:65					5		7						14			17							
*02:73					5			8	9						16	?							
*03:01:01:01-03:01:01:15, 03:01:03, 03:05:01:01Q-03:05:01:02Q, 03:09, 03:11N-03:13, 03:16-03:17			3						10	11				15		17							
*03:01:02:01-03:01:02:04, 03:05:02Q, 03:18			3						10	11				15									
*03:01:04			3							11						17							
*03:02			3					9	10					15		?							
*03:03				4					10	11				15		?							
*03:04			3						10	11			14	15		17							
*03:06:01			3			6		8		11				15									
*03:06:02			3			6		8		11				15	16	?							
*03:06:03			3			6		8		11				15		17							
*03:07:01									10							?							
*03:08					5				10							17	18						
*03:10N									10	11				15		17							
*03:14			3						10	11		13		15		17							
*04:01:01:01-04:01:02		1			5						12												
*04:02:01:01-04:04, 04:06		1			5						12					17							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23

Negative Control