

*12:01:01-12:01:04, 12:03:02-12:03:03, 12:05- 12:12, 12:14, 12:17, 12:24N-12:25, 12:28- 12:30, 12:34-12:36	DR12, Null, -												
*12:02:01-12:02:05, 12:13, 12:15, 12:18-12:21, 12:23, 12:26-12:27, 12:31N- 12:33, 12:37	DR12, Null, -												
*12:04	-										18		
*12:16:01	-										18		
*12:22	-									17	18		
*13:01:01-13:02:01, 13:02:03-13:02:07, 13:04, 13:08, 13:16, 13:20, 13:22- 13:24, 13:28-13:29, 13:31- 13:32, 13:34-13:36, 13:38- 13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01- 13:61:02, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:01-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143	DR11, DR13, DR14, Null, -						14						
*13:02:02	DR13						14						
*13:03:01-13:03:06, 13:10, 13:33:01-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N	DR13, Null, -						14						
*13:05:01-13:05:02, 13:07:01-13:07:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:21:01-13:21:02, 13:42, 13:46, 13:49-13:50:02, 13:62, 13:100, 13:108, 13:132, 13:136	DR13, DR13/D R11, DR6, -												
*13:06, 13:12:01-13:12:03, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:110, 13:118, 13:134	DR11, DR13, DR6, -												
Well No.	DR	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	DR	9	10	11	12	13	14	15	16	17	18	19	20
DRB1 allele ⁴	ser ⁵												
*13:09	DR13												
*13:13, 13:119, 14:84, 14:116 ¹⁰	DR13, -												
*13:15, 13:19, 13:53, 13:57, 13:104	DR13, -						14	15					

*14:42	-													
*14:46, 14:52	DR14													
*14:49, 14:119	DR14, -							15						
*14:50	DR14									17				
*14:58	DR14													
*14:65	DR6							w						
*14:68, 14:93	DR14, -										18			
*14:69	-													
*14:73	-													
*14:74	-													
*14:82	-							14						
*14:89	-								15					
*14:95	-							14						
*14:98	-								15					
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:65, 15:67-15:81	DR15, DR2, Null, -			11										
*15:21	DR2			11										
*15:25	-			11		13								
*15:27, 15:34, 15:66	-			11										
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19	DR16, Null, -				12									
*16:04, 16:18	DR16, -				12									
<i>DRB3*01:01:02:01-01:15, DRB3*02:01:02:26, DRB3*02:28-02:29N, DRB3*03:01:01-03:03 DRB3*02:27</i>	DR52, Null, -													
<i>DRB4*01:01:01:01-01:03:01:01, DRB4*01:03:01:03-01:08 DRB4*01:03:01:02N</i>	DR53, - Null													
<i>DRB5*01:01:01-01:14, DRB5*02:02-02:06</i>	DR51, Null, -													
DRB1 allele ⁴	ser ⁵													
Well No.	DR	9	10	11	12	13	14	15	16	17	18	19	20	

				25									DR17, DR3, Null, -	*03:01:01:01-03:01:20, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18- 03:20, 03:22-03:23, 03:25- 03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70- 03:73, 03:75, 03:77-03:84,
				25									DR18, DR3, -	*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74
				25									DR3, -	*03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39- 03:41, 03:49
21													DR3, -	*03:08, 03:65
						27	28						DR3	*03:10
													-	*03:42, 03:69
													-	*03:46, 03:64
		23		25									-	*03:76
													DR4, DR4/DR 3, Null, -	*04:01:01-04:05:11, 04:05:13-04:61, 04:63- 04:68, 04:70-04:72:02, 04:74-04:104, 04:106- 04:121, 04:123-04:133
						26							DR4, -	*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122
													DR7, Null, -	*07:01:01:01-07:01:04, 07:03-07:22
			24										DR8, -	*08:01:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39, 08:42- 08:44
													DR8, -	*08:03:02-08:03:03, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29- 08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:49
			24		26								DR8	*08:08
			24				28						DR8	*08:09, 14:15 ⁷
		23	24	25			28						DR13, -	*08:20, 13:18, 13:47,
		23	24				28						DR8	*08:21
21			24										DR8, DR11, -	*08:31, 08:41, 11:67 ⁹
	22						28						-	*08:32
							28						-	*08:35, 08:36:02
													-	*08:40
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
												ser ⁵	DRB1 allele ⁴	
						27							DR9, -	*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04- 09:20

Negative control

													DR9	*09:01:06, 09:03
													DR10, -	*10:01:01-10:04
21		23	24										DR11, -	*11:01:01-11:01:17, 11:04:01-11:04:10, 11:06:01-11:06:03, 11:09- 11:10:02, 11:12:01- 11:12:02, 11:15, 11:24, 11:27:01-11:30, 11:32- 11:33, 11:35, 11:37:01- 11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01- 11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62, 11:66, 11:72, 11:74:01-11:75, 11:77- 11:78, 11:81, 11:84:01- 11:84:02, 11:88, 11:90- 11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120- 11:121, 11:123, 11:126, 11:128-11:129, 11:133-
21		23	24										DR11, DR11/D R13, -	*11:02:01-11:03, 11:11:01- 11:11:02, 11:14:01- 11:14:02, 11:16, 11:20- 11:21, 11:36, 11:40-11:41, 11:48, 11:63, 11:65:01- 11:65:02, 11:68, 11:70, 11:76, 11:85-11:86, 11:93, 11:118, 11:122, 11:124, 11:127
21			24										DR11	*11:05
21													DR11, -	*11:07, 11:53, 11:103, 11:105, 11:107, 11:125
21		23											DR11, -	*11:08:01-11:08:02, 11:18- 11:19:03, 11:42, 11:57
21						27	28						DR11	*11:13:01-11:13:02
21						27	28						DR11, DR14	*11:17, 11:52
21													-	*11:22, 11:98, 11:104
21		23	24				28						DR11, -	*11:23, 11:25, 11:96
21													DR11	*11:26, 11:34
21		23					28						DR11, -	*11:31, 11:45, 11:64,
21			24				28						DR11	*11:55
21		23	24										DR11, -	*11:59, 11:80, 11:83, 11:87
21			24		26								-	*11:69, 11:82
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
												ser ⁵	DRB1 allele ⁴	
21		23										-	*11:73, 11:79, 11:131- 11:132	
21						28						-	*11:89	

Negative control

	22											DR12, Null, -	*12:01:01-12:01:04, 12:03:02-12:03:03, 12:05- 12:12, 12:14, 12:17, 12:24N-12:25, 12:28- 12:30, 12:34-12:36
	22		24									DR12, Null, -	*12:02:01-12:02:05, 12:13, 12:15, 12:18-12:21, 12:23, 12:26-12:27, 12:31N- 12:33, 12:37
	22											-	*12:04
	22		24									-	*12:16:01
	22											-	*12:22
		23	24	25								DR11, DR13, DR14, Null, -	*13:01:01-13:02:01, 13:02:03-13:02:07, 13:04, 13:08, 13:16, 13:20, 13:22- 13:24, 13:28-13:29, 13:31- 13:32, 13:34-13:36, 13:38- 13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01- 13:61:02, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:01-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143
				25								DR13	*13:02:02
		23		25								DR13, Null, -	*13:03:01-13:03:06, 13:10, 13:33:01-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N
		23	24	25								DR13, DR13/D R11, DR6, -	*13:05:01-13:05:02, 13:07:01-13:07:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:21:01-13:21:02, 13:42, 13:46, 13:49-13:50:02, 13:62, 13:100, 13:108, 13:132, 13:136
		23		25								DR11, DR13, DR6, -	*13:06, 13:12:01-13:12:03, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:110, 13:118, 13:134
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.
												ser ⁵	DRB1 allele ⁴
			24	25								DR13	*13:09
		23		25			28					DR13, -	*13:13, 13:119, 14:84, 14:116 ¹⁰
		23	24	25								DR13, -	*13:15, 13:19, 13:53, 13:57, 13:104

Negative control

			24										DR13, -	*13:17, 13:116
		23	24	25									DR14, -	*13:26:01-13:26:02
		23	24	25									DR13	*13:27, 13:41, 13:71
		23	24			27	28						DR13	*13:43
				25									-	*13:44, 13:86
		23	24		26		28						DR13	*13:45
		23	24										DR13, -	*13:67, 13:103
		23		25									-	*13:85
		23											-	*13:122
			24	25									-	*13:129
						26	27	28					DR14, DR1404 /DR14,D R4/DR6, -	*14:01:01-14:01:02, 14:04, 14:07:01-14:07:02, 14:10, 14:26, 14:28, 14:31, 14:35, 14:38-14:39, 14:54:01, 14:55, 14:57, 14:60-14:62, 14:70-14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99, 14:101, 14:104, 14:107, 14:110-14:114, 14:117-14:118, 14:120, 14:122, 14:124-14:125, 14:129, 14:131
							27	28					DR14, Null, -	*14:01:03, 14:08, 14:23:02, 14:34, 14:54:02, 14:72, 14:92N, 14:97, 14:126, 14:130
				25			27						DR14, DR6, -	*14:02, 14:06:01-14:06:02, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94, 14:106, 14:108, 14:121
		23		25				28					DR1403 /DR14, DR6, -	*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85, 14:102, 14:115
				25			27	28					DR14, -	*14:05:01-14:05:04, 14:14, 14:23:01, 14:23:03, 14:36, 14:43-14:45, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100, 14:103, 14:123,
							27	28					DR14	*14:11
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
21	22	23	24	25	26	27	28	29	30	31	32	DR	Well No.	
												ser ⁵	DRB1 allele ⁴	
		23	24		26	27	28						DR6	*14:16
				25		27	28						DR14, -	*14:18, 14:81
		23		25		w							DR14, -	*14:19, 14:21, 14:109
		23	24		26	27	28						DR14, -	*14:22, 14:105
			24	25									DR14	*14:24
		23	24		26		28						DR6, DR13/ 14, -	*14:25, 14:53, 14:128
		23	24	25			28						DR14	*14:27
					26	27	28						DR14	*14:32:01-14:32:02
			24	25	26								DR14	*14:37

Negative control

