

Well No.	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47		
*11:73, 11:79, 11:131-11:132, 11:153	-					21							28		30											41								
*11:80, 11:87, 11:135, 11:142, 11:182	-					21						27	28		30	31										41								
*11:83	-					21						27	28		30	31																		
Well No.																																		
*11:89, 11:159, 11:266	-												28								35					41	42							
*11:96, 11:148	-												28		30	31					35					41								
*11:107	-					20						26	28																					
*11:119	-												28		30							37				41	42							
*11:124:02	-					20	21						28		30	31										41								
*11:136	-					20							28													41								
*11:149	-												28		30	31												42						
*11:184	-					21							28		30																			
*11:193:01-11:193:02	-											25	28			31																		
*11:202	-												28								34	35				41	42							
*11:209	-												28			31																		
*11:229, 11:270	-					21							28			31																		
*11:261	-												28										37											
*11:264	-														30	31	32										42							
*12:01:01:01-12:01:09, 12:03:02-12:04, 12:06-12:08, 12:10-12:12, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36, 12:40-12:41, 12:47, 12:54, 12:58-12:59, 12:63, 12:68, 12:70, 12:72N, 12:77, 12:79, 12:82, 12:84	DR12, Null, -													29																				
*12:02:01:01-12:02:10, 12:13, 12:18-12:20, 12:23, 12:26-12:27, 12:31N-12:33, 12:37, 12:43-12:45, 12:49-12:52, 12:55-12:56, 12:60N-12:61, 12:64-12:65, 12:67, 12:69, 12:71, 12:74N-12:75, 12:78, 12:80-12:81, 12:85	DR12, Null, -													29		31																		
*12:05	DR12												29																		44			
*12:09, 12:46, 12:62	DR12, -												29								36													
*12:15	DR12												29		31																	44		
*12:16:01-12:16:03	-											25	29		31																			
*12:21, 12:42, 12:76	-												29		31																			
*12:22, 12:39	-											25	29																					
*12:38, 12:53, 12:83	-												29																					
*12:48	-												29								w													
*12:57, DRB3*02:111 ⁴	-					19							29		30	31															44			
*12:66	-												29								34										w			
*12:73	-					21							29		31																			
*13:01:01:01-13:01:08, 13:01:10-13:01:22, 13:01:24-13:01:26, 13:01:28-13:02:01:09, 13:02:03-13:02:17, 13:16, 13:20, 13:28:01-13:29, 13:31, 13:34, 13:39, 13:51-13:52, 13:59, 13:61:01, 13:63-13:64, 13:68-13:69, 13:73-13:74, 13:78-13:80, 13:83, 13:87, 13:91-13:92, 13:96:02-13:97:01, 13:98-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117, 13:121, 13:123-13:128, 13:130-13:131, 13:138, 13:140-13:143, 13:145, 13:147-13:148, 13:153, 13:155, 13:166, 13:168, 13:171:02, 13:173, 13:184-13:187, 13:190, 13:200N-13:202, 13:205, 13:207-13:209, 13:212, 13:215, 13:218, 13:221, 13:223, 13:225-13:226, 13:233-13:234, 13:236-13:245, 13:249N-13:252N, 13:254-13:261, 13:263-13:275, 13:278Q-13:283, 13:285, 13:287-13:288, 13:290-13:294, 13:296-13:298N, 13:300-13:301, 13:303-13:305	DR13, Null, -					21									30	31	32									40	41							
Well No.																																		
*13:01:09, 13:01:23, 13:35-13:36, 13:40, 13:72, 13:76, 13:84, 13:135, 13:177, 13:182, 13:262	DR13, -					21									30	31	32									40								
*13:01:27	-					21									30	31	32									40	41	42						
*13:02:02	DR13					?											32									41								

Negative Control

Well No.	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
*14:78							22								30		32		35			37		39			42					
*14:82						21						27						32	33	34	35		37				42					
*14:84															30		32										42					
*14:89							22										32			35							42					
*14:95						21						27					32		34	35						41	42					
*14:98							22								30		32										?					
*14:104, 14:111					20													33	34	35							?					
*14:116, 14:144															30		32			35							42					
*14:119, 14:146							22											33	34	35							?					
*14:120, 14:122, 14:178						21												33	34	35							42					
*14:126:01																		34	35	36							42					
Well No.																																
*14:128															30	31		33		35							41	?				
*14:132						21						27							34	35							41	42				
*14:136, 14:161																	32										42					
*14:138																		33	34	35			38				42					
*14:141																											42		44			
*14:168																		w	34	35							42					
*14:169																		32		34	35			39			42					
*14:170							22											32		34				39			?	43				
*14:173															30			33	34	35			37				42					
*14:175																		34	35	36		38					42					
*14:179						21	22					27						32		34							41	?	43			
*14:183							22										32										?	43				
*14:186, 14:208																		33		35						41	42					
*14:200							22												34	35							42		45			
*14:217																		33	34	35						41						
*14:221							22										32		34							41						
*14:222N	Null						22								30				35								42					
*15:01:01:01-15:09, 15:11:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:157, 15:159N-15:166, 15:168-15:186, 15:188-15:189	DR15, DR2, Null, -			18																												
*15:10	DR2			18																					40							
*15:21	DR2			18															w			37										
*15:25, 15:37:01, 15:100, 15:104:01	-			18	20																											
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-			18															34													
*15:141	-			18																											45	
*15:158, 15:187	-			18															34	w												
*15:167	-			18														33														
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:65, 16:67-16:68	DR16, Null, -			19																												
*16:04:01-16:04:02, 16:18	DR16, -			19																w												
*16:66	-			18	19																											
DRB3*01:01:02:01-01:41, DRB3*01:43-01:90, DRB3*02:01-02:26, DRB3*02:28-02:57, DRB3*02:59-02:110, DRB3*02:112-02:130, DRB3*02:132-02:145N, DRB3*02:147-02:150, DRB3*03:01:01:01-03:48	DR52, Null, -																												44			
DRB3*01:42	-									26																				44		
DRB3*02:27	-														30													43	44			
DRB3*02:58	-																											43	44			
DRB3*02:131	-														30					35												
DRB3*02:146	-				20																								44			
DRB4*01:01:01:01, DRB4*01:01:01:03-01:03:01:01, DRB4*01:03:01:03-01:30, DRB4*01:32-01:65N, DRB4*01:67-01:137, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																													45		
DRB4*01:03:01:02N	Null																	33											45			
DRB4*01:31	-																			35									45			
DRB4*01:66	-									24																				45		
DRB5*01:01:01:01-01:07, DRB5*01:09-01:99, DRB5*02:02:01-02:25N, DRB5*02:27-02:29	DR51, Null, -																														46	
DRB5*01:08:01N-01:08:02N, DRB5*02:26N	Null																							39						46		
Well No.																																

Negative Control