

DQ-DR 2022 7N1 DQ

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
DQB1 allele <sup>1,2</sup>	ser. <sup>3</sup>															
*02:01:01:01-02:01:09, 02:01:11-02:02:16, 02:04-02:76, 02:78-02:179, 02:181-02:195	DQ2, Null, -			3												
*02:03:02	-		2	3												
*02:77, 02:180	-			3			6									
*03:01:01:01-03:01:01:12, 03:01:01:14-03:01:01:36, 03:01:03-03:01:06, 03:01:09-03:01:54, 03:04:01:01-03:04:04, 03:09, 03:10:03, 03:16, 03:19:01:01-03:19:05, 03:21-03:22:02, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:196-03:198:02, 03:201-03:202, 03:206-03:208, 03:216, 03:218-03:219, 03:231, 03:235-03:236, 03:241-03:243, 03:246, 03:253-03:257, 03:260, 03:264, 03:266-03:268, 03:271, 03:275-03:276N, 03:281, 03:284-03:286, 03:288, 03:290, 03:292-03:294, 03:297, 03:302-03:303N, 03:305-03:307, 03:309, 03:311-03:312, 03:317:01-03:318, 03:326-03:331, 03:338N, 03:340N-03:342, 03:347, 03:350, 03:353, 03:358N, 03:360-03:361, 03:366, 03:370, 03:372-03:373, 03:376N-03:378, 03:380-03:381, 03:385N, 03:387, 03:389-03:390, 03:394, 03:396, 03:399N-03:401, 03:404, 03:407N-03:408, 03:417, 03:419-03:421, 03:423-03:428, 03:430-03:432, 03:434-03:436, 03:438-03:439, 03:448, 03:451, 03:454-03:455, 03:458, 03:460, 03:465, 03:467-03:470	DQ7, Null, -				4			7				12				
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:154, 03:162, 03:252, 03:291, 03:335, 03:354N, 03:391, 03:443	DQ7, Null, -				4			7								
*03:01:07, 03:13, 03:58, 03:194, 03:232	DQ7, -				4			w					12			
*03:02:01:01-03:02:02, 03:02:04-03:02:05, 03:02:07-03:02:12, 03:02:14-03:02:17, 03:02:19-03:02:29, 03:02:31-03:02:35, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45:01-03:45:02, 03:63-03:64, 03:66N-03:68, 03:85, 03:106, 03:146, 03:153, 03:161, 03:174-03:175, 03:178, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215, 03:220-03:221, 03:224, 03:229, 03:240, 03:245, 03:247, 03:251, 03:261, 03:263:01:01-03:263:01:02, 03:265, 03:269N, 03:273-03:274, 03:277-03:279, 03:287, 03:289, 03:295-03:296, 03:298-03:301, 03:310N, 03:315, 03:320-03:324, 03:333-03:334N, 03:339N, 03:343-03:345, 03:348-03:349, 03:362, 03:364, 03:367-03:369, 03:379, 03:383, 03:386, 03:388, 03:392, 03:403N, 03:409-03:410, 03:412, 03:415-03:416, 03:422N, 03:429, 03:433, 03:437, 03:440N-03:442, 03:444, 03:446-03:447, 03:450, 03:452, 03:456-03:457, 03:459, 03:462-03:464, 03:466	DQ8, Null, -					5		7					12			
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204, 03:223, 03:226, 03:233, 03:237N, 03:352, 03:371	DQ8, Null, -					5		7								
*03:02:06, 03:02:18, 03:02:30, 03:125, 03:228	-					5							12			
*03:03:02:01-03:03:02:07, 03:03:04-03:03:10, 03:03:12-03:03:15, 03:03:17-03:03:25, 03:03:27, 03:15, 03:23:03, 03:26, 03:30-03:31, 03:33-03:34, 03:38:01, 03:39-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212, 03:222, 03:227, 03:230, 03:234, 03:239, 03:248-03:249, 03:270, 03:280, 03:282N, 03:304, 03:313, 03:316, 03:319, 03:332, 03:337, 03:356N-03:357N, 03:359, 03:363, 03:365, 03:374-03:375N, 03:382, 03:384, 03:393, 03:395, 03:397-03:398, 03:402, 03:405-03:406, 03:414, 03:445, 03:453, 03:461	DQ9, Null, -						6	7					12			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15

DQ-DR 2022 7N1 DQ

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
*03:03:03:01-03:03:03:02, 03:03:16, 03:20, 03:38:02, 03:99Q, 03:105, 03:112, 03:156, 03:238, 03:258, 03:336, 03:351	DQ9, -						6	7								
*03:03:11	-						6	7				11	12			
*03:03:26, 03:62, 03:78, 03:81, 03:110, 03:151, 03:225, 03:244, 03:272, 03:308, 03:314, 03:413	-							7					12			
*03:05:01, 03:05:05, 03:61, 03:181, 03:250, 03:346	DQ8, -					5		7				11				
*03:05:03	DQ8					5		7				11	12			
*03:06, 03:25:01-03:25:02, 03:411N	DQ3, Null, -						6						12			
*03:08, 03:107	DQ8, -					5		w					12			
*03:10:01-03:10:02:02, 03:14:01-03:14:02, 03:195	DQ3, -				4			7					12			15
*03:12	DQ9						6	7					12			15
*03:17:01	DQ9							7				11	12			
*03:17:02	-							7								
*03:23:01	-		2					7								
*03:23:02, 03:217, 03:259	-		2					7					12			
*03:65, 03:137, 03:283	-						6	w					12			
*03:70, 03:179	-					5		7					12			15
*03:72, 03:100	-							7				11				
*03:104	-					5	6	7								
*03:132	-					5		7	8							
*03:147, 03:418, 03:449	-				4								12			
*03:183	-				4			7								15
*03:262	-					5		w								
*03:355	-		2		4			7					12			
*04:01:01:01-04:01:02, 04:01:04-04:02:01:01, 04:02:01:04-04:02:15, 04:02:17-04:02:18, 04:02:20-04:02:21, 04:06-04:09, 04:11-04:21, 04:23, 04:25N-04:32, 04:34-04:37, 04:39-04:48, 04:50-04:74, 04:76-04:87	DQ4, Null, -								8	9						
*04:01:03, 04:02:16	-							8	9				12			
*04:02:19, 04:04-04:05, 04:33, 04:49	-							8								
*04:03:01-04:03:02	-						6	8	9							
*04:03:03	-						6		9				12			
*04:10	-		2					8	9						13	
*04:22, 04:24, 04:38	-							8	w							
*04:75	-					5		8	9							
*05:01:01:01-05:01:15, 05:01:17-05:03:23, 05:03:26-05:03:27, 05:05:01-05:43:02, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:73-05:76, 05:78-05:81, 05:84-05:97, 05:99-05:104, 05:106-05:113, 05:115, 05:117-05:127, 05:129-05:131, 05:133-05:145, 05:147-05:174, 05:177-05:206N, 05:208N-05:217, 05:219-05:243, 05:245-05:257, 05:259-05:261, 05:263-05:292, 06:325 <sup>4</sup>	DQ5, Null, -	1										10				
*05:01:16, 05:03:25, 05:60, 05:82-05:83, 05:98, 05:116, 05:175, 05:218, 05:258	-											10				
*05:03:24, 05:04, 05:44, 05:52, 05:54, 05:72, 05:77, 05:105, 05:114, 05:128N, 05:132Q, 05:146, 05:207, 05:244, 05:262	DQ5, Null, -	1														
*05:176	-	1	2													
*06:01:01:01-06:01:06, 06:01:08-06:01:30, 06:43, 06:54N-06:58, 06:98, 06:100-06:105, 06:108, 06:120, 06:132, 06:140, 06:153:01-06:153:02, 06:157, 06:167, 06:177, 06:181, 06:194, 06:205, 06:214, 06:229, 06:239, 06:243, 06:245-06:247, 06:251, 06:257-06:258, 06:260, 06:263, 06:268, 06:274, 06:277, 06:285, 06:302, 06:305, 06:307, 06:309-06:310, 06:312, 06:321, 06:323, 06:330N, 06:342, 06:359, 06:382, 06:415	DQ6, Null, -		2													15
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15

DQ-DR 2022 7N1 DQ

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
*06:01:07, 06:03:11, 06:04:01:01-06:05:01, 06:07:01-06:07:02, 06:09:01:01-06:09:12, 06:12, 06:17, 06:21-06:22:03, 06:25, 06:28, 06:34-06:36, 06:38-06:39, 06:42, 06:45, 06:52-06:53:02, 06:69:01-06:69:02, 06:85-06:86, 06:88:01:01-06:89, 06:92:02-06:94, 06:118:01-06:119, 06:121, 06:129, 06:135, 06:142, 06:144N, 06:149, 06:155, 06:158N, 06:160, 06:164, 06:171, 06:180, 06:186, 06:189, 06:193N, 06:199, 06:202-06:204, 06:207, 06:210, 06:212, 06:217, 06:231, 06:241, 06:252N, 06:254, 06:261, 06:265-06:267, 06:275, 06:280-06:283, 06:287-06:288, 06:291-06:292, 06:299, 06:301, 06:303N, 06:313, 06:318, 06:320, 06:332, 06:339-06:340, 06:343, 06:348-06:349, 06:351, 06:353, 06:358, 06:361, 06:369, 06:375, 06:381, 06:385, 06:387, 06:398, 06:407, 06:414N	DQ6, DQ1, Null, -		2													
*06:02:01:01-06:02:12, 06:02:14-06:02:42, 06:02:44-06:02:57, 06:14:01-06:14:03, 06:16, 06:19:01-06:20, 06:23-06:24, 06:33, 06:37, 06:47-06:50, 06:51:02, 06:68, 06:70-06:78, 06:80-06:84:01:02, 06:95, 06:96:02-06:97, 06:107, 06:109, 06:111-06:117, 06:122, 06:124-06:127, 06:136-06:138, 06:147, 06:150-06:152, 06:159, 06:161, 06:163, 06:166, 06:173-06:175, 06:178-06:179N, 06:182-06:183, 06:188, 06:192, 06:197-06:198, 06:200-06:201, 06:208, 06:211, 06:213, 06:215-06:216N, 06:219, 06:224-06:228, 06:232, 06:236-06:237, 06:240, 06:242, 06:249, 06:255-06:256, 06:262, 06:264, 06:270:02-06:271, 06:273, 06:284, 06:286, 06:289-06:290, 06:293-06:298, 06:300, 06:304N, 06:306N, 06:308N, 06:311, 06:314-06:315, 06:317N, 06:324, 06:326, 06:333, 06:335, 06:338, 06:341N, 06:344, 06:347, 06:354-06:357, 06:363-06:364, 06:366, 06:370, 06:372, 06:374, 06:376, 06:379N-06:380, 06:384, 06:388, 06:390, 06:395, 06:397N, 06:401-06:402, 06:404-06:406, 06:409, 06:411-06:413, 06:416Q	DQ6, Null, -		2											13	14	
*06:02:13, 06:15:01-06:15:02, 06:46, 06:79:01-06:79:02, 06:146:01-06:146:02, 06:235, 06:386	DQ6, -		2											13		
*06:02:43, 06:03:10, 06:03:33, 06:51:01, 06:96:01, 06:322:01-06:322:02	-		2			6									14	
*06:03:01:01-06:03:09, 06:03:12-06:03:32, 06:03:34-06:03:44, 06:08:01-06:08:03, 06:10-06:11:04, 06:13:01-06:13:03, 06:18:01-06:18:02, 06:26N-06:27:02, 06:30-06:32:02, 06:40-06:41:01:03, 06:44, 06:59-06:65, 06:67, 06:87, 06:90-06:91, 06:99:01, 06:106, 06:110, 06:128, 06:130-06:131, 06:133-06:134, 06:141, 06:143, 06:145, 06:148, 06:154, 06:165, 06:170, 06:176, 06:184-06:185, 06:187, 06:190:01-06:191, 06:195-06:196, 06:206:01-06:206:02, 06:218, 06:221-06:223, 06:230, 06:233-06:234, 06:238, 06:244, 06:248, 06:250, 06:253, 06:259, 06:269-06:270:01, 06:272, 06:276, 06:278-06:279, 06:316, 06:319, 06:327-06:329, 06:331, 06:334, 06:336, 06:345N-06:346, 06:350, 06:352, 06:360, 06:362, 06:365, 06:367-06:368, 06:371, 06:373, 06:378, 06:391-06:394N, 06:396, 06:400, 06:403, 06:408, 06:410	DQ6, DQ1, Null, -		2												14	
*06:05:02	DQ6		2												?	
*06:06	DQ5		2												?	?
*06:29, 06:123, 06:139, 06:337	DQ6, -		2			5									14	
*06:66, 06:172	-		2			6										
*06:99:02-06:99:03	-		2												14	15
*06:156, 06:162	-									10				13	14	
*06:168, 06:377	-		2			6										15
*06:169	-									10					14	
*06:209	-		2		4											15
*06:383N	Null													13	14	
*06:389	-		1	2											14	
*06:399	-														14	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15