

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
DQB1 allele ^{1,2}	ser. ³															
*02:01:01:01-02:01:09, 02:01:11-02:01:13, 02:01:15-02:02:24, 02:04-02:76, 02:78-02:179, 02:181-02:212	DQ2, Null, -			3												
*02:03:02	-		2	3												
*02:77, 02:180	-			3			6									
*03:01:01:01-03:01:01:12, 03:01:01:14-03:01:01:58, 03:01:03-03:01:06, 03:01:09-03:01:58, 03:04:01:01-03:04:04, 03:09, 03:10:03, 03:16, 03:19:01:01-03:19:06, 03:21-03:22:02, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:196-03:198:02, 03:201-03:202, 03:206-03:208, 03:216, 03:218-03:219, 03:231, 03:235-03:236, 03:241-03:243, 03:246, 03:253-03:257, 03:260, 03:264, 03:266-03:268, 03:271, 03:275-03:276N, 03:281, 03:284-03:286, 03:288, 03:290, 03:292-03:294, 03:297, 03:302-03:303N, 03:305-03:307, 03:309:01-03:309:02, 03:311-03:312, 03:317:01-03:318, 03:326-03:331, 03:338N, 03:340N-03:342, 03:347, 03:350, 03:353, 03:358N, 03:360-03:361, 03:366, 03:370, 03:372-03:373, 03:376N-03:378, 03:380-03:381, 03:385N, 03:387, 03:389-03:390, 03:394, 03:396, 03:399N-03:401, 03:404, 03:407N-03:408, 03:417, 03:419-03:421, 03:423-03:428, 03:430-03:432, 03:434-03:436, 03:438-03:439, 03:448, 03:451, 03:454-03:455, 03:458, 03:460, 03:465, 03:467-03:470, 03:472-03:473N, 03:475-03:476, 03:480Q, 03:482-03:483, 03:485-03:486, 03:488N, 03:491-03:492, 03:496-03:497, 03:499N, 03:503, 03:506, 03:508	DQ7, Null, -				4			7				12				
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:154, 03:162, 03:252, 03:291, 03:335, 03:354N, 03:391, 03:443	DQ7, Null, -				4			7								
*03:01:07, 03:13, 03:58, 03:194, 03:232	DQ7, -				4			w					12			
*03:02:01:01-03:02:02:02, 03:02:04-03:02:05, 03:02:07-03:02:12, 03:02:14-03:02:17, 03:02:19-03:02:29, 03:02:31-03:02:37, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45:01-03:45:02, 03:63-03:64, 03:66N-03:68, 03:85, 03:106, 03:146, 03:153, 03:161, 03:174-03:175, 03:178, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215, 03:220-03:221, 03:224, 03:229, 03:240, 03:245, 03:247, 03:251, 03:261, 03:263:01:01-03:263:01:02, 03:265, 03:269N, 03:273-03:274, 03:277-03:279, 03:287, 03:289, 03:295-03:296, 03:298-03:301, 03:310N, 03:315, 03:320-03:324, 03:333-03:334N, 03:339N, 03:343-03:345, 03:348-03:349, 03:362, 03:364, 03:367-03:369, 03:379, 03:383, 03:386, 03:388, 03:392, 03:403N, 03:409-03:410, 03:412, 03:415-03:416, 03:422N, 03:429, 03:433, 03:437, 03:440N-03:442, 03:444, 03:446-03:447, 03:450, 03:452, 03:456-03:457, 03:459, 03:462-03:464, 03:466, 03:471, 03:479, 03:481, 03:484, 03:490, 03:493, 03:495, 03:498, 03:500-03:502, 03:504	DQ8, Null, -					5		7					12			
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204, 03:223, 03:226, 03:233, 03:237N, 03:352, 03:371	DQ8, Null, -					5		7								
*03:02:06, 03:02:18, 03:02:30, 03:125, 03:228	-					5							12			
*03:03:02:01-03:03:02:11, 03:03:04-03:03:10, 03:03:12-03:03:15, 03:03:17-03:03:25, 03:03:27-03:03:29, 03:15, 03:23:03, 03:26, 03:30-03:31, 03:33-03:34, 03:38:01, 03:39-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212, 03:222, 03:227, 03:230, 03:234, 03:239, 03:248-03:249, 03:270, 03:280, 03:282N, 03:304, 03:313, 03:316, 03:319, 03:332, 03:337, 03:356N-03:357N, 03:359, 03:363, 03:365, 03:374-03:375N, 03:382, 03:384, 03:393, 03:395, 03:397-03:398, 03:402, 03:405-03:406, 03:414, 03:445, 03:453, 03:461, 03:477-03:478, 03:487, 03:489, 03:494, 03:505, 03:507, 03:509N-03:510	DQ9, Null, -						6	7					12			
Well No.	ser.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
*03:03:03:01-03:03:03:02, 03:03:16, 03:20, 03:38:02, 03:99Q, 03:105, 03:112, 03:156, 03:238, 03:258, 03:336, 03:351	DQ9, -						6	7								
*03:03:11	-						6	7				11	12			
*03:03:26, 03:03:30, 03:62, 03:78, 03:81, 03:110, 03:151, 03:225, 03:244, 03:272, 03:308, 03:314, 03:413	-							7					12			
*03:05:01, 03:05:05, 03:61, 03:181, 03:250, 03:346	DQ8, -					5		7				11				

<p>*06:02:01:01-06:02:12, 06:02:14-06:02:42, 06:02:44-06:02:59, 06:14:01-06:14:03, 06:16, 06:19:01-06:20, 06:23-06:24, 06:33, 06:37, 06:47-06:50, 06:51:02, 06:68, 06:70-06:78, 06:80-06:84:01:02, 06:95, 06:96:02-06:97, 06:107, 06:109, 06:111-06:117, 06:122, 06:124-06:127, 06:136-06:138, 06:147, 06:150-06:152, 06:159, 06:161, 06:163, 06:166, 06:173-06:175, 06:178-06:179N, 06:182-06:183, 06:188, 06:192, 06:197-06:198, 06:200-06:201, 06:208:01-06:208:02, 06:211, 06:213, 06:215-06:216N, 06:219, 06:224-06:228, 06:232, 06:236-06:237, 06:240, 06:242, 06:249, 06:255-06:256, 06:262, 06:264, 06:270:02-06:271, 06:273, 06:284, 06:286, 06:289-06:290, 06:293-06:298, 06:300, 06:304N, 06:306N, 06:308N, 06:311, 06:314-06:315, 06:317N, 06:322:03, 06:324, 06:326, 06:333, 06:335, 06:338, 06:341N, 06:344, 06:347, 06:354-06:357, 06:363-06:364, 06:366, 06:370, 06:372, 06:374, 06:376, 06:379N-06:380, 06:384, 06:388, 06:390, 06:395, 06:397N, 06:401-06:402, 06:404-06:406, 06:409, 06:411-06:413, 06:416Q, 06:422N, 06:430-06:431, 06:437-06:438, 06:445-06:448, 06:457, 06:461, 06:468</p>	<p>DQ6, Null, -</p>				<p>13 14</p>											
<p>*06:02:13, 06:15:01-06:15:02, 06:46, 06:79:01-06:79:02, 06:146:01-06:146:02, 06:235, 06:386, 06:436, 06:451</p>	<p>DQ6, -,</p>	<p>2</p>			<p>13</p>											
<p>*06:02:43, 06:03:10, 06:03:33, 06:51:01, 06:96:01, 06:322:01-06:322:02</p>	<p>-</p>	<p>2</p>	<p>6</p>		<p>14</p>											
<p>*06:03:01:01-06:03:09, 06:03:12-06:03:32, 06:03:34-06:03:47, 06:08:01-06:08:03, 06:10-06:11:04, 06:13:01-06:13:03, 06:18:01-06:18:02, 06:26N-06:27:02, 06:30-06:32:02, 06:40-06:41:01:03, 06:44, 06:59-06:65, 06:67, 06:87, 06:90-06:91, 06:99:01, 06:106, 06:110, 06:128, 06:130-06:131, 06:133-06:134, 06:141, 06:143, 06:145:01-06:145:02, 06:148, 06:154, 06:165, 06:170, 06:176, 06:184-06:185, 06:187, 06:190:01-06:191, 06:195-06:196, 06:206:01-06:206:02, 06:218, 06:221-06:223, 06:230, 06:233-06:234, 06:238, 06:244, 06:248, 06:250, 06:253, 06:259, 06:269-06:270:01, 06:272, 06:276, 06:278-06:279, 06:316, 06:319:01-06:319:02, 06:327-06:329, 06:331, 06:334, 06:336, 06:345N-06:346, 06:350, 06:352, 06:360, 06:362, 06:365, 06:367-06:368, 06:371, 06:373, 06:378, 06:391-06:394N, 06:396, 06:400, 06:403, 06:408, 06:410, 06:417, 06:423N-06:425, 06:428, 06:433, 06:440, 06:442-06:443, 06:450, 06:453-06:455, 06:459-06:460, 06:470</p>	<p>DQ6, DQ1, Null, -</p>	<p>2</p>			<p>14</p>											
<p>*06:05:02</p>	<p>DQ6</p>	<p>2</p>			<p>?</p>											
<p>*06:06</p>	<p>DQ5</p>	<p>2</p>			<p>? ?</p>											
<p>*06:29, 06:123, 06:139, 06:337</p>	<p>DQ6, -</p>	<p>2</p>	<p>5</p>		<p>14</p>											
<p>*06:66, 06:118:04, 06:172</p>	<p>-</p>	<p>2</p>	<p>6</p>													
<p>*06:99:02-06:99:03</p>	<p>-</p>	<p>2</p>			<p>14 15</p>											
<p>*06:156, 06:162</p>	<p>-</p>			<p>10</p>	<p>13 14</p>											
<p>*06:168, 06:377</p>	<p>-</p>	<p>2</p>	<p>6</p>		<p>15</p>											
<p>*06:169</p>	<p>-</p>			<p>10</p>	<p>14</p>											
<p>*06:209</p>	<p>-</p>	<p>2</p>	<p>4</p>		<p>15</p>											
<p>*06:383N, 06:456N</p>	<p>Null</p>				<p>13 14</p>											
<p>*06:389</p>	<p>-</p>	<p>1</p>	<p>2</p>		<p>14</p>											
<p>*06:399</p>	<p>-</p>				<p>14</p>											
<p>Well No.</p>	<p>ser.</p>	<p>1</p>	<p>2</p>	<p>3</p>	<p>4</p>	<p>5</p>	<p>6</p>	<p>7</p>	<p>8</p>	<p>9</p>	<p>10</p>	<p>11</p>	<p>12</p>	<p>13</p>	<p>14</p>	<p>15</p>
<p>Well No.</p>	<p>ser.</p>	<p>1</p>	<p>2</p>	<p>3</p>	<p>4</p>	<p>5</p>	<p>6</p>	<p>7</p>	<p>8</p>	<p>9</p>	<p>10</p>	<p>11</p>	<p>12</p>	<p>13</p>	<p>14</p>	<p>15</p>