

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
DQB1 allele <sup>1,2</sup>	ser. <sup>3</sup>															
*02:01:01:01-02:01:09, 02:01:11-02:01:13, 02:01:15-02:02:30, 02:04-02:76, 02:78-02:179, 02:181-02:217, 02:219-02:233, 02:235-02:238	DQ2, Null, -			3												
*02:03:02	-		2	3												
*02:77, 02:180:01-02:180:02	-			3			6									
*03:01:01:01-03:01:01:12, 03:01:01:14-03:01:01:70, 03:01:03-03:01:06, 03:01:09-03:01:58, 03:01:60-03:01:68, 03:04:01:01-03:04:04, 03:09, 03:10:03, 03:16, 03:19:01:01-03:19:06, 03:21-03:22:02, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:196-03:198:02, 03:201-03:202, 03:206-03:208, 03:216, 03:218-03:219, 03:231, 03:235-03:236, 03:241-03:243, 03:246, 03:253-03:257, 03:260, 03:264, 03:266-03:268, 03:271, 03:275-03:276N, 03:281, 03:284-03:286, 03:288, 03:290, 03:292-03:294, 03:297, 03:302-03:303N, 03:305-03:307, 03:309:01-03:309:02, 03:311-03:312, 03:317:01-03:318, 03:326-03:331, 03:338N, 03:340N-03:342, 03:347, 03:350, 03:353, 03:358N, 03:360-03:361:02, 03:366, 03:370, 03:372-03:373, 03:376N-03:378, 03:380-03:381, 03:385N, 03:387, 03:389-03:390, 03:394, 03:396, 03:399N-03:401, 03:404, 03:407N-03:408, 03:417, 03:419-03:421, 03:423-03:428, 03:430-03:432, 03:434-03:436, 03:438-03:439, 03:448, 03:451, 03:454-03:455, 03:458, 03:460, 03:465, 03:467-03:470, 03:472-03:473N, 03:475-03:476, 03:480Q, 03:482-03:483, 03:485-03:486, 03:488:01N-03:488:02N, 03:491-03:492, 03:496-03:497, 03:499N, 03:503, 03:506:01:01-03:506:01:02, 03:508, 03:513-03:514, 03:519-03:520, 03:522, 03:524, 03:526-03:531, 03:534, 03:537-03:539, 03:542N, 03:544, 03:546, 03:549-03:550, 03:552-03:553, 03:555-03:557, 03:562-03:565	DQ7, Null, -				4			7				12				
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:154, 03:162, 03:252, 03:291, 03:335, 03:354N, 03:391, 03:443	DQ7, Null, -				4			7								
*03:01:07, 03:13, 03:58, 03:194, 03:232	DQ7, -				4			w					12			
*03:01:59, 03:147, 03:418, 03:449	-				4								12			
*03:02:01:01-03:02:02:02, 03:02:04-03:02:05, 03:02:07-03:02:12, 03:02:14-03:02:17, 03:02:19-03:02:29, 03:02:31-03:02:46, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45:01-03:45:02, 03:63-03:64, 03:66N-03:68, 03:85, 03:106, 03:146, 03:153, 03:161, 03:174-03:175, 03:178, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215, 03:220-03:221, 03:224, 03:229, 03:240, 03:245, 03:247, 03:251, 03:261, 03:263:01:01-03:263:01:02, 03:265, 03:269N, 03:273-03:274, 03:277-03:279, 03:287, 03:289, 03:295-03:296, 03:298-03:301, 03:310N, 03:315, 03:320-03:324, 03:333-03:334N, 03:339N, 03:343-03:345, 03:348-03:349, 03:362, 03:364, 03:367-03:369, 03:379, 03:383, 03:386, 03:388, 03:392, 03:403N, 03:409-03:410, 03:412, 03:415-03:416, 03:422N, 03:429, 03:433, 03:437, 03:440N-03:442, 03:444, 03:446-03:447, 03:450, 03:452, 03:456-03:457, 03:459, 03:462-03:464, 03:466, 03:471, 03:479, 03:481, 03:484, 03:490, 03:493, 03:495, 03:498, 03:500:01, 03:501-03:502, 03:504, 03:511-03:512, 03:515, 03:517, 03:525, 03:533, 03:535, 03:540-03:541N, 03:543, 03:547-03:548, 03:554, 03:558-03:559, 03:561	DQ8, Null, -					5		7				12				
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204, 03:223, 03:226, 03:233, 03:237N, 03:352, 03:371	DQ8, Null, -					5		7								
*03:02:06, 03:02:18, 03:02:30, 03:125, 03:228	-					5							12			
*03:03:02:01-03:03:02:20Q, 03:03:04-03:03:10, 03:03:12-03:03:15, 03:03:17-03:03:25, 03:03:27-03:03:29, 03:03:31-03:03:34, 03:15, 03:23:03, 03:26, 03:30-03:31, 03:33-03:34, 03:38:01, 03:39-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212, 03:217:02, 03:222, 03:227, 03:230, 03:234, 03:239, 03:248-03:249, 03:270, 03:280, 03:282N, 03:304, 03:313, 03:316, 03:319, 03:332, 03:337, 03:356N-03:357N, 03:359, 03:363, 03:365, 03:374-03:375N, 03:382:01-03:382:02, 03:384, 03:393, 03:395, 03:397-03:398, 03:402, 03:405-03:406, 03:414, 03:445, 03:453, 03:461, 03:477-03:478, 03:487, 03:489, 03:494, 03:505, 03:507, 03:509N-03:510, 03:516, 03:518, 03:521, 03:523, 03:532, 03:536, 03:545, 03:551N, 03:560N	DQ9, Null, -						6	7				12				
Well No.	ser.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
*03:03:03:01-03:03:03:02, 03:03:16, 03:20, 03:38:02, 03:99Q, 03:105, 03:112, 03:156, 03:238, 03:258, 03:336, 03:351	DQ9, -						6	7								
*03:03:11	-						6	7				11	12			
*03:03:26, 03:03:30, 03:62, 03:78, 03:81, 03:110, 03:151, 03:225, 03:244, 03:272, 03:308, 03:314, 03:413, 03:500:02	-							7					12			
*03:05:01, 03:05:05, 03:61, 03:181, 03:250, 03:346	DQ8, -					5		7				11				
*03:05:03	DQ8					5		7				11	12			
*03:06, 03:25:01-03:25:02, 03:411N	DQ3, Null, -						6						12			
*03:08, 03:107	DQ8, -					5		w					12			

*03:10:01-03:10:02:02, 03:14:01-03:14:02, 03:195, 03:474	DQ3, -			4		7				12			15			
*03:12	DQ9					6	7			12			15			
*03:17:01	DQ9						7			11	12					
*03:17:02	-						7									
*03:23:01	-			2			7									
*03:23:02, 03:217:01, 03:259	-			2			7				12					
*03:65, 03:137, 03:283	-						6	w			12					
*03:70, 03:179	-					5	7				12		15			
*03:72, 03:100	-						7			11						
*03:104	-					5	6	7								
*03:132	-					5	7	8								
*03:183	-				4		7						15			
*03:262	-					5	w									
*03:355	-			2	4		7				12					
*04:01:01:01-04:01:02, 04:01:04-04:02:01:01, 04:02:01-04:04:02:15, 04:02:17-04:02:18, 04:02:20-04:02:27, 04:06-04:09, 04:11-04:21, 04:23, 04:25N-04:32, 04:34-04:37, 04:39-04:48, 04:50-04:74, 04:76-04:104	DQ4, Null, -							8	9							
*04:01:03, 04:02:16	-							8	9		12					
*04:02:19, 04:04-04:05, 04:33, 04:49	-							8								
*04:03:01-04:03:02	-						6	8	9							
*04:03:03	-						6	8	9		12					
*04:10	-			2				8	9				13			
*04:22, 04:24, 04:38	-							8	w							
*04:75	-					5		8	9							
*05:01:01:01-05:01:15, 05:01:17-05:02:26, 05:02:28-05:03:23, 05:03:26-05:03:37, 05:05:01-05:43:02, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:73-05:76, 05:78-05:81, 05:84-05:97, 05:99-05:104, 05:106-05:113, 05:115, 05:117-05:127, 05:129-05:131, 05:133-05:145, 05:147-05:174, 05:177-05:206N, 05:208N-05:217, 05:219-05:243, 05:245-05:257, 05:259-05:261, 05:263-05:299, 05:301-05:303, 05:305-05:343, 05:345-05:349, 05:351-05:352, 06:325 <sup>4</sup>	DQ5, Null, -			1							10					
*05:01:16, 05:03:25, 05:60, 05:82-05:83, 05:98, 05:116, 05:175, 05:218, 05:258	-										10					
*05:02:27, 05:03:24, 05:04:01-05:04:02, 05:44, 05:52, 05:54, 05:72, 05:77, 05:105, 05:114, 05:128N, 05:132Q, 05:146, 05:207, 05:244, 05:262, 05:300, 05:304, 05:350	DQ5, Null, -			1												
*05:176	-			1	2											
*05:344, 06:01:07, 06:01:32, 06:01:35, 06:03:11, 06:04:01-01-06:05:01, 06:07:01-06:07:02, 06:09:01-01-06:09:13, 06:12, 06:17, 06:21-06:22:03, 06:25, 06:28, 06:34-06:36, 06:38-06:39, 06:42, 06:45, 06:52-06:53:02, 06:69:01-06:69:02, 06:85-06:86, 06:88:01-01-06:89, 06:92:02-06:94, 06:118:01-06:118:03, 06:119, 06:121, 06:129, 06:135, 06:142, 06:144N, 06:149, 06:155, 06:158N, 06:160, 06:164, 06:171, 06:180, 06:186, 06:189, 06:193N, 06:199, 06:202-06:204, 06:207, 06:210, 06:212, 06:217, 06:231, 06:241, 06:252N, 06:254, 06:261, 06:265-06:267, 06:275, 06:280-06:283, 06:287-06:288, 06:291-06:292, 06:299, 06:301, 06:303N, 06:313, 06:318, 06:320, 06:332, 06:339-06:340, 06:343, 06:348-06:349, 06:351, 06:353, 06:358, 06:361, 06:369, 06:375, 06:381, 06:385, 06:387, 06:398, 06:407, 06:414N, 06:420, 06:426, 06:429, 06:432, 06:434, 06:439Q, 06:441, 06:444, 06:449, 06:452N, 06:458N, 06:465, 06:467, 06:473, 06:479, 06:484, 06:489-06:490, 06:496, 06:498, 06:505-06:506, 06:508, 06:511, 06:519 <sup>4</sup>	DQ6, DQ1, Null, -			2												
*06:01:01:01-06:01:06, 06:01:08-06:01:31, 06:01:33-06:01:34, 06:01:36, 06:43, 06:54N-06:58, 06:98, 06:100-06:105, 06:108, 06:120, 06:132, 06:140, 06:153:01-06:153:02, 06:157, 06:167, 06:177, 06:181, 06:194, 06:205, 06:214, 06:229, 06:239, 06:243, 06:245-06:247, 06:251, 06:257-06:258, 06:260, 06:263, 06:268, 06:274, 06:277, 06:285, 06:302, 06:305, 06:307, 06:309-06:310, 06:312, 06:321, 06:323, 06:330N, 06:342, 06:359, 06:382, 06:415, 06:418-06:419, 06:421, 06:427, 06:435, 06:462-06:464, 06:466, 06:469, 06:471-06:472, 06:482, 06:492, 06:501, 06:507N, 06:509, 06:513, 06:516	DQ6, Null, -			2									15			
Well No.	ser.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15

