

Well No.		16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
DRB1 allele ^{1,2,5}	ser ³																																	
*01:01:01:01:01:02:14, 01:04:01:01:12, 01:14:01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:126, 01:128-01:138, 01:140-01:154	DR1, Null, -	16																																
*01:03:01:01:01:03:05, 01:39N, 01:99, 01:102, 01:112, 01:127, 01:155	DR103, DR1, Null, -	17																																
*01:13	DR1	16								24																								
*01:42, 01:61, 01:89	-	16	17																															
*01:139	-	16																														45		
*03:01:01:01:03:01:22, 03:01:25-03:01:41, 03:04:01:03:05:03, 03:09, 03:11:01, 03:11:03, 03:13:01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:25:02, 03:28, 03:30, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:55-03:59, 03:61-03:63, 03:66-03:68N, 03:70-03:73, 03:77-03:79, 03:81, 03:83-03:84, 03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121, 03:123-03:124, 03:127-03:129, 03:132-03:137, 03:139, 03:141-03:149, 03:151-03:153, 03:159-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180, 03:182-03:184, 03:186-03:187, 03:191-03:205, 03:207-03:210, 03:214, 03:216-03:217	DR3, DR17, Null, -						20	21										32									41							
*03:01:23, 03:06, 03:26, 03:31, 03:60, 03:75, 03:80, 03:82, 03:85, 03:122, 03:138, 03:181	DR3, -					20	21											32																
*03:01:24, 03:07:01:01-03:07:02, 03:21, 03:24, 03:32, 03:39-03:40, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:171, 03:178, 03:185, 03:188, 03:213	DR3, -					20												32									41							
*03:02:01:01:03:03, 03:27, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:131, 03:154, 03:167, 03:176, 03:179, 03:206, 03:212, 03:215	DR3, DR18, -					20	22											32																
*03:08, 03:65, 03:140	DR3, -					20	21				26		28								34	35					39	41						
*03:10	DR3					20	21																					41						
*03:12	DR3					20	21											32										41						
*03:17, 03:35, 03:41:01-03:41:02	DR3, -					20												32																
*03:29, 03:119	DR3, -					20	22											32										41						
*03:42, 03:69, 03:87, 03:111, 03:173	-					20																						41						
*03:46, 03:64, 03:130, 03:156N, 03:189N	Null, -					20	21																					41						
*03:54	-					20	21			24								32										41						
*03:76, 03:174N	Null, -					20	21					27				30		32										41	43					
*03:92	-					20	21															36		38										
*03:125	-					20	21					27				30	31	32								40	41							
*03:126, 13:193 ⁴	-					21	22											32		34							41	43						
*03:155	-					20												32								39		41	42					
*03:157	-					20	21											32		35														
*03:158	-					20	21											32							39									
*03:162	-					20															36													
*03:177	-					20	21	22										32		34							41	43						
*03:190	-					20																	37											
*03:211	-					20	22											32														45		
*03:218N, 13:176 ⁴	Null, -					21						27				30		32									41	43						
*04:01:01:01:04:05:11, 04:05:13-04:11:08, 04:13-04:17:02, 04:19-04:24, 04:26-04:57, 04:59-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:161, 04:163-04:210, 04:212N-04:229, 04:231-04:250, 04:252-04:283, 04:285-04:381, 04:383-04:391	DR4, DR3, Null, -							23																										
*04:12, 04:18, 04:25, 04:58	DR4							23																37										
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:162, 04:251, 04:284:01-04:284:02	DR4, -							23										33																
*04:211	-							23												34									43					
*04:230	-							23																					43					
*07:01:01:01:07:01:01:02, 07:01:01:04:07:01:35, 07:03-07:05, 07:07-07:127, 07:129N-07:166	DR7, Null, -								24																									
*07:06	DR7								24	25																								
*07:128	-								24										33															
Well No.		16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
*08:01:01:01:08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:07, 08:05, 08:16, 08:24:01-08:24:02, 08:26, 08:39, 08:42-08:44, 08:50, 08:55, 08:64, 08:72-08:73, 08:77-08:79, 08:82, 08:84-08:86, 08:88-08:90, 08:93, 08:96, 08:100, 08:105-08:109, 08:111-08:112, 08:114, 08:116-08:118, 08:120, 08:123, 08:125, 08:127, 08:130, 08:132-08:134	DR8, Null, -									25							31					36												

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]