

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
<b>DQB1 allele<sup>1,2,3</sup></b>																																		
<b>*02:01:01-02:01:24, 2:43-02:45, 02:47, 02:49, 02:52, 02:55-02:57, 02:60-02:61, 02:63, 02:66, 02:68-02:71, 02:73-02:76, 02:78, 02:81, 02:82-02:83, 02:85-02:88, 02:90-02:94, 02:98-02:106<sup>5</sup></b>	1		3																															
<b>*02:02:01:01-02:02:05, 02:06, 02:10, 02:65, 02:95, 02:97<sup>5</sup></b>	1			4																														
<b>*02:03</b>		2		4																														
<b>*02:04</b>	1		3		5																													
<b>*02:05</b>	1		3			6																												
<b>*02:07:01-02:07:02, 02:16<sup>4</sup></b>	1		3						9																									
<b>*02:08</b>	1		3							10																								
<b>*02:09, 02:24<sup>4</sup></b>	1		3								11																							
<b>*02:11</b>	1			4								12																						
<b>*02:12</b>	1			4						10																								
<b>*02:13</b>	1		3									12																						
<b>*02:14:01-02:14:02</b>	1		3																		20													
<b>*02:15, 02:29<sup>4</sup></b>	1		3																18															
<b>*02:17</b>			3										13																					
<b>*02:18N, 02:34<sup>4</sup></b>	1		3											14																				
<b>*02:19</b>	1		3				7																											
<b>*02:20N, 02:22<sup>4</sup></b>	1		3													16																		
<b>*02:21, 02:35<sup>4</sup></b>	1		3												15																			
<b>*02:23</b>	1		3		5																	21												
<b>*02:25</b>			3								12																							
<b>*02:26</b>	1			4													17																	
<b>*02:27-02:28<sup>4</sup></b>	1		3																					22										
<b>*02:30</b>	1		3								12																							
<b>*02:31</b>	1		3																				21											
<b>*02:32</b>	1		3									13																						
<b>*02:33</b>	1		3																	19														
<b>*02:36</b>	1		3																				19	20										
<b>*02:37</b>	1		3		5												17																	
<b>*02:38</b>	1		3										13	14																				
<b>*02:39</b>	1		3												15	16																		
<b>*02:40</b>			3																				21											
<b>*02:41, 02:53Q<sup>4</sup></b>	1		3																					23										
<b>*02:42</b>	1		3																						24									
<b>*02:46</b>	1		3																							25								
<b>*02:48</b>	1																17																	
<b>*02:50</b>	1			4																							26							
<b>*02:51</b>	1		3																						24	25								
<b>*02:54</b>	1		3										14	16																				
<b>*02:58N</b>	1		3																											30				
<b>*02:59</b>	1		3																				22								31			
<b>*02:62</b>	1			4																								27						
<b>*02:64</b>	1			4																										28				
<b>*02:67N</b>	1		3																												29			
<b>*02:72</b>			3																									27						
<b>*02:77</b>		2	3																															
<b>*02:79</b>	1		3																											28				
<b>*02:80</b>	1																															31		
<b>*02:84, 02:89</b>	1																																	
<b>*02:96N</b>	1		3		5																									28				
<b>*03:01:01:01-03:01:01:12, 03:01:01:14-03:20, 03:22-03:23:02, 03:25:01-03:78, 03:80-03:96, 03:98-03:163, 03:166-03:240, 03:242-03:278, 03:280, 04:01:01:01-04:02:01:01, 04:02:01:04-04:42, 05:01:01:01-05:02:10, 05:02:12-05:13, 05:15-05:34, 05:36-05:83, 05:85-05:160, 06:01:01:01-06:37, 06:39-06:43, 06:45-06:46, 06:48-06:85, 06:87-06:101, 06:105-06:217, 06:219, 06:221-06:226, 06:228-06:242, 06:244-06:252N</b>			3																															
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
<b>*03:21, 05:35, 06:243</b>			3																															
<b>*03:24, 03:79</b>																	17																	
<b>*03:241, 06:44, 06:47</b>			3																													31		
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		

Negative Control

N.C.