

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DQB1 allele ^{1,2,3}																																	
*02:01:01:01-02:01:35, 02:01:37-02:01:38, 02:43-02:45, 02:47, 02:49, 02:52, 02:55-02:57, 02:60-02:61, 02:66, 02:68-02:71, 02:73-02:76, 02:78, 02:82-02:83, 02:85-02:88, 02:90-02:94, 02:98-02:109, 02:111-02:112, 02:114-02:115, 02:118-02:119, 02:123, 02:125, 02:128-02:130, 02:132N-02:136, 02:139-02:140, 02:148-02:149, 02:151-02:152, 02:154-02:155, 02:157-02:160, 02:164, 02:166, 02:168, 02:170	1		3																														
*02:01:36	1																																
*02:02:01:01-02:02:12, 02:06, 02:65, 02:84, 02:89:01-02:89:02, 02:95, 02:97, 02:110, 02:113, 02:116-02:117, 02:120-02:122, 02:124, 02:126-02:127, 02:131, 02:137-02:138, 02:141-02:147, 02:150, 02:153, 02:156, 02:161-02:162N, 02:165, 02:167N, 02:169, 02:171Q	1			4																													
*02:03:01	w	2		4																													
*02:03:02, 02:77		2	3																														
*02:04	1		3		5																												
*02:05	1		3			6																											
*02:07:01-02:07:02, 02:16 ⁴	1		3					9																									
*02:08	1		3						10																								
*02:09, 02:24 ⁴	1		3						11																								
*02:10	1			4				8																									
*02:11	1			4									12																				
*02:12	1			4					10				12																				
*02:13	1		3										12																				
*02:14:01-02:14:02	1		3																	20													
*02:15	1		3															18			20												
*02:17	w		3										13																				
*02:18N, 02:34 ⁴	1		3											14																			
*02:19	1		3			7																											
*02:20N, 02:22 ⁴	1		3													16																	
*02:21, 02:35 ⁴	1		3												15																		
*02:23	1		3		5																21												
*02:25			3									12										21											
*02:26	1			4														17															
*02:27-02:28 ⁴	1		3																			22											
*02:29, 02:58N ⁴	1		3																											30			
*02:30	1		3									12									20												
*02:31	1		3																		21												
*02:32	1		3										13																				
*02:33	1		3																														
*02:36	1		3																	19													
*02:37	1		3		5													17		19	20												
*02:38	1		3										13	14																			
*02:39	1		3												15	16																	
*02:40			3																			21											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*02:41, 02:53Q ⁴	1		3																				23										
*02:42	1		3																					24									
*02:46	1		3																						25								
*02:48	1																	17															
*02:50	1			4																19													
*02:51	1		3																					24	25								
*02:54	1		3											14		16																	
*02:59	1		3																												31		
*02:62	1			4																								27					
*02:63	1		3																						25				29				
*02:64	1			4																									28				
*02:67N	1		3																											29			
*02:72			3																										27				
*02:79	1		3																										28				
*02:80	1			4																												31	
*02:81	1		3					8																									
*02:96N	1		3		5																									28			
*02:163N	1		3																							26							

Negative Control

[illegible]