

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	
DQA1 or DQB1 allele^{1,2}																								
DQA1*01:01:01:01:01:01:01:03, 01:01:01:05-01:111									9															
DQA1*02:01:01:01:02:01:08, 02:01:10-02:02N, 02:04-02:05, 02:07-02:15, 02:18-02:31	1							8																
DQA1*02:01:09:01:02:01:09:02, 02:03, 02:17	1																							
DQA1*02:16								8																
DQA1*03:01:01:01, 03:01:01:03, 03:01:03-03:01:09, 03:06, 03:10, 03:12, 03:15, 03:18, 03:22, 03:25, 03:27N, 03:29-03:30, 03:36, 03:40-03:41, 03:44		2						8																
DQA1*03:02:01:01:03:04, 03:07-03:09, 03:11, 03:13-03:14, 03:17, 03:19-03:21, 03:23-03:24, 03:26, 03:28, 03:31-03:35, 03:37-03:39, 03:42-03:43		2																						
DQA1*03:05, 03:16		2						?																
DQA1*05:01:01:01:05:01:02, 05:01:04-05:01:07, 05:01:09-05:01:11, 05:18-05:19, 05:23, 05:31, 05:33, 05:38, 05:47, 05:49, 05:54Q-05:57, 05:70-05:71, 05:74			3		5					11														
DQA1*05:01:08:01:05:01:08:02, 05:48			3		5																			
DQA1*05:02			3	4	5	?				?														
DQA1*05:03:01:01:05:03:02, 05:06:01:01:05:07			3	4	5					11														
DQA1*05:04			3	?		?				?	?													
DQA1*05:05:01:01:05:05:09, 05:05:12:01:05:05:16:02, 05:08, 05:11:01:01:05:11:01:02, 05:13-05:14, 05:20, 05:24-05:25, 05:29Q, 05:35:02, 05:53, 05:58-05:59, 05:65:01:01:05:65:01:02, 05:72-05:73			3		5	6				10														
DQA1*05:05:10			3		6					10														
DQA1*05:05:11, 05:51					5	6				10														
DQA1*05:05:17			3		5	6				10	11													
DQA1*05:09:01:01:05:09:01:02			3		5	6	7																	
DQA1*05:10						?				?														
DQA1*05:12, 05:17N, 05:21, 05:26, 05:28, 05:39, 05:42-05:46, 05:50, 05:60, 05:62, 05:64, 05:66-05:69			3		5	?				?														
DQA1*05:15N, 05:22, 05:27, 05:35:01, 05:40-05:41, 05:52, 05:61, 05:63			3		5	?				?	11													
DQA1*05:16, 05:30, 05:32, 05:34, 05:37			3		5	?				?	?													
DQA1*05:36N					5					11														
DQB1*02:01:01:01:02:01:06, 02:01:08-02:01:09, 02:01:11-02:01:13, 02:01:15-02:01:20, 02:01:22-02:01:27, 02:01:29-02:01:33, 02:01:35, 02:01:37, 02:01:39-02:01:45, 02:04, 02:07:01-02:09, 02:14:01:01-02:14:02, 02:27, 02:53Q, 02:57, 02:59, 02:63, 02:79, 02:81-02:83, 02:85-02:88, 02:90-02:94, 02:96N, 02:98-02:109, 02:111-02:112, 02:114-02:115, 02:118-02:119, 02:123, 02:125, 02:128-02:130, 02:132N-02:136, 02:139-02:140, 02:148-02:149, 02:151-02:152, 02:154-02:155, 02:157-02:160, 02:163N-02:164, 02:166, 02:168, 02:170, 02:173-02:174, 02:177N-02:178, 02:182, 02:184-02:186, 02:189-02:193, 02:196-02:198, 02:202, 02:208-02:209												12		14		16								
DQB1*02:01:07, 02:01:21, 02:01:28, 02:01:34, 02:01:38, 02:03:02, 02:181, 02:188												12		14										
DQB1*02:01:36, 02:48, 02:195, 02:205												12				16								
DQB1*02:02:01:01:02:02:10, 02:02:12-02:02:23, 02:06, 02:10-02:12, 02:20:01:01N-02:20:01:02N, 02:26, 02:50, 02:62, 02:64-02:65, 02:71, 02:80, 02:84, 02:89:01-02:89:02, 02:97, 02:110, 02:113, 02:117, 02:120-02:122, 02:124, 02:126-02:127, 02:131, 02:137-02:138, 02:141-02:146, 02:150, 02:153, 02:156, 02:161-02:162N, 02:165, 02:167N, 02:169, 02:171Q, 02:175-02:176N, 02:179, 02:183N, 02:187, 02:194N, 02:199-02:201, 02:203-02:204N, 02:206N-02:207, 02:210-02:211												12	13			16								
DQB1*02:02:11, 02:95, 02:116, 02:147, 02:172												12	13											
DQB1*02:05, 02:13, 02:15-02:16, 02:18N-02:19, 02:21-02:25, 02:28-02:36, 02:38-02:39, 02:41-02:47, 02:49, 02:51-02:52, 02:54-02:56, 02:58N, 02:60-02:61, 02:66-02:70, 02:73-02:76, 02:78												12	?	14		16								
DQB1*02:17, 02:37												12	?	14										
DQB1*02:40													?	14		16								
DQB1*02:72, DQB1*06:320														14		16								
DQB1*02:77												12	?	14	15									
DQB1*02:180												12	13		15									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	

Negative Control

Table with columns for Well No. (1-23) and rows containing alphanumeric codes and their corresponding values. The table is divided into sections, including 'Negative Control' and 'Well No.'.

DQB1*03:189										14	16	17	?	22	
DQB1*03:308, 03:413										14	16	17	19		
DQB1*03:335										14				20	
DQB1*03:349, 03:447											16	17	19	22	
DQB1*03:381													19	20	21
DQB1*03:443, DQB1*06:246										14			18		
DQB1*04:01:01:01-04:01:02, 04:01:04-04:02:01:01, 04:02:01:04-04:02:15, 04:02:17-04:02:24, 04:04-04:08, 04:10-04:43, 04:45-04:62, 04:64-04:69, 04:72-04:74, 04:76-04:80, 04:82-04:95, DQB1*05:01:01:01-05:02:10, 05:02:12-05:13, 05:15-05:83, 05:85-05:195, 05:197-05:210, 05:212-05:283N, 05:285-05:308, 05:310-05:318, DQB1*06:01:01:01-06:02:04, 06:02:06-06:02:42, 06:02:44-06:02:46, 06:02:48-06:02:52, 06:02:54-06:03:09, 06:03:11-06:03:25, 06:03:27-06:03:32, 06:03:35-06:04:06, 06:04:08-06:04:15, 06:04:17-06:18:02, 06:20-06:28, 06:30-06:37, 06:39-06:50, 06:51:02-06:62, 06:64-06:65, 06:67-06:85, 06:88:01:01-06:95, 06:96:02-06:101, 06:105-06:118:03, 06:119-06:122, 06:124-06:138, 06:140-06:167, 06:169-06:171, 06:173-06:189, 06:190:02-06:217, 06:219, 06:221-06:226, 06:228-06:245, 06:247-06:283, 06:285-06:319:02, 06:321, 06:323-06:336, 06:338-06:349, 06:351-06:362, 06:364-06:365, 06:367-06:376, 06:378-06:386, 06:388-06:394N, 06:396-06:423N, 06:425-06:427, 06:429-06:466										14					
DQB1*04:03:01-04:03:02, DQB1*06:02:05, 06:03:26, 06:03:34, 06:04:07, 06:19:01-06:19:02, 06:87, 06:190:01, 06:322:03										14		17			
DQB1*06:02:43, 06:03:33, 06:51:01, 06:66, 06:96:01, 06:118:04, 06:172										14	15				
DQB1*06:29, 06:123										14	16				22
DQB1*06:63										14	16	17			
DQB1*06:139, 06:337										14	16	17			22

Negative Control