

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
DQA1 or DQB1 allele^{1,2}																							
DQA1*01:01:01-01:01:01:01:01:03, 01:01:01:05-01:65									9														
DQA1*02:01:01:01-02:02N, 02:04-02:05, 02:07-02:15, 02:18	1							8															
DQA1*02:03, 02:17	1																						
DQA1*02:16								8															
DQA1*03:01:01:01, 03:01:03-03:01:06, 03:06, 03:10, 03:12, 03:15, 03:18		2						8															
DQA1*03:02:01:01-03:04, 03:07-03:09, 03:11, 03:13-03:14, 03:17, 03:19-03:20		2																					
DQA1*03:05, 03:16		2						?															
DQA1*05:01:01:01-05:01:02, 05:01:04-05:01:06, 05:18-05:19, 05:23, 05:31, 05:38			3		5						11												
DQA1*05:02			3	4	5	?				?	?												
DQA1*05:03:01:01-05:03:02, 05:06:01:01-05:07			3	4	5						11												
DQA1*05:04			3	?		?				?	?												
DQA1*05:05:01:01-05:05:06, 05:08, 05:11, 05:13-05:14, 05:20, 05:24, 05:29Q			3		5	6					10												
DQA1*05:09			3		5	6	7																
DQA1*05:10						?					?												
DQA1*05:12, 05:17N, 05:21, 05:25-05:26, 05:28			3		5	?					?												
DQA1*05:15N, 05:22, 05:27, 05:35			3		5	?					?	11											
DQA1*05:16, 05:30, 05:32-05:34, 05:37			3		5	?					?	?											
DQA1*05:36N					5						11												
DQB1*02:01:01:01-02:01:06, 02:01:08-02:01:20, 02:01:22-02:01:27, 02:01:29-02:01:33, 02:01:35, 02:01:37, 02:01:39, 02:04, 02:07:01-02:09, 02:14:01:01-02:14:02, 02:27, 02:53Q, 02:57, 02:59, 02:63, 02:79, 02:81-02:83, 02:85-02:88, 02:90-02:94, 02:96N, 02:98-02:109, 02:111-02:112, 02:114-02:115, 02:118-02:119, 02:123, 02:125, 02:128-02:130, 02:132N-02:136, 02:139-02:140, 02:148-02:149, 02:151-02:152, 02:154-02:155, 02:157-02:160, 02:163N-02:164, 02:166, 02:168, 02:170, 02:173-02:174, 02:177N-02:178, 02:182												12		14		16							
DQB1*02:01:07, 02:01:21, 02:01:28, 02:01:34, 02:01:38, 02:03:02, 02:181												12		14									
DQB1*02:01:36, 02:48												12				16							
DQB1*02:02:01:01-02:02:10, 02:02:12-02:02:14, 02:06, 02:10-02:12, 02:26, 02:50, 02:62, 02:64-02:65, 02:80, 02:84, 02:89:01-02:89:02, 02:97, 02:110, 02:113, 02:117, 02:120-02:122, 02:124, 02:126-02:127, 02:131, 02:137-02:138, 02:141-02:146, 02:150, 02:153, 02:156, 02:161-02:162N, 02:165, 02:167N, 02:169, 02:171Q, 02:175-02:176N, 02:179												12	13			16							
DQB1*02:02:11, 02:95, 02:116, 02:147, 02:172												12	13										
DQB1*02:05, 02:13, 02:15-02:16, 02:18N-02:19, 02:21-02:25, 02:28-02:36, 02:38-02:39, 02:41-02:47, 02:49, 02:51-02:52, 02:54-02:56, 02:58N, 02:60-02:61, 02:66-02:70, 02:73-02:76, 02:78												12	?	14		16							
DQB1*02:17, 02:37												12	?	14									
DQB1*02:20N, 02:71												12	13	14		16							
DQB1*02:40													?	14		16							
DQB1*02:72, DQB1*06:320														14		16							
DQB1*02:77												12	?	14	15								
DQB1*02:180												12	13		15								
DQB1*03:01:01:01-03:01:12, 03:01:01:14-03:01:07, 03:01:09-03:01:49, 03:10:01-03:10:03, 03:13, 03:16, 03:21-03:22:02, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:47, 03:49:01:01-03:51, 03:55, 03:73, 03:83-03:84N, 03:92-03:94, 03:114-03:116, 03:119, 03:139, 03:169, 03:172, 03:180, 03:182, 03:186, 03:191, 03:195-03:198:02, 03:206, 03:231, 03:241-03:243, 03:246, 03:252-03:254, 03:264, 03:266, 03:271, 03:276N, 03:281, 03:284-03:285, 03:288, 03:290-03:294, 03:297, 03:302-03:303N, 03:305-03:307, 03:309, 03:311-03:312, 03:317:01-03:317:02, 03:326, 03:328, 03:330-03:331, 03:338N, 03:340N-03:342, 03:350, 03:354N-03:355, 03:358N, 03:360-03:361, 03:366, 03:370, 03:372-03:373, 03:377-03:378, 03:380, 03:385N, 03:389-03:391, 03:396, 03:399N-03:400N, 03:404, 03:407N-03:408, 03:417-03:421, 03:423-03:424, 03:426-03:428, 03:430-03:432, 03:434-03:436, 03:439, 03:448, 03:451, 03:454														14				19		21			
DQB1*03:01:08, 03:05:01-03:05:05, 03:72, 03:100, 03:132, 03:181, 03:250, 03:272, 03:314, 03:346, DQB1*04:01:03, 04:02:16, 04:09, 04:63, 04:75														14					19				

Negative Control

