

*05:12			3															23														
*05:13			3																25													
*05:14																		24														
*05:15			3																25													
*05:16			3																25													
*06:01:01-06:01:06			3										15											27								
*06:02:01			3																			26	27	28								
*06:02:02			3									12										26	27	28								
*06:03:01			3																			26			29	30	31					
*06:03:02			3									12										26			29	30	31					
*06:04:01, 06:04:03			3																							32						
*06:04:02			3																						30	32						
*06:05:01			3																				?									
*06:05:02			3																				?	?								
*06:06			3																				?	?								
*06:07:01			3																				?		29	30	32					
*06:07:02			3																						29	32						
*06:08:01			3																				?		30	31						
*06:08:02			3																				?		31							
*06:09			3																													
*06:10			3																				?	27								
*06:11:01			3																				?	27	30							
*06:11:02			3																				?	27	29	30	31					
*06:12			3																							31						
*06:13			3																				?	27								
*06:14:01			3																				?	28	29	31						
*06:14:02			3																				?	28								
*06:15			3																				?	28								
*06:16			3																				?	27	28							
*06:17			3																				?			32						
*06:18			3																				?	27								
*06:19			3																				?	27	28							
*06:20			3																				?	27	28							
*06:21			3																				?			31	32					
*06:22			3																				?									
*06:23			3														17						?	28								
*06:24			3																				?	27	28							
*06:25			3																				?			32						
*06:26N			3																				?		29	30	31					
*06:27			3																				?									
*06:28			3														18						?		29	30	31					
*06:29			3												8								?	27								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*06:30			3																						?		29	30				
*06:31			3																						?		29	30	31			
*06:32			3																						?		29	30				
*06:33			3																					?	27	28						
*06:34			3														17													32		
*06:35, 06:45 ³			3																					?	27							
*06:36			3																												32	
*06:37			3																						?	27	28					

INTERPRETATION TABLE

DQB1 high resolution SSP typing

Amplification patterns of the DQB1*02:01 to DQB1*06:52 alleles

Well ^{4,5}																																																
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	Neg. Control	Length of spec. PCR product(s)															
170	130	255	210	180	185	130	180	150	95	300	205	185	195	135	100	155	110	160	110	130	210	160	155	110	165	155	110	240	200	245	125			260	95	190	190	115	220	100	185	185	95	190	200	225	180	
430	430	515	430	430	430	430	430	430	430	430	515	515	430	515	515	430	430	430	430	430	430	430	430	430	430	430	430	430	430	430	430			430	430	430	430	430	430	430	430	430	430	430	430	430	430	430
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64			Length of int. pos. control ¹														
																																Well No.																
																																DQB1 allele ²																
																																*02:01:01-02:01:05																
																																*02:02																
																																*02:03																
																																*02:04																
																																*02:05																
																																*02:06																
																																*02:07																
																																*03:01:01:01-03:01:02, 03:01:05-03:01:06																
																																*03:01:03																
																																*03:01:04																
																																*03:02:01-03:02:05																
																																*03:03:02:01-03:03:04																
																																*03:04																
																																*03:05:01-03:05:04																
																																*03:06																
																																*03:07																
																																*03:08																
																																*03:09																
																																*03:10																
																																*03:11																
																																*03:12																
																																*03:13																
																																*03:14																
																																*03:15																
																																*03:16																
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	Well No.																
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	Well No.																
																																*03:17																
																																*03:18																

