

Well No.	DR ser ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*14:168	-																	w	19	20													27	
*14:169	-																	17	19	20						24							27	
*14:170	-							7										17	19							24							? 28	
*14:173	-																	15	18	19	20					22							27	
*14:175	-																		19	20	21	23											27	
*14:179	-							6	7										17	19													26 ? 28	
*14:183	-								7										17														? 28	
*14:186, 14:208	-																		18	20													26 27	
*14:200	-								7											19	20												28	
*14:217	-																		18	19	20												27	
*14:221	-								7										17	19													26	
*14:222N	Null								7									15			20												27	
*14:235	-																	15	18	19	20												27	
*14:250	-																		18															?
*15:01:01:01-15:09, 15:11:01-15:20, 15:22-15:23, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:57:01, 15:58-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:157, 15:159N-15:166, 15:168-15:186, 15:188:01:01-15:213, 15:215-15:216	DR2, DR15, Null, -																																	
*15:10	DR2																																	25
*15:21	DR2																			w														
*15:24	-																				w													
*15:25, 15:37:01, 15:57:02, 15:100, 15:104:01	-																																	
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-																																	
*15:141	-																																	
*15:158, 15:187	-																																	
*15:167, 15:214	-																																	
*16:01:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:65, 16:67-16:73	DR16, Null, -																																	
*16:04:01-16:04:02, 16:18	DR16, -																																	
*16:66	-																																	
DRB3*01:01:02:01-01:41, DRB3*01:43-01:98, DRB3*01:100-01:112, DRB3*02:01-02:26, DRB3*02:28-02:57, DRB3*02:59-02:86, DRB3*02:88-02:110, DRB3*02:112-02:130, DRB3*02:132-02:159, DRB3*02:161-02:189, DRB3*03:01:01:01-03:21, DRB3*03:23-03:61	DR52, Null, -																																	
DRB3*01:42	-																																	
DRB3*01:99, DRB3*02:87	-																																	
DRB3*02:27	-																																	
DRB3*02:58	-																																	
DRB3*02:131	-																																	
DRB3*02:160	-																																	
DRB3*02:190	-																																	
Well No.	DR ser ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB4*01:01:01:01, DRB4*01:01:01:03-01:03:01:01, DRB4*01:03:01:03-01:03:01:12, DRB4*01:03:01:14-01:13, DRB4*01:15-01:30, DRB4*01:32-01:65N, DRB4*01:67-01:167, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																																	
DRB4*01:03:01:02N, DRB4*01:03:01:13N, DRB4*01:14N	Null																																	
DRB4*01:31	-																																	
DRB4*01:66	-																																	
DRB5*01:01:01:01-01:07, DRB5*01:09-01:126, DRB5*01:128-01:129, DRB5*02:02:01-02:25N, DRB5*02:27-02:30, DRB5*02:32-02:33, DRB5*02:35-02:37	DR51, Null, -																																	
DRB5*01:08:01N-01:08:02N, DRB5*01:127N, DRB5*01:130, DRB5*02:26N	Null, -																																	
DRB5*02:31, DRB5*02:34	-																																	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control

Negative Control