

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DRB1 allele^{1,2,3}																								
*01:01:01:01:01:35, 01:01:37-01:01:42, 01:01:44-01:01:46, 01:38, 01:41, 01:45, 01:47-01:49, 01:53, 01:56, 01:59-01:60, 01:62:02N-01:66, 01:69-01:73, 01:75, 01:78-01:80, 01:82, 01:84, 01:91Q-01:92, 01:95, 01:98, 01:100, 01:103-01:110, 01:113, 01:115, 01:117, 01:120-01:121, 01:124-01:125, 01:128-01:130, 01:132-01:133, 01:135, 01:137, 01:139, 01:141	1		3																					
*01:01:36	1	2	3																					
*01:01:43	1		3										13	14										
*01:02:01:01:01:02:05, 01:02:07-01:02:13, 01:43, 01:46, 01:57, 01:76, 01:81, 01:83, 01:86-01:88, 01:93-01:94, 01:96-01:97, 01:122-01:123, 01:126, 01:136, 01:138, 01:140		2	3																				23	
*01:02:06, 01:58, 01:116			3																				23	
*01:03:01:01-01:03:05, 01:102, 01:127	1			4			7																	
*01:04			3		5			9																
*01:05	1		3			6																		
*01:06			3					9	10															
*01:07	1		3				8																	
*01:08	1		3					9																
*01:09	1		3						10															
*01:10	1		3	4						11														
*01:11:01-01:11:02	1		3		5																			
*01:12, 01:37	1		3							12														
*01:13	1		3									13												
*01:14	1		3										14											
*01:15	1					7		10																
*01:16	1		3		5									15										
*01:17	1		3			7																		
*01:18:01-01:18:02	1		3						11															
*01:19	1		3												16									
*01:20:01, 01:85			3					9																
*01:20:02		2	3					9																
*01:21	1		3											15	16									
*01:22	1		w													17								
*01:23, 01:111		2											13	14									23	
*01:24:01-01:24:02	1												14	16										
*01:25	1		3													18								
*01:26		2	3									12											23	
*01:27	1		3				8										17							
*01:28	1		3										13	15										
*01:29:01	1		3		5	6																		
*01:29:02	1				6																			
*01:30	1		3														17							
*01:31	1		3		6							12												
*01:32	1		3																			21		
*01:33N	1																			20				
*01:34		2	3																		21		23	
*01:35			3		5																	23		
*01:36	1		3																			22		
*01:39N	1			4			7											19						
*01:40N, 01:67 ⁴	1		3														19							
*01:42				4			7																23	
*01:44:01-01:44:02		2	3							11													23	
*01:50	1		3				8											18						
*01:51	1												13	14										
*01:52N	1		3							10													22	
*01:54		2												14	16								23	
*01:55	1				5										15									
*01:61				4			7		9															
*01:62:01N	1		3	4																		21		
*01:68N	1		3							11			14											
*01:74		2	3		5	6																	23	
*01:77, 01:119	1		3																19	20				
*01:89	1			4	5		7																	
*01:90	1													15										
*01:99	1			4																				
*01:101			3		5				9					15										
*01:112, 01:131N, 01:134	1																							
*01:114		2	3				7																23	
*01:118		2	3		5	6	7																23	
*10:40, DRB5*01:60																	18							
*13:165, DRB3*02:63																	17							
*14:112								8												19	20			
*14:196																								
DRB3*01:47								8									17							
DRB3*03:01:08			3		5																			
DRB4*01:132																				19				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control