

DR11 2017 7F2

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	
DRB1 allele ^{1,2,3}																																																	
*11:01:01:01-11:01:11, 11:01:13-11:01:27, 11:01:29, 11:01:95, 11:100, 11:117, 11:121, 11:126, 11:133-11:134, 11:140-11:141, 11:144, 11:155, 11:157-11:158, 11:162, 11:166, 11:172, 11:177-11:178, 11:186, 11:188, 11:195, 11:199, 11:204-11:205, 11:208, 11:212, 11:215	1		3		5		7																						30																				
*11:01:12	1		3		5		7																		25				30																				
*11:01:28			3		5		7																							30																			
*11:02:01-11:02:04	1			4		6		8																				28																					
*11:02:05	1			4		6		8																	w			28																					
*11:03:01-11:03:02, 11:139, 11:176	1		3			6		8																																									
*11:04:01-11:04:03, 11:04:05-11:04:12, 11:04:14-11:04:15, 11:123, 11:146, 11:163, 11:181, 11:194, 11:201, 11:203, 11:206-11:207, 11:210-11:211, 11:216	1		3		5			8																						30	31																		
*11:04:04	1		3		5			8																	25				30	31																			
*11:04:13					5																								30	31																			
*11:05		2	3		5		7																						30																				
*11:06:01-11:06:03, 11:129, 11:174	1		3		5			8			11														24				30	31																			
*11:07:01-11:07:02	1							8	9	10								18																															
*11:08:01-11:08:03	1				5		7				11																																						
*11:09	1		3		5		7						13												25	26			30																				
*11:10:01-11:10:02	1		3		5		7								15				20						25	26			30																				
*11:11:01, 11:11:03	1		3			6	7																					27	30																				
*11:12:01, 11:12:03	1		3		5		7								15				20										30																				
*11:12:02	1		3		5		7								15				20						25				30																				
*11:13:01-11:13:02	1				5			8							15				20						25	26																						41	
*11:14:01-11:14:02	1			4		6	7																																										
*11:15	1		3		5		7									16				22									30																				
*11:16	1			4		6		8					13												25	26																							
*11:17	1				5			8						14	15		17		20						25	26																							
*11:18	1			4		5		8																																									

Negative Control

DR11 2017 7F2

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*04:15			3		5			8										21												31																					
*04:61, 04:145					5		7											21																								41									
*04:132																											27																								
*04:138						6		8										21									28																								
*04:154					5		8											17																									41								
*04:170					5		7											17																									41								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*08:05, 08:24:01-08:24:02, 12:66, 14:04:01-14:04:03, 14:28, 14:31, 14:52, 14:68:01-14:68:02, 14:71, 14:120, 14:126:01-14:126:02, 14:145, 14:152N, 14:175, 14:180																																																			47
*08:17, 12:16:02-12:16:03, 13:05:01-13:05:03, 13:14:01-13:14:03, 13:21:01-13:21:02, 13:46, 13:50:01-13:50:03, 13:132, 13:197, 13:199, 13:203, 16:09:01-16:09:02, <i>DRB5*01:34-01:35</i>																															30																				
*08:18, 08:29																														29																			47		
*08:20, 13:164													14						19											31																					
*08:28, 12:02:01:01-12:02:03, 12:02:05-12:02:07, 12:13, 12:15, 12:18-12:20, 12:23, 12:26-12:27, 12:31N-12:33, 12:37, 12:42-12:45, 12:50-12:52, 12:55-12:56, 12:60N-12:61, 12:64-12:65, 12:67, 13:11:01-13:11:02, 13:42, 13:169, 16:33																																																			
*08:31		2	3		5			8																																									47		
*08:40																												27	29																					47	
*08:41		2	3		5		7																																										47		
*08:63		2			5		7			10								17											29																		45				
*08:75		2	3		5		8			10								17																												45					
*12:01:02, 14:73																																																	47		
*12:02:04																																																	47		
*12:04		2			5			8																		25																								47	

Negative Control

DR11 2017 7F2

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
*12:16:01																																															47			
*12:49		2	3		5			8																	25																						47			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
*13:02:01:01-13:02:03, 13:02:05-13:02:09, 13:02:11-13:02:13, 13:23:01-13:23:02, 13:31, 13:41, 13:66:01-13:66:02, 13:73, 13:96:01-13:97:02, 13:99, 13:104, 13:123-13:124, 13:126, 13:128, 13:139, 13:143, 13:145, 13:147, 13:155, 13:165, 13:168, 13:170-13:172, 13:202, 13:207-13:212, 13:220, 13:225, 13:236-13:237, 13:239-13:240				4																								27	29																					
*13:02:04, 13:109, 13:137N, 13:142N				4																																														
*13:03:01-13:03:06, 13:33:01-13:33:03, 13:36-13:38, 13:65, 13:85, 13:90, 13:95, 13:101, 13:115, 13:120, 13:122, 13:167, 13:188, 13:194, 13:198, 13:219, 13:227				4									14														27	29																						
*13:06				4																																														
*13:08, 13:19, 13:72, 13:76, 13:84				4									14							20								28																						
*13:09, 13:129, 13:192																					22																													
*13:12:01-13:12:04, 13:228				4									14																		29																			
*13:13, 14:63, 14:67				4									14							19										29																				
*13:16				4																19																														
*13:17, 13:116, 13:175, 13:214																													28																		47			
*13:18, 13:144																				19												30	31																	
*13:20, 13:71												11																		28																				
*13:24, 13:75, 13:111																														28			30																	
*13:25, 13:56, 13:224												11																																						
*13:29												11																																						
*13:30, 13:82, 13:157, 13:161				4																									27																					
*13:32, 13:40, 13:48, 13:53, 13:70, 13:89, 13:94:01				4									14																																					

Negative Control

DR11 2017 7F2

*14:74, 14:84, 14:173												11		14				19	20											31																					
*14:77												11		14		16		19																																	
*14:78			4											14				19																																	
*14:86, 14:153														14				19	20			24																													
*14:96														14					20			24																													
*14:97														14		16		19	20																																
*14:99												11		14				19	20			24																													
*14:105														14				19	20														31																		
*14:110														14		16		19																																	
*14:115												11		14				19																					40												
*14:128																		19																																	
*14:138																																									47										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*14:148		2													15				19	20						25	26																					47			
*14:150																																																			
*14:161														14																																					
*15:99							7																				25			29																					
*15:109								8																			25																								
*16:16					5		7																				25																								
*16:27			3		5		7																				25																								
DRB3*01:14															14		16																																		
DRB3*01:25															?		?																																		
DRB3*01:42							7		9	10		12				15			18			21					25	26																							
DRB3*02:18							7					12							18									25																							
DRB3*02:54															?		?					?																													
DRB5*01:13			3		5		7																																												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			

Negative Control