

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DRB1 allele ^{1,2,3}																								
*12:01:01:01-12:01:01:14, 12:01:04-12:01:12, 12:30, 12:34-12:35, 12:40-12:41, 12:54, 12:59, 12:68, 12:70, 12:72N, 12:77, 12:79, 12:82, 12:84, 12:86N-12:87, 12:89-12:91, 12:95, 12:99	1	2		4	5	6	7	8	9												21			
*12:01:02-12:01:03	1	2		4	5	6	7	8													21			
*12:02:01:01-12:02:01:11, 12:02:03-12:02:06, 12:02:08-12:02:13, 12:32, 12:44, 12:50, 12:61, 12:67, 12:69, 12:71, 12:74N-12:75, 12:78, 12:80, 12:85, 12:92-12:94, 12:97-12:98N, 12:100-12:101, 12:103	1	2		4	5	6	7		9	10											21			
*12:02:02:01-12:02:02:03, 12:55, 12:81	1	2		4	5	6	7			10											21			
*12:02:07, 12:51-12:52	1	2		4	5	6	7		9	10														
*12:03:02	1	2		4	5			8			11										21			
*12:03:03	1	2		4	5			8	9		11										21			
*12:04	1	2		4		6	7	8	9			12									21			
*12:05	1	2			5	6	7	8					13								21			
*12:06	1	2		4	5	6	7	8	9					14							21			
*12:07	1	2		4	5	6	7		9						15						21			
*12:08	1	2		4	5	6		8	9							16					21			
*12:09	1	2	3	4		6	7	8	9												21			
*12:10	1	2		4	5	6	7	8	9								17				21			
*12:11	1	2		4	5	6	7	8	9									18			21			
*12:12	1	2		4	5	6	7		9										19		21			
*12:13	1	2		4	5	6	w		9	10										20	21			
*12:14	1	2			5	6	7	8					13											
*12:15	1	2			5	6	7			10			13								21			
*12:16:01, 12:16:03	1	2		4	5					10											21	22		
*12:16:02	1	2		4	5				9	10											21	22		
*12:17	1	2		4	5	6	7	8	9												21		23	
*12:18	1	2		4		6	7		9	10					15						21			
*12:19:01	1	2		4	5				9	10	11										21			
*12:19:02	1	2		4	5					10	11										21			
*12:20	1	2		4	5	6	7		9	10			13								21			
*12:21		2		4	5	6	7		9	10											21	22		
*12:22	1			4	5			8													21	22		
*12:23	1	2		4	5	6			9	10						16				20	21			
*12:24N	1	2		4	5	6	7	8	9				13								21			
*12:25	1	2		4	5	6	7	8	9				13				17				21			
*12:26	1	2		4	5	6	7		9	10								18			21			
*12:27	1	2		4	5	6			9	10						16					21			
*12:28	1	2		4	5	6	7	8	9									18	19		21			
*12:29, 12:36, 12:63	1	2		4	5	6	7	8	9															
*12:31N	1	2		4	5	6	7		9	10						16					21			
*12:33	1	2		4		6	7		9	10											21			
*12:37, 12:64-12:65	1	2		4	5	6			9	10											21			
*12:38		2			5	6	7	8														22		
*12:39:01	1	2		4	5			8	9												21	22		
*12:39:02	1	2		4	5			8													21	22		
*12:42, 12:83							7		9															
*12:43	1			4	5	6	7			10											21			
*12:45	1	2		4	5	6	7		9	10									19					
*12:46	1	2		4		6	7	8	9												21			
*12:47	1	2		4		6	7	8	9						15						21			
*12:48	1	2	3	4		6	7	8	9								17				21			
*12:49	1	2		4		6	7		9	10		12									21			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*12:53		2		4	5	6	7	8	9												21	22		
*12:56	1	2			5	6	7			10			13											
*12:57																				20				

Negative Control

*08:75										10	11	12													
*08:81			3																						
*08:87				4						10					16							21			
*08:92, 08:102		2																							
*08:103	1		3							10															
*08:113	1				5					10															
*11:67	1									10	11	12													
*11:76, 11:114, 11:183, 11:196, 11:302, 13:34, 13:64, 13:136, 13:174, 13:204, 13:221, 13:250, 14:41, 14:77, 14:110, 14:218, DRB3*01:36, DRB3*02:37, DRB3*02:83, DRB3*02:112, DRB3*03:12															16										
*11:193:01-11:193:02	1	2								10		12													
*11:209	1	2								10	11	12													
*11:229, 11:270	1	2									11	12													
*13:17, 13:175, 13:214	1	2	3								11														
*13:116, 14:52		2	3								11														
*13:145																						22			
*13:163, DRB5*02:07															17										
*14:04:01:01-14:04:09, 14:50:01-14:50:02, 14:76, 14:79, 14:107, 14:120, 14:126:02, 14:145, 14:152N, 14:180, 14:196, 14:201, 14:205:01-14:206, 14:210Q, 14:220, 14:224, 14:229, 14:233, 14:241, 14:249- 14:250											11														
*14:11:01:01-14:11:01:02, 14:148											11	12													
*14:28						6																			
*14:31		2									11														
*14:126:01, 14:251			3								11														
*14:138	1										11														
*15:102														w	16										
DRB3*02:02:24														14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Negative Control