

INTERPRETATION TABLE

DRB1*14 SSP subtyping

Amplification patterns of the DRB1*14:01 to DRB1*14:109 alleles

	Well ⁵																							
	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.	210	175	140	85	260	260	225	170	130	175	85	225	115	60	185	105	220	85	145	110	170	145	145	75
PCR product(s)									250					130						160	215			125
Length of int.	515	430	430	430	430	515	430	430	430	430	430	515	430	515	430	430	430	430	430	430	215	215	145	145
pos. control ¹																								
5'-primer(s) ²	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-AgA ³ 37(196)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-ggg ³ 13(125)	5'-CAA ³ 16(135)	5'-ACC ³ 11(118)	5'-gTT ³ 16(133)	5'-gTC ³ 13(125)	5'-CAG ³ 34(189)	5'-AgA ³ 37(196)	5'-AgA ³ 37(196)	5'-gAT ³ 57(258)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-gTT ³ 37(197)	5'-gTT ³ 16(133)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)	5'-gTC ³ 13(125)
								5'-gTT ³ 16(133)	5'-CTA ³ 57(256)	5'-ACA ³ 13(125)	5'-AgC ³ 60(265)			5'-AgA ³ 37(196)	5'-gTT ³ 37(197)					5'-CAA ³ 34(189)				
														5'-AgC ³ 60(265)										
3'-primer(s) ³	5'-TCC ³ 70(296)	5'-CAG ³ 57(257)	5'-CTg ³ 70(295)	5'-CTC ³ 28(171)	5'-CAC ³ 86(344)	5'-CCA ³ 86(344)	5'-CAG ³ 74(307)	5'-CAG ³ 57(257)	5'-CCA ³ 86(344)	5'-CAG ³ 57(257)	5'-CCT ³ 58(260)	5'-CCT ³ 74(308)	5'-gTg ³ 60(265)	5'-gAA ³ 67(286)	5'-CAG ³ 85(341)	5'-CAA ³ 78(321)	5'-CCT ³ 71(298)	5'-CTg ³ 28(171)	5'-CTC ³ 71(298)	5'-CAG ³ 57(257)	5'-gCT ³ 57(256)	5'-ggT ³ 47(227)	5'-ggA ³ 47(227)	5'-ACT ³ 25(161)
			5'-CTT ³ 71(298)								5'-CgC ³ 74(307)	5'-CCT ³ 74(308)					5'-ggC ³ 77(316)	5'-CTg ³ 28(171)	5'-CAG ³ 74(307)	5'-CAG ³ 74(307)	5'-CgC ³ 71(298)			
												5'-CCT ³ 74(308)												5'-CgC ³ 38(199)
																								5'-CCg ³ 45(220)
																								5'-CCC ³ 55(250)
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.	210	175	140	85	260	260	225	170	130	175	85	225	115	60	185	105	220	85	145	110	170	145	145	75

PCR product(s)									250		215	170			130					215	160	215				165	125	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24				
DRB1 allele ⁴																												
*14:01:01-14:01:03	1	2				6						12	13					18								22		
*14:02			3	4	5																						22	
*14:03:01-14:03:02				4	5		7																				22	
*14:04								8					13								20							
*14:05:01-14:05:02	1					6			9			12				16		18								22		
*14:05:03	1					6			9			12						18								22		
*14:06:01-14:06:02			3	4		6																					22	
*14:07:01-14:07:02	1	2			5							12	13					18									22	
*14:08	1					6						12	13			16		18									22	
*14:09			3		5													18									22	
*14:10										10			13															
*14:11											11					16												
*14:12:01-14:12:02				4		6	7																				22	
*14:13			3	4	5																	21	22					
*14:14	1				5							12				16		18									22	
*14:15																				19	20							
*14:16		2				6					11		13					18	19							22		
*14:17			3			6												18									23	
*14:18	1			4		6						12				16											22	
*14:19			3	4	5													17									22	
*14:20				4		6																					22	
*14:21			3			6												17	18								23	
*14:22		2			5						11		13	14				18									22	
*14:23:01-14:23:03	1					6						12				16		18									22	
*14:24				4	5																	21	22					
*14:25		2			5						11			14				18		20							22	
*14:26	1	2				6						12	13					18									22	24
*14:27				4	5		7							14													22	
*14:28								8					13		15						20							
*14:29			3	4		6									15												22	
*14:30			3		5													18									23	
*14:31								8			11		13							20								
*14:32:01-14:32:02	1	2				6					11		13					18									22	
*14:33			3			6						12						18									23	
*14:34	1					6					11		13			16		18									22	
*14:35	1	2				6						12	13					18									23	
*14:36	1				5							12				16		18									22	24
*14:37						6			9									18				21	22					
*14:38	1	2				6						12	13					18									22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24				
Length of spec.	210	175	140	85	260	260	225	170	250	130	175	85	225	115	60	185	105	220	85	145	110	170	145	145	75			
PCR product(s)									250		170			130						160	215					125		

											215							215				165			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
DRB1 allele ⁴																									
*14:39	1	2			6					10	12	13						18				22			
*14:40				4	5		7												19			22			
*14:41				4	5																	22		24	
*14:42	1				5							12				16		18					23		
*14:43	1				6				9			12				16	17	18				22			
*14:44:01	1				5							12				16		18				22			
*14:44:02	1				5							12						18				22			
*14:45	1				6				9			12				16		18				22			
*14:46				3																					
*14:47				3	4	5																	22		
*14:48				3	4	5																	22		
*14:49		2		4	5						11		13										22		
*14:50								8					13								20				
*14:51				3	4	5						12											22		
*14:52				3								11													
*14:53		2			5						11			14				18		20			23		
*14:54	1	2			6							12	13					18				22			
*14:55	w	2			6	7							13					18	19			22			
*14:56	1				6				9			12				16		18				22			
*14:57										10										19					
*14:58	1	2			6							12						18		20		22			
*14:59	1				6							12				16		18				22			
*14:60	1	2			6							12	13					18				22			
*14:61								8					13												
*14:62	1	2			6				9			12	13					18				22			
*14:63				4	5		7															21	22		
*14:64	1				6							12				16		18					23		
*14:65	1				6				9									18				21	23		
*14:67				4	5		7																22		
*14:68								8					13									20			
*14:69		2			5						11							18		20		22			
*14:70	1	2			6							12	13									22			
*14:71								8					13								20				
*14:72	1				6							12	13			16		18					23		
*14:73								8					13	14							20				
*14:74		2			6						11		13					18				22			
*14:75	1	2			6							12	13					18				22			
*14:76								8													20				
*14:77				4	5		7												19			22		24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Length of spec.	210	175	140	85	260	260	225	170	250	130	175	85	225	115	60	185	105	220	85	145	110	170	145	145	75
PCR product(s)									250		170			130						160	215				165
											215									215					165

*13:66:01-13:66:02					5										17	18			21	23				
*13:67, 13:103																								
*13:72					6			9								18	19			22				
*13:76					6				12				16		18	19			22					
*13:77					6										18					23				
*13:79					6										18					23				
*13:81, 13:89, 13:94					6			9							17	18			21	22				
*13:83					6										18	19				23				
*13:84					6										18					22				
*13:85			3	4	5										17					22				
*13:86					5															23				
*13:92					6										18					23	24			
*13:93					6			9							18			21		23				
*13:101					5										17			21	22					
*13:104				4	5															23				
*13:107					5										18	19				23				
*15:03:01:01- 15:03:01:02, 15:23																								
*16:08													14											
<i>DRB3*01:08</i>																								
<i>DRB3*01:14</i>					5										18					22	24			
<i>DRB3*02:16</i>																								
DRB1 allele ⁴																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

250	210			230	220	175			145							155	250	165		200				220	165		190	PCR product(s)			
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.							
																													DRB1 allele ⁴		
	26							33																				*14:01:01-14:01:03			
																													*14:02		
																													*14:03:01-14:03:02		
																													*14:04		
																													*14:05:01-14:05:02		
																													*14:05:03		
					30																								*14:06:01-14:06:02		
25								33																					*14:07:01-14:07:02		
								33																					*14:08		
																													*14:09		
																													*14:10		
																													*14:11		
					30																								*14:12:01-14:12:02		
							32																						*14:13		
																													*14:14		
																													*14:15		
								33																					*14:16		
					30																								*14:17		
					30																								*14:18		
							32																						*14:19		
					30																								*14:20		
							32																						*14:21		
25								33																					*14:22		
																													*14:23:01-14:23:03		
																													*14:24		
25								33																					*14:25		
								33																					*14:26		
																													*14:27		
																													*14:28		
					30																								*14:29		
																													*14:30		
																													*14:31		
								33																					*14:32:01-14:32:02		
					30																								*14:33		
								33																					*14:34		
							31	33																					*14:35		
																													*14:36		
																													*14:37		
																													*14:38		
				28				33																							
25	120	250	220	160	125	190	80	100	175	205	250	130	175	215	120	215	115	225	95	215	125	105	60	65	Well No.						
210			230	175			145								155	250	165		200			165		190	Length of spec.						
																												PCR product(s)			

250				220														245						220		225	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.			
																								DRB1 allele ⁴			
								33																			*14:39
																											*14:40
																											*14:41
																											*14:42
																											*14:43
25																											*14:44:01
25																											*14:44:02
																								48			*14:45
		27																									*14:46
			28																								*14:47
				29										39													*14:48
25								33																			*14:49
			28						34	35																	*14:50
														39													*14:51
					30					35																	*14:52
25						31		33																			*14:53
								33																			*14:54
								33																			*14:55
25																											*14:56
																											*14:57
								33																			*14:58
					30									39							45						*14:59
				29				33																			*14:60
																											*14:61
								33																			*14:62
						32					36													47			*14:63
					30									39													*14:64
																											*14:65
											36													47			*14:67
25									34																		*14:68
25								33																			*14:69
								33												44							*14:70
									34	35					40												*14:71
					31																						*14:72
										35																	*14:73
								33																			*14:74
								33								41											*14:75
										35							42										*14:76
																											*14:77
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.			
120	250	220	160	125	190	80	100	175	205	250	130	175	215	120	215	115	225	95	215	125	105	60	65	Length of spec.			
210			230	175			145							155	250	165		200			165		190	PCR product(s)			
250				220												245					220		225				

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
																								DRB1 allele ⁴
					30		32				36												47	*14:78
										35						42								*14:79
					30																			*14:80
					30																			*14:81
								33										43						*14:82
					30														44					*14:83
																								*14:84
							32																	*14:85
								33												45				*14:86
								33													46			*14:87
			28					33													46			*14:88
																					46			*14:89
								33								41								*14:90
																41								*14:91
								33													46			*14:92N
25									34								42							*14:93
																41								*14:94
					30									39				43						*14:95
																				45				*14:96
				29				33								41								*14:97
		28																					48	*14:98
								33																*14:99
													38											*14:100
																			43					*14:101
																								*14:102
																41								*14:103
				29				33																*14:104
								33																*14:105
					30										39									*14:106
										35					40									*14:107
					30										39									*14:108
							32								39									*14:109
																								*01:02:01-01:02:05, 01:23, 01:26, 01:34
25																								*01:14
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
120	250	220	160	125	190	80	100	175	205	250	130	175	215	120	215	115	225	95	215	125	105	60	65	Length of spec.
210			230	175			145							155	250	165		200			165		190	PCR product(s)
250				220												245					220		225	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
																								DRB1 allele ⁴

