

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
<b>DRB1 allele</b> <sup>1,2,3</sup>																																
*15:01:01:01-15:01:24, 15:51-15:53, 15:62, 15:71-15:72, 15:76, 15:79, 15:81-15:83, 15:86-15:87, 15:90, 15:92-15:93, 15:95, 15:97-15:98, 15:102, 15:106-15:107	1	2	3	4																												
*15:02:01-15:02:06, 15:02:08-15:02:09, 15:60, 15:99, 15:101, 15:105	1	2	3		5																											
*15:02:07	1		3		5																											
*15:02:10		2	3		5																											
*15:03:01:01-15:03:02	1	2		4		6															21											
*15:04	1			4						10																						
*15:05, 15:73	1			4				8																								
*15:06:01-15:06:02, 15:77	1	2	3	4						9																						
*15:07:01-15:07:02		2	3	4			7																									
*15:08	1	2	3		5											16																
*15:09, 15:48		2	3	4								12																				
*15:10	1			4									13		15																	
*15:11		2	3		5		7																									
*15:12	1	2	3	4						10																						
*15:13	1	2	3	4										14																		
*15:14	1	2	3		5	6																						27				
*15:15	1				5					10																						
*15:16, 15:35 <sup>4</sup>	1	2	3	4																									28			
*15:17N	1			4										14																		
*15:18	1	w	w	4													17															
*15:19	1	2	3		5				9																							
*15:20	1	2	3	4															18													
*15:21	1			4										15						19												
*15:22	1	2	3	4																	20											
*15:23	1			4				8														21										
*15:24, 15:28 <sup>5</sup>	1	2	3	4																		22										
*15:25	1			4				8												19			23									
*15:26	1	2	3		5																			24								
*15:27	1				5			8																								
*15:29	1	2	3		5																											
*15:30, 15:104	1	2	3		5															19												
*15:31	1				5			8																								
*15:32	1	2	3	4									13																			
*15:33	1	2	3	4																												
*15:34					5		7	8																								
*15:36	1	2	3	4				8																								
*15:37:01-15:37:02	1	2	3	4																	19											
*15:38	1	2	3		5																											
*15:39	1	2	3		5																											
*15:40	1	2	3	4																			22	24								
*15:41	1	2	3	4																												
*15:42, 15:64 <sup>4</sup>	1	2	3	4																												
*15:43	1	2	3	4																				24								
*15:44	1	2	3		5																											
*15:45	1	2	3	4																												
*15:46	1	2	3	4																												
*15:47	1	2	3		5						11																					
*15:49	1			4							11																					
*15:50N	1			4																												
*15:54, 15:66:01- 15:66:02	1			4				8													20											
*15:55	1	2	3	4																												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control

