

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
DRB1 allele^{1,2,3}																																		
*15:01:01:01-15:01:51, 15:51-15:53, 15:62, 15:71-15:72, 15:76, 15:79, 15:81-15:83, 15:86-15:87, 15:90, 15:92, 15:95, 15:97-15:98, 15:106-15:109, 15:111, 15:114, 15:116, 15:121, 15:124, 15:127-15:128, 15:132-15:133, 15:135, 15:139, 15:141, 15:144-15:146, 15:148N, 15:150-15:151, 15:153, 15:160, 15:162, 15:164Q, 15:166, 15:168-15:177, 15:180N, 15:182-15:184, 15:186-15:187, 15:190-15:193, 15:195-15:197, 15:199-15:205, 15:209N, 15:211, 15:215-15:216	1	2	3	4																														
*15:02:01:02-15:02:09, 15:02:11-15:02:20, 15:38:02, 15:60, 15:99, 15:101, 15:105:01-15:105:02, 15:119, 15:126, 15:131, 15:140, 15:149, 15:152, 15:154N, 15:159N, 15:161, 15:167, 15:179, 15:188:01:01-15:189, 15:194, 15:206, 15:210, 15:212-15:213	1	2	3		5																													
*15:02:10, 15:118:02		2	3		5																													
*15:03:01:01-15:03:05, 15:163N, 15:165, 15:178, 15:185	1	2		4		6														21														
*15:04, 15:147, 15:208	1			4					10																									
*15:05, 15:73	1			4				8																										
*15:06:01-15:06:04, 15:77, 15:110, 15:125, 15:214 ⁴	1	2	3	4					9																									
*15:07:01-15:07:03, 15:198		2	3	4			7																											
*15:08	1	2	3		5										16																			
*15:09	1	2	3	4						12																								
*15:10	1			4									13	15																				
*15:11:01-15:11:02		2	3		5		7																											
*15:12	1	2	3	4					10																									
*15:13	1	2	3	4									14																					
*15:14	1	2	3		5	6																					27							
*15:15:01-15:15:03	1				5				10																									
*15:16, 15:35 ⁴	1	2	3	4																								28						
*15:17N	1			4									14																					
*15:18	1	w	w	4												17																		
*15:19, 15:155 ⁴	1	2	3		5				9																									
*15:20	1	2	3	4													18																	
*15:21	1			4											15				19															
*15:22	1	2	3	4																20														
*15:23	1			4				8													21													
*15:24	1	2	3	4																		22												
*15:25	1			4				8											19	20			23											
*15:26	1	2	3		5																			24										
*15:27	1				5		8								15					20														
*15:28			3																			22												
*15:29	1	2	3		5																		23											
*15:30, 15:104:01-15:104:03 ⁴	1	2	3		5														19															
*15:31:01-15:31:02	1				5		8																											
*15:32	1	2	3	4									13																					
*15:33	1	2	3	4																					25									
*15:34					5		7	8							15					20														
*15:36	1	2	3	4				8																										
*15:37:01-15:37:02	1	2	3	4															19															
*15:38:01	1	2	3		5																						26							
*15:39	1	2	3		5																					25								
*15:40	1	2	3	4																		22		24										
*15:41	1	2	3	4																									29					
*15:42, 15:64 ⁴	1	2	3	4																								27						
*15:43	1	2	3	4																				24										
*15:44	1	2	3		5																											30		
*15:45	1	2	3	4																											30			
*15:46	1	2	3	4																												31		
*15:47	1	2	3		5					11																								
*15:48		2	3	4							12																							
*15:49	1			4							11																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*15:50N	1			4																														
*15:54, 15:66:01-15:66:02	1			4				8							15					20														
*15:55	1	2	3	4																			23											
*15:56	1	2	3	4				7																										
*15:57:01	1	2		4		6														19														

