

*03:76	1		4	6			9		12																	30						
*03:77, 03:84, 03:89, 13:144, 14:95 ⁴	1		4	6			9																				30					
*03:79	1		4	6				11					18														30					
*03:80	1		4				9	10				15															30					
*03:81	1	3	4	6			9			13										27							30					
*03:82	1		4				9						18														30					
*03:85	1		4					10																			30					
*03:88		2	3				8	10	11											24	25						30					
*03:90		2	3					10					16							24							30					
*03:92	1		4	6			9						18													29	31					
*01:16, 01:55, 04:22, DRB3*01:09, DRB3*02:04, DRB3*02:22, DRB3*03:03							8																									
*04:98:01-04:98:02													16																			
*08:20, 13:03:01-13:03:06, 13:07:01-13:08, 13:12:01-13:13, 13:33:01-13:33:03, 13:37-13:38, 13:47-13:49, 13:55, 13:60, 13:70, 13:72, 13:76, 13:84, 13:88-13:90, 13:94:01-13:95, 13:101, 13:108, 13:115, 13:133-13:134, 13:149, 13:151-13:152, 13:164, 13:167, 14:05:01-14:05:04, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36-14:37, 14:43-14:45, 14:56, 14:84, 14:91, 14:100, 14:103, 14:116, 14:123, 14:136								10																			30					
*09:01:08, 09:02:01-09:02:02, DRB3*01:07, DRB3*01:15								9																								
*09:06	1																															
*09:08												14																				
*11:06:01-11:06:03, 11:21, 11:47, 11:57, 11:129														17																		
*11:07, 11:103, 11:125							8							18													29					
*11:09, 11:20, 11:28:02, 11:122, 13:103, 13:165, 16:08, DRB3*01:08, DRB3*02:06			3																													
*11:10:01-11:10:02, 11:12:01-11:13:02, 11:30, 12:05, 12:14-12:15, 14:35, 14:72, 14:138																			21													
*11:16, 11:40, 11:58:01, 11:113, 13:43, 13:160, 15:32			4																													
*11:17, 11:37:01-11:37:02, 11:41, 11:48, 11:52, 11:56, 11:149, 13:122, 14:01:01-14:01:04, 14:07:01-14:08, 14:16, 14:22, 14:23:02, 14:25-14:26, 14:32:01-14:32:03, 14:34, 14:39, 14:54:01-14:55, 14:58, 14:60, 14:62, 14:69-14:70, 14:74-14:75, 14:87-14:88, 14:90, 14:92N, 14:101, 14:104-14:105, 14:111-14:114, 14:117-14:119, 14:122, 14:124-14:125, 14:129-14:131, 14:137N, 14:139-14:140, 14:142, 14:146								10																								
*11:39, 11:43, 11:151													15																			
*11:44, 11:49:01-11:49:02, 11:85, 11:127																											26					
*11:50								10					15														26					
*11:53																											27					
*11:59	1		4	6	7				w																							
*11:76																											26	27				
*11:80, 11:142					6	7			w																		20					
*11:83	1		4		7			10	w																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*11:87			3		6	7				w																						

Negative Control

*11:105									8		10				16		18												29			
*11:107									8		10																		29			
*11:114																																
*11:115				4											15																	
*11:116, 11:120												11																				
*11:135	1							6	7																							
*11:136															13																	
*11:145, 14:86												10				15																
*13:01:01-13:01:14, 13:06, 13:09-13:10, 13:18, 13:20, 13:28, 13:35, 13:42, 13:51, 13:59, 13:61:01-13:61:02, 13:69, 13:80, 13:87, 13:91-13:93, 13:98, 13:102, 13:105, 13:110-13:114, 13:117, 13:121, 13:127, 13:130, 13:137N-13:138, 13:141, 13:146, 13:148, 13:153, 13:156, 13:166, 14:17, 14:21, 14:33				4																										30		
*13:02:01-13:02:03, 13:02:05-13:02:09, 13:05:01-13:05:02, 13:29, 13:31, 13:50:02, 13:56, 13:63, 13:73-13:74, 13:96:01-13:96:02, 13:99, 13:119, 13:123-13:124, 13:126, 13:128, 13:139-13:140, 13:142N-13:143, 13:145, 13:147, 13:155, 13:157-13:158, 13:168, 14:30, 14:134				3																										30		
*13:02:04, 13:04, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:16, 13:21:01-13:25, 13:30, 13:44, 13:46, 13:50:01, 13:52, 13:62, 13:66:01-13:66:02, 13:68, 13:75, 13:82, 13:97:01-13:97:02, 13:109, 13:132, 13:154, 13:161, 13:169, 14:42																														30		
*13:15		2	4																											30		
*13:19, 14:20, 14:40, 14:94, 14:102		2									10																			30		
*13:26:01-13:26:02, 13:85, 14:02-14:03:01, 14:13, 14:19, 14:24, 14:27, 14:47, 14:51, 14:63, 14:67, 14:85, 14:89, 14:98, 14:135		2	3								10																24			30		
*13:27, 13:71, 13:129	1			4				6			9																			30		
*13:32, 13:40, 14:80				4							10																			30		
*13:34, 13:136				3																										30		
*13:36, 13:65, 13:120, 14:09, 14:144				3							10																			30		
*13:39				3								11																		30		
*13:41				3				6			9																			30		
*13:53, 14:06:01-14:06:03, 14:12:01-14:12:02, 14:18, 14:78, 14:81, 14:106, 14:108		2		4							10																			30		
*13:54, 13:100, 13:162												11																		30		
*13:57		2																												30		
*13:58, 13:81											10						17													30		
*13:64																														30		
*13:67				3																										30		
*13:77-13:79, 14:64				4								11																		30		
*13:83, 13:107, 14:65																														30		
*13:86		2																												30		
*13:104, 14:03:02		2	3																											30		
*13:106																														30		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*13:118, 13:135, 14:96											10					15																30
*13:125				4														17													30	
*13:131				4												15																30
*13:150																15																30
*13:159				3							10																					30

Negative Control

