

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
DRB1 allele^{1,2,3}																																
*03:01:01-03:01:41, 03:52:02, 03:56, 03:59, 03:61, 03:66, 03:70, 03:78, 03:83, 03:86, 03:91, 03:94, 03:98-03:99, 03:104, 03:106-03:109, 03:114, 03:116, 03:121, 03:127-03:129, 03:133-03:137, 03:139, 03:141-03:144, 03:146-03:153, 03:159-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180, 03:182-03:184, 03:186-03:187, 03:192-03:203	1			4		6			9									18												30		
*03:02:01-03:02:06, 03:74, 03:102-03:103, 03:131, 03:154, 03:176, 03:179		2	3					8		10														24						30		
*03:03		2	4					8		10														24						30		
*03:04:01-03:04:02	1				5	6			9									18												30		
*03:05:01-03:05:03			3			6			9									18												30		
*03:06, 03:60	1			4					9	10																				30		
*03:07:01-03:07:02, 03:171				4					8									18												30		
*03:08	1			4		6	7											18												30		
*03:09			3			6			9	11								18						23						30		
*03:10	1			4		6						12						18												30		
*03:11:01, 03:11:03, 03:177	1			4		6			9				13					18												30		
*03:12	1			4		6								14				18												30		
*03:13:01-03:13:02	1			4		6		w		11								18						25						30		
*03:14			3			6			9							16		18												30		
*03:15:01-03:15:01:02	1			4		6			9							16		18												30		
*03:15:02	1			4		6			9							16		18												30		
*03:16	1			4		6			9					14				18												30		
*03:17									10				13		15			18												30		
*03:18, 03:48 ⁵	1			4		6			9									18				22								30		
*03:19	1			4		6			9									18					23							30		
*03:20, 03:93 ⁵	1			4		6			9								17	18												30		
*03:21								8										18			21									30		
*03:22	1			4		6			9									18							25				29	30		
*03:23	1			4		6			9	11								18					23							30		
*03:24				4									13					18												30		
*03:25:01	1					6			9									18		20										30		
*03:25:02, 03:118	1					6			9									18		20								29	30			
*03:26	1			4					9	10															26					30		
*03:27		2	3						10				13					18						24						30		
*03:28	1			4		6			9									18		20										30		
*03:29		2	3					8										18					24							30		
*03:30	1					6			9									18			21									30		
*03:31	1								9	10										20										30		
*03:32	1			4		6			9									18						25						30		
*03:33	1			4		6			9									18							26					30		
*03:34	1			4		6			9								17	18										27		30		
*03:35, 14:127:01-14:127:02 ⁴									10				13					18												30		
*03:36	1			4		6			9									18										28		30		
*03:37	1			4		6			9					15				18												30		
*03:38		2	3					8		10	11												24							30		
*03:39	1			4		6			9									18											29	30		
*03:40, 03:105								8										18											29	30		
*03:41:01-03:41:02, 03:155								8		10								18											29	30		
*03:42, 03:87	1			4		6			9									18			21									30		
*03:43	1					6			9									18			21				26	27				30		
*03:44	1					6			9									18							26					30		
*03:45	1			4		6			9									18										28		30	31	
*03:46	1			4		6			9									18										27		30		
*03:47	1			4		6			9				13					18												30		
*03:49								8										18												30		
*03:50	1			4		6			9									18												30	31	
*03:51	1			4		6			9			12						18												30		
*03:52:01	1					6			9									18											29	30		
*03:53		2	3					8																24			27			30		
*03:54, 03:113				4														18												30		
*03:55	1			4		6			9									18										27		30		
*03:57, 03:63, 03:123						6			9									18												30		
*03:58	1			4		6			9									18		19										30		

