

INTERPRETATION TABLE								
DRB1*07 PCR-SSP subtyping								
Amplification patterns of the DRB1*0701 to 0712 alleles								
	Tube⁵							
	1	2	3	4	5	6	7	8
Length of spec.	230	185	105	90	190	170	165	105
PCR product			185	230	230			165
Length of int.	515	430	430	430	430	430	430	430
pos. control¹								
5'-primer(s)²	14	29	29	14	14	14	14	37
	5'-AT A ^{3'}	5'-AgA ^{3'}	5'-AgT ^{3'}	5'-AT A ^{3'}	5'-AT A ^{3'}	5'-AT A ^{3'}	5'-AT A ^{3'}	5'-Ag g ^{3'}
	14		30					57
	5'-AT A ^{3'}		5'-gA T ^{3'}					5'-CT A ^{3'}
			57					
			5'-CT A ^{3'}					
3'-primer(s)³	78	78	78	29	64	57	55	78
	5'-CAC ^{3'}	5'-CAC ^{3'}	5'-CAC ^{3'}	5'..T ^{3'}	5'-T CC ^{3'}	5'-C Ag ^{3'}	5'-CCA ^{3'}	5'-CAC ^{3'}
	78			77	77			
	5'-gTA ^{3'}			5'-A AT ^{3'}	5'-A gg ^{3'}			
Tube No.	1	2	3	4	5	6	7	8
DRB1 allele⁴								
*070101-070102	+	+						
*0703	+		+					
*0704	+			+				
*0705	+	+			+			
*0706	+	+				+		
*0707	+	+					+	
*0708	+	+						+
*0709	+	+	+					
*0710N	+	+		+				
*0711	+				+			
*0712	+	+	+					+
DRB1 allele⁴								
Tube No.	1	2	3	4	5	6	7	8