

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
DRB4 allele ^{1,2,3}																																		
*01:01:01, 01:01:01:03-01:01:01:04, 01:01:03-01:01:04, 01:01:06-01:01:13, 01:75, 01:78-01:79, 01:95, 01:102, 01:125, 01:127, 01:136, 01:141, 01:144, 01:151, 01:156, 01:164, 01:168-01:169, 01:177, 01:183	1		3	4																														
*01:01:02	1		?	4																														
*01:01:05, 01:70			3	4																														
*01:02		2		4			7								?		?																	
*01:03:01:01, 01:03:01:03-01:03:01:12, 01:03:01:14-01:03:01:24, 01:03:01:26-01:03:14, 01:03:16-01:03:18, 01:03:20-01:03:36, 01:61:02N, 01:63, 01:72-01:01:72:02, 01:74, 01:76-01:01:77, 01:80N-01:81, 01:84N-01:89, 01:94, 01:96-01:97, 01:103-01:105, 01:107, 01:112, 01:117-01:124, 01:126, 01:128N-01:132, 01:134-01:135, 01:137-01:138, 01:142-01:143, 01:145N-01:150, 01:152-01:155, 01:157-01:162N, 01:166-01:167, 01:171, 01:173-01:176, 01:178-01:182, 01:184, 01:186-01:190	1			4			7																											
*01:03:01:02N, 01:03:01:13N	1				5		7																											
*01:03:15, 01:90, 01:165N	1			4																														
*01:03:19, 01:163, 01:172				4			7																											
*01:04			?	4		6	?																											
*01:05	1	?	?	?			?		9													?												
*01:06	1		3	4							11												?											
*01:07:01	1	?	?	4			?					12											?											
*01:07:02	1	?	?	4			?					12																						
*01:08, 01:39 ¹	1	?	?	4			?						13																					
*01:09	1			4			7							14																				
*01:10	1			4			7								15																			
*01:11	1	?	?	4			?																								29			
*01:12	1	?	?	4			6	?					13																					
*01:13	1	?	?	4			?									16																		
*01:14N	1				5	6	7					12																						
*01:15	1	?	?	4			?																			25								
*01:16N	1	?	?	4			?																							28				
*01:17	1	?	?	4			?									16	17																	
*01:18	1	?	?	4			?																					26						
*01:19	1	?	?	4			?																						27					
*01:20, 01:36 ¹	1	?	?	4			?														21													
*01:21	1	?	?	4			?								15		17																	
*01:22	1	?	?	4			?										17				20													
*01:23	1	?	?	4			?										17				20													
*01:24, 01:37 ¹	1	?	?	4			?																23											
*01:25	1	?	?	4			?																											
*01:26	1	?	?	4			?											17											26		28			
*01:27	1	?	?	4			?												19	20														
*01:28	1	?	?	4			?												19															
*01:29	1	?	?	4			?												18	19														
*01:30	1	?	?	4			?								15				18	19														
*01:31	1	?	?	4			?												18															
*01:32, 01:43, 01:54-01N-01:54:02N ¹	1	?	?	4			?																		24									
*01:33, 01:38N, 01:40 ¹	1	?	?	4			?															22												
*01:34	1	?	?	4			?																						26	27				
*01:35	1	?	?	4			?																		24	25								
*01:41	1	?	?	4			?																									30		
*01:42	1	?	?	4			?															21												
*01:44	1			4			7																											
*01:45	1	?	?	4			?						13																					
*01:46-01:52, 01:55, 01:58-01:60, 01:62, 01:64, 01:66, 01:68-01:69, 01:71N, 01:73, 01:82-01:83, 01:91, 01:93, 01:98-01:101, 01:106, 01:108N-01:111, 01:113N-01:116, 01:133, 01:140, 01:170	1	?	?	4			?																											
*01:53	1	?	?	4			?																										31	
*01:56N	1	?	?	4			?																											
*01:57N	1	?	?	4			?																											
*01:61:01N	1	?	?	4			?																											
*01:65N	1			4			7																											
*01:67, 01:92, 01:139				?	4		?																											
*01:185	1			4			7																											
*02:01N				3	4						10																							
*03:01N				3				8																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
DRB1*01:139, DRB1*03:211, DRB1*10:19, DRB1*14:217, DRB1*15:141			3																															
DRB1*03:109, DRB1*12:64, DRB1*13:187																					19	20												
DRB1*04:109, DRB1*12:65, DRB1*13:258																						20												
DRB1*15:01:01:01-15:01:30, DRB1*15:01:32-15:02:17, DRB1*15:02:19-15:41, DRB1*15:43-15:92, DRB1*15:94-15:140, DRB1*15:142-15:179, DRB1*15:181-15:200:01:02N, DRB1*15:202-15:235, DRB1*16:01:01:01-16:05:02, DRB1*16:07-16:79							w																											
DRB1*15:42							w														20													
DRB1*15:93																																		

Negative Control

Negative Control