

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
DRB5 allele^{1,2,3}																						
*01:01:01:01:01:01:07, 01:01:04-01:01:07, 01:01:09, 01:01:11-01:01:13, 01:19, 01:22:01-01:24, 01:26, 01:29-01:30, 01:33-01:34, 01:36-01:38, 01:40, 01:42, 01:45, 01:50, 01:53N-01:54, 01:57-01:58N, 01:62-01:63:01, 01:65, 01:80-01:81N, 01:83N-01:85, 01:88-01:89, 01:91-01:96, 01:99, 01:101N, 01:104-01:109, 01:111-01:112, 01:114-01:119, 01:121N-01:122, 01:124, 01:126, 01:128, 01:132-01:135, 01:137-01:142	1	2	3	4	5																	21
*01:01:02	1	2	3	4									?									
*01:01:03, 01:66, 01:75, 01:86	1	2	3	4																		21
*01:01:08	1	2	3	4																		
*01:01:10, 01:51, 01:63:02, 01:73, 01:78, 01:87, 01:123	1	2	3		5																	21
*01:02:01-01:02:04, 01:25, 01:28, 01:32, 01:35, 01:59-01:60, 01:64, 01:67N-01:68N, 01:70, 01:72, 01:76, 01:90, 01:102-01:103, 01:110, 01:113, 01:120N, 01:125N, 01:130	1	2	3			6	7															21
*01:03	1	2				6	7	8														21
*01:04	1	2	3	4					9													
*01:05	1	2	3		5	6																21
*01:06, 01:100			4					8			11											
*01:07	1	2	3	4	5					10			?									21
*01:08:01N-01:08:02N, 01:56, 01:127N	1	2	3			6	7						13									21
*01:09	1	2	3	4	5			8					?									21
*01:10N	1	2	3			6	7								15							21
*01:11	1			4				8			11											
*01:12	1	2	3							10					15							
*01:13, 01:41	1	2	3		5				9													21
*01:14	1	2	3								11											21
*01:15	1	2	3	4						10				15								
*01:16	1	2	3	4	5				9													21
*01:17	1	2	3	4	5		7															21
*01:18	1	2	3	4	5								13									21
*01:20	1	2	3			6	7									16						21
*01:21, 01:136				4				8			11					16						
*01:27N, 01:46	1	2	3			6	7										17					21
*01:31, 01:44			2	3	4	5																21
*01:39						6	7															21
*01:43			2	4				8								16						
*01:47	1	2	3	4	5											16						21
*01:48N	1	2	3	4	5					10											20	21
*01:49N	1	2	3	4	5																19	21
*01:52N, 01:77	1	2	3																			21
*01:55			2	4				8														
*01:61	1	2	3				7															21
*01:69	1	2				6	7	8									17					21
*01:71N	1			4																		
*01:74	1	2	3			6	7															
*01:79Q, 01:97-01:98				4	5																	21
*01:82, 01:129	1	2		4	5																	21
*02:02:01-02:02:05, 02:09, 02:11, 02:14-02:16, 02:18, 02:20-02:23, 02:27-02:30, 02:33, 02:35-02:39								8			11	12										
*02:03	1							8			11											
*02:04		2						8				12	?									
*02:05			3				7					12										
*02:06				4				8			11	12										
*02:07								8			11	12		14								
*02:08		2	3			6	7					12				16						21
*02:10								8			11	12				16						
*02:12		2	3			6	7					12						17				21
*02:13								8			11											
*02:17, 02:32								8			11											
*02:19N								8			11	12						17	18			
*02:24								8				12										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*02:25N			2	3		6	7					12										21
*02:26N			2	3		6	7					12	13									21
*02:31	1			3				8			11				15							
*02:34				3				8			11	12			15							
<i>DRB1*03:112, DRB1*09:06, DRB4*01:117</i>																						
<i>DRB1*08:59, DRB1*11:210</i>																						
<i>DRB1*09:07</i>	1	2	3											15								
<i>DRB1*09:53</i>						6	7															
<i>DRB1*15:02:03, DRB1*16:01:02, DRB1*16:02:02, DRB4*01:05, DRB4*01:07:01</i>													?									
<i>DRB1*15:86</i>													13									
<i>DRB1*16:60</i>				4	5																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22

Negative Control

Negative Control