

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-A allele ^{1,2}	ser ³																																		
*01:01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:02, 01:04N, 01:07, 01:09-01:11N, 01:16N, 01:18N, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N, 01:54-01:62, 01:64, 01:67-01:01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146	A1, Null, -	1			4																						26								
*01:01:23, 01:01:48, 01:08, 01:30, 01:79, 01:99, 01:145	A1, -	1																									26								
*01:01:32, 01:69:01-01:69:02, 01:143	-				4																						26								
*01:01:56	-	1			4									w												26						30			
*01:03, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130	A1, Null, -	1			4																														
*01:12, 01:19	A1, -	1																																27	
*01:13	A1	1			4							11	13	15													26		28						
*01:14-01:15N	A1, Null	1																																	
*01:17	A1	1			4																						26		28						
*01:21, 01:126	A1, -	1			4																						26	27							
*01:28	-	1			4							11		15													26								
*01:43	-	1			4							10															26								
*01:51	-	1			4				8			10															26								
*01:95	-	1			4														17								26								
*01:121	-	1			4																19						26								
*01:134	-	1			4																						26			29					
*01:136	-	1			4							11																							
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24-01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79-01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:155, 02:157-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:478	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																																	
*02:17:01-02:17:03	A2, -	2							w																										
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2	2												13							20		22											28	
*02:35:02	A2													13							20		22											28	
*02:46, 02:70	A2	2																			20		22												
*02:48, 02:129	A2, -																				20		22												
*02:55	A2/A2	2							8													21		23											
*02:65, 02:407, 02:449	A2/A1	2																				19													
*02:78	-	w			4																20		22											28	
*02:135	-	2												13																				30	
*02:149	-	2																				21			24										
*02:156, 02:338	-	2																																27	
*02:169	-	2			4																														
*02:237	-	2													14	16																			
*02:243:01-02:243:02	-	2																			18		21												
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*02:309	-	2											12	14									21			24									
*02:454	-	2											12	14																					

Negative Control

*23:57	-				5															21															
*24:02:01:01-24:06, 24:08-24:11N, 24:13:01-24:13:02, 24:17, 24:20- 24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56, 24:58-24:63, 24:66- 24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:107, 24:109-24:111, 24:113, 24:115-24:130, 24:132N- 24:137, 24:139-24:187, 24:189- 24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:270	A24, Low A24, A2403 /A24, A9, A24/A 9, Null, -					5	7																												
*24:07, 24:108, 24:112, 24:131	A24, -					5	7																												28
*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, C*04:01:03 ^d	A24, Cw4, Null, -					5																													
*24:18, 24:204, 24:213	A24/A 3, -					5	7														23														
*24:19, 24:44	A9					5	7			11																									
*24:24	A9					5	6	7																										28	
*24:82	-					5	7														21														
*24:92	-					5																													27
*24:243	-					5	7									17																			
*25:01:01:25:04, 25:07-25:12N, 25:14- 25:16, 25:19-25:24	A25, Null, -								8	9			13																						30
*25:05	A25								8	9		12	13							20														30	
*25:06	A25								8	9		12																							
*25:13	-								8	9			13											24										30	
*25:17	-									9			13																					30	
*25:18	-								8	9			13																						
*26:01:01:26:01:20, 26:01:22- 26:01:31, 26:08, 26:10-26:15, 26:17- 26:18, 26:20, 26:23-26:28, 26:31- 26:33, 26:35-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:63, 26:66- 26:71N, 26:74-26:77, 26:79-26:88, 26:90, 26:93-26:98	A26, A10, Null, -								8	10			13																						30
*26:01:21, 26:04, 26:34, 26:73	A26, -								8	10																									
*26:02:01, 26:29, 26:49	A26, -								8	10		w																							30
*26:02:02	-								8	10																									30
*26:03:01-26:03:02, 26:06, 26:21	A26, -								8		11	13									23										29	30			
*26:05	A26								8			13									23													30	
*26:07:01-26:07:02, 26:64, 26:89	A26, -									10		13																							30
*26:09, 26:91	A26, -								8	10	12																								30
*26:16	A26									7		10	13																					30	
*26:19	-			4						10		13	14																					30	
*26:22	A26								8	10		13	14							21														30	
*26:30	A26								8			13									23	24								29	30				
*26:54	-								8	10	12	13								20														30	
*26:65	-								8	10		13												24										30	
*26:72	-			4						10		13																						30	
*26:78	-								8		11																							29	
*26:92	-										11	13																						29	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*29:01:01:29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40- 29:47, 29:50-29:58	A29, Null, -														14											25			28						
*29:07, 29:49	A29, -					5	6								14											25			28						
*29:13, 29:39	-														14			17							25			28							
*29:14	-														14		16								25			28							
*29:19	-														14										25			29							
*29:20, 29:34	-														14										25										
*29:28	-								8						14										25			28							
*29:30	-														14											25									
*29:48	-														14			18								25			29						
*30:01:01:30:01:07, 30:01:09, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48- 30:49, 30:52-30:54, 30:56, 30:58- 30:59N, 30:62-30:63, 30:65, 30:71- 30:75, 30:78N-30:79	A30, Null, -															15										25			28						

Negative Control

