

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
HLA-A allele ^{1,2}	ser ³																																			
*01:01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:02, 01:07, 01:09-01:11N, 01:16N, 01:18N, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175	A1, Null, -	1			4																						26									
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N-01:179N	A1, Null, -	1																									26									
*01:01:32, 01:01:60, 01:69:01-01:69:02, 01:143, 01:171	-				4																						26									
*01:01:56	-	1			4									w													26				30					
*01:03, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130	A1, Null, -	1			4																															
*01:04N	Null	1			4																						26						31			
*01:12, 01:19	A1, -	1																																		
*01:13	A1	1			4							11	13	15													26	28								
*01:14-01:15N	A1, Null	1																																		
*01:17	A1	1			4																						26	28								
*01:21, 01:126	A1, -	1			4																						26	27								
*01:28	-	1			4							11		15													26									
*01:43	-	1			4						10																26									
*01:51	-	1			4				8		10																26									
*01:95	-	1			4														17								26									
*01:121	-	1			4																19						26									
*01:134	-	1			4																						26			29						
*01:136	-	1			4								11																							
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:01, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:155, 02:157-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:570, 02:572-02:573	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																																		
*02:17:01-02:17:03	A2, -		2					w														21														
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2		2											13							20		22									28				
*02:35:02	A2													13							20		22								28					
*02:46, 02:70	A2		2																		20		22													
*02:48, 02:129	A2, -																				20		22													
*02:55	A2/A28		2					8														21		23												
*02:65, 02:407, 02:449	A2/A19, -		2																			19														
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*02:78	-		w		4																20		22													
*02:108, 02:110, 02:268, 02:300, 02:303	A2, -		2																			21														
*02:135	-		2											13												24								30		
*02:156, 02:338	-		2																																27	

Negative Control

*02:169	-	2	4																																						
*02:237	-	2											14	16																											
*02:243:01-02:243:02	-	2																18	21																						
*02:309, 02:454	-	2								12	14																														
*02:527	-	2					8																																		
*03:01:01:01-03:01:05, 03:01:07-03:01:13, 03:01:15-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:29, 03:01:31-03:01:34, 03:01:36-03:01:46, 03:01:51, 03:01:53-03:04:03, 03:06-03:07, 03:11N, 03:13-03:14, 03:16-03:17:02, 03:20, 03:22:01-03:23:02, 03:25-03:28, 03:31, 03:33, 03:35, 03:37-03:39, 03:44:01-03:49, 03:51-03:53, 03:55-03:56, 03:58, 03:60-03:61, 03:65, 03:67-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:97, 03:99-03:104, 03:106, 03:109-03:110, 03:112-03:121, 03:123:01-03:126, 03:128-03:134, 03:136-03:139, 03:141, 03:143-03:151, 03:154-03:164, 03:166, 03:168N-03:171, 03:174-03:175, 03:179-03:185, 03:188-03:197N, 03:201-03:202, 03:206-03:207, 03:209-03:210, 03:212-03:214, 03:216-03:218, 03:221	A3, A3/blank, Null, -			3									13															27	28												
*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:66, 03:127, 03:140, 03:153, 03:165, 03:200Q	A3, -			3									13																										28		
*03:01:19	-			3							12	13									24																		27	28	
*03:01:23, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211, 03:219	Null, -			3																																			27	28	
*03:01:30, 03:01:35, 03:01:48, 03:01:50, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208	A3/blank, -												13																										27	28	
*03:01:49, 03:204, 03:215	-												13																										28		
*03:08, 03:32, 03:36N, 03:89, 03:107, 03:176	A3, Null, -			3																																			27		
*03:09	A3			3									13																									27	28	29	
*03:12	A3			3	4																																		27	28	
*03:15, 03:19, 03:30	A3			3		5							13																										27	28	
*03:18, 03:135	-				4								13																										26	28	
*03:21N	Null			3									13																										27	28	31
*03:24	A3			3			8						13																										27	28	
*03:29, 03:34, 03:62	A3, -			3									13																										27		
*03:41	-			3																																			27		
*03:43, 03:82, 03:186	-			3									13	15	17																							27	28		
*03:50	-			3			8						13																										27	28	
*03:63	-			3							11	13																											27		
*03:75, 03:177	-												13																										23	28	
*03:88	-			3	4						11																												23	27	
*03:95	-												13	14	16																								23	28	
*03:108, 03:172, 03:198	-			3																																			27	29	
*03:152	-			3		5																																	27		
*03:187	-												13																										22	28	
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32							
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32							

Negative Control

