

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
HLA-A allele <sup>1,2</sup>	ser <sup>3</sup>																																			
*01:01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:02, 01:07, 01:09:01-01:11N, 01:16N, 01:18N, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:180-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207	A1, Null, -				4																						26									
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:30, 01:79, 01:99, 01:147Q, 01:149, 01:159, 01:167, 01:178N-01:179N, 01:205, 01:208Q	A1, Null, -	1																									26									
*01:01:32, 01:01:60, 01:69:01-01:69:02, 01:143, 01:171, 01:188	-				4																						26									
*01:01:56	-	1			4								w													26						30				
*01:03, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130	A1, Null, -	1			4																															
*01:04N	Null	1			4																						26					30	31			
*01:12, 01:19, 01:200	A1, -	1																																		
*01:13, 01:176	A1, -	1			4							11	13	15													26		28							
*01:14-01:15N	A1, Null	1																																		
*01:17	A1	1			4																						26		28							
*01:21, 01:126	A1, -	1			4																						26	27								
*01:28	-	1			4							11		15													26									
*01:43	-	1			4						10																26									
*01:51	-	1			4				8	10																	26									
*01:95	-	1			4														17								26									
*01:121	-	1			4																19						26									
*01:134	-	1			4																						26			29						
*01:136, 01:192	-	1			4							11															26									
*01:145	-	1												w													26									
*01:194	-	1			4									13	15												26		28							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24-01:02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:155, 02:157:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:570, 02:572-02:579, 02:581, 02:583-02:590, 02:592-02:616, 02:618Q-02:634	A2, Low A2, -, A203/ A2, A210/ A2, A2/blank, Null, -		2																																	
*02:17:01-02:17:03	A2, -		2					w														21														
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -		2											13								20	22						28							
*02:35:02	A2													13								20	22						28							
*02:46, 02:70	A2		2																			20	22													
Negative Control																																				

*02:48, 02:129, 02:571	A2, -																	20	22																																	
*02:55	A2/A2 8		2				8													21	23																															
*02:65, 02:407, 02:449	A2/A1 9, -		2															19																																		
*02:78	-	w		4															20	22																		28														
*02:108, 02:110, 02:268, 02:300, 02:303, 02:617	A2, -		2																	21																																
*02:135	-		2				8					13												24															30													
*02:156, 02:338	-		2																																					27												
*02:169	-		2	4																																																
*02:237	-		2										14	16																										23												
*02:243:01-02:243:02	-		2																	18																				21												
*02:309, 02:454	-		2									12	14																												24											
*02:527, 02:582	-		2				8																																													
*02:591	-		2																																						29											
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32																		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32																			
*03:01:01:01-03:01:05, 03:01:07- 03:01:13, 03:01:15-03:01:18, 03:01:20-03:01:22, 03:01:24- 03:01:29, 03:01:31-03:01:34, 03:01:36-03:01:37, 03:01:39- 03:01:46, 03:01:51, 03:01:53- 03:04:03, 03:06-03:07, 03:11N, 03:13- 03:14, 03:16-03:17:02, 03:20, 03:22:01-03:23:02, 03:25-03:29, 03:31, 03:33, 03:35, 03:37-03:39, 03:44:01-03:49, 03:51-03:53, 03:55- 03:56, 03:58, 03:60-03:61, 03:65, 03:67-03:71, 03:73-03:74, 03:76- 03:81, 03:83-03:87, 03:90-03:94, 03:96-03:97, 03:99-03:104, 03:106, 03:109-03:110, 03:112-03:121, 03:123:01, 03:124-03:126, 03:128- 03:134, 03:136-03:139, 03:141, 03:143-03:151, 03:154:01-03:164, 03:166, 03:168N-03:170, 03:174- 03:175, 03:179-03:185, 03:188- 03:193, 03:195-03:197N, 03:201- 03:202, 03:206-03:207, 03:209- 03:210, 03:212-03:214, 03:216- 03:218, 03:220-03:224, 03:226- 03:230, 03:232-03:233, 03:235- 03:245, 03:247	A3, A3/bla nk, Null, -																																																			
*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:66, 03:127, 03:140, 03:153, 03:165, 03:200Q, 03:234Q, 03:249	A3, -			3										13																												28										
*03:01:19	-			3									12	13										24																												
*03:01:23, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211	Null, -			3																																																
*03:01:30, 03:01:35, 03:01:48, 03:01:50, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208	A3/bla nk, -													13																														27	28							
*03:01:38, 03:123:02, 03:171	-			3										13	14																												27	28								
*03:01:49, 03:204, 03:215, 03:225	-													13																														28								
*03:08, 03:36N, 03:89:01, 03:107, 03:176	A3, Null, -			3																																									27							
*03:09	A3			3											13																														27	28	29					
*03:12	A3			3	4																																								27	28						
*03:15, 03:19, 03:30	A3			3		5									13																															27	28					
*03:18, 03:135	-				4										13																															27	28					
*03:21N	Null			3											13																															27	28	30	31			
*03:24	A3			3											8																																27	28				
*03:32	-			3																																												27				
*03:34, 03:62	A3, -			3												13																																27				
*03:41	-			3																																													27			
*03:43, 03:82, 03:186	-			3											13	15																																27	28			
*03:50	-			3											8																																		28			
*03:63	-			3										11		13																																	27			
*03:75, 03:177	-															13																																	28			
*03:88	-			3	4																																													27		
*03:89:02, 03:108, 03:172, 03:198	-			3																																															27	29

Negative Control



Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
*24:06, 24:13:01, 24:22, 24:82, 24:167, 24:207	A24, -					5		7														21															
*24:07:01-24:07:02, 24:108, 24:112, 24:131, 24:288, 24:290, 24:294Q, 24:347	A24, -					5		7																				28									
*24:09N	Null					5		7																					29								
*24:11N	Null					5		7																						30	31						
*24:14:01:01-24:15, 24:51-24:53, 24:57, 24:64, 24:114, 24:222N, 24:291, 24:296, 24:304, 24:316, 24:324, C*04:01:03 <sup>5</sup>	A24, Cw4, Null, -					5																															
*24:18, 24:204, 24:213	A24/A3, -					5		7															23														
*24:19, 24:44	A9					5		7				11																									
*24:24	A9					5	6	7																					28								
*24:92	-					5																						27									
*24:104, 24:243	-					5		7										17																			
*24:339	-					5		7														21							28								
*24:340	-					5		7													20	22															
*25:01:01-25:01:08, 25:02, 25:04, 25:07-25:12N, 25:14-25:17, 25:19:01-25:29, 25:31-25:33	A25, Null, -								8	9				13																	30						
*25:01:09, 25:18	-								8	9				13																							
*25:03	A25								8	9				13				17														30					
*25:05	A25								8	9			12	13							20												30				
*25:06	A25								8	9			12																								
*25:13	-								8	9				13												24						30					
*25:30	-								8	9				13				17												21			30				
*25:34-25:35	-								8	9				13																21			30				
*26:01:01:01-26:01:20, 26:01:22-26:02:02, 26:08, 26:10-26:15, 26:17-26:18, 26:20, 26:23-26:28, 26:31-26:33, 26:35-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:64, 26:66-26:71N, 26:74-26:77, 26:79-26:90, 26:93-26:102, 26:104-26:110, 26:112-26:119, 26:124	A26, A10, Null, -								8		10			13																		30					
*26:01:21, 26:04, 26:34, 26:73	A26, -								8		10																										
*26:03:01, 26:06, 26:21, 26:111	A26, -								8			11		13											23						29	30					
*26:05	A26								8					13											23								30				
*26:07:01-26:07:02, 26:65	A26, -								8		10			13													24						30				
*26:09, 26:91	A26, -								8		10	12																									
*26:16	A26								7	8				13																				30			
*26:19	-				4				8		10			13	14																				30		
*26:22	A26								8		10			13	14																				30		
*26:29, 26:49	A26, -								8		10			w									21												30		
*26:30	A26								8					13											23	24							29	30			
*26:54	-								8		10		12	13								20													30		
*26:72	-				4				8		10			13																					30		
*26:78	-								8				11												23									29			
*26:92	-								8				11		13											24								29	30		
*26:103	-								8			10		13																							
*26:120	-	1							8			10		13																					30		
*26:121, 26:123	-								8			10		13																					30		
*26:122	-								8		10			13								20	21												30		
*29:01:01:01-29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40-29:47, 29:50-29:65, 29:68-29:73, 29:75-29:90	A29, Null, -														14												25			28							
*29:07, 29:49	A29, -					5	6								14												25			28							
*29:13, 29:39	-													14					17								25			28							
*29:14	-													14					16								25			28							
*29:19	-													14													25							29			
*29:20, 29:34, 29:66, 29:74	-													14													25										
*29:28	-								8					14													25			28							
*29:30	-													14																							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32				
*29:48	-														14					18							25								29		
*29:67	-				4										14																					28	

Negative Control

*30:01:01-30:01:07, 30:01:09-30:01:11, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48-30:49, 30:52-30:54, 30:56, 30:58-30:59N, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:88, 30:91-30:98, 30:102	A30, - Null, -														15						25					28										
*30:01:08, 30:18, 30:41, 30:60	A30, -														15											28										
*30:02:01:01-30:03, 30:07, 30:10, 30:22, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85	A30, - Null, -														15						25															
*30:04:01-30:04:02, 30:06, 30:09, 30:29, 30:77, 30:90, 30:99	A30, -														15						25				27											
*30:08	A30			4											15						25				28											
*30:12, 30:31, B*07:260	A30, -														15																					
*30:13, 30:44	A30, -														15						25				29											
*30:16	A30														15						25			28	29											
*30:17	A30														15						25			27	28											
*30:46	-														15						25			27	29											
*30:55	-									13					15						25				28											
*30:89	-			3											15	17					25			27	28											
*31:01:02:01-31:02, 31:05-31:07, 31:10-31:20, 31:22-31:23, 31:25-31:27, 31:30-31:34, 31:36-31:61, 31:63-31:84, 31:86-31:88, 31:90-31:108, 31:110-31:111	A31, - Null, -														16										25											
*31:03	A31									12		14			16									25		27										
*31:04	A31									12		14			16						24			25		27										
*31:08, 33:53 <sup>4</sup>	A31/A 24, -			5											16								25													
*31:09, 31:28	A31, -														16																					
*31:21	-														16	17							25													
*31:24	-										13				16								25													
*31:29	-				5	6									16								25													
*31:35	-														15	16	17						25													
*31:62	-														16						22		25													
*31:85	-				6										16								25													
*31:89	-														16								25			28										
*31:109	-														16		18						25													
*32:01:01-32:02, 32:06-32:14, 32:16, 32:18-32:23, 32:25-32:37, 32:39-32:41, 32:43-32:48N, 32:50-32:59, 32:61, 32:63, 32:65-32:70, 32:73-32:77, 32:81-32:86	A32, - Null, -								9														25													
*32:03, 32:38, 32:60, 32:71, 32:78, 32:80	A32, -																						25													
*32:04	A32/A 3			3					9																	27										
*32:05, 32:79	A32, -				5										16	17							25													
*32:15	A32						8	9								17	18						25													
*32:17	-							9								17							25			28										
*32:24, 32:49	-							9								17																				
*32:42	-							9					14			17							25													
*32:62	-							9								17		19					25													
*32:64	-			4				9								17							25													
*32:72	-					6		9								17							25													
*33:01:01-33:01:08, 33:03:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:114	A33, - Null, -																		18				25													
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*33:13, 33:48	-					5		7				10							18					25												
*33:19	-					5		7											18				25													
*33:22	-																		18			21			25											
*33:24	-																		18					23	25						29					
*33:49	-																		18					25				27								
*33:51	-							8											18					25												
*33:61	-												13						18					25												
*34:01:01-34:01:02, 34:05, 34:11-34:12, 34:14	A34, -							8			11	12																								31

Negative Control

*34:02:01-34:02:03, 34:07, 34:10N, 34:13, 34:15	A34, Null, -						8		11	12												27		31											
*34:02:04	-						8		11	12					20	22						27		31											
*34:03	A34						8		11	12												27		31											
*34:04	A34						8		11	12		14										27		31											
*34:06	A10						8		11	12																									
*34:08	A34						8		11	12	13											27		31											
*34:09	-						8			12									24			27	28	31											
*36:01, 36:03, 36:05	A36, -	1														22																			
*36:02	A36	1		3								14				22	23																		
*36:04	A36	1			4											22						26													
*43:01	A43						8		10	12	13											26		30											
*66:01:01, 66:01:03, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:20, 66:22, 66:24	A66, A26, -						8			11	13													30											
*66:01:02	-						8			11																									
*66:02, 66:16, 66:21	A66, -						8			12							24					28													
*66:03	A10						8			12							24																		
*66:06	A66						8		11	13					21								30												
*66:09	-						8		11	13	14				21								30												
*66:10	-						8		11	w													30												
*66:12	-						8			13						24						28	30												
*66:15	-						8			13													30												
*66:23	-			4			8		11	13													30												
*66:25	-						8			12					21	24						28													
*66:26Q	-						8									24						28													
*68:01:01:01-68:01:17, 68:01:19-68:01:31, 68:02:01:01-68:02:12, 68:06-68:08:01, 68:09-68:10, 68:12, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30, 68:32-68:44, 68:46-68:65, 68:67-68:70, 68:73-68:83, 68:85-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:148Q	A68, A28, Null, -						8								20							28													
*68:01:18, 68:03:01-68:04:01, 68:31, 68:72, 68:90, 68:93, 68:97	A68, -						8								20																				
*68:01:32	-						8								20	22						28													
*68:04:02	-						8								20	21																			
*68:05, 68:15, 68:20	A68						8								20		23						29												
*68:08:02, 68:105	A68, -						8								20	21						28													
*68:11N	Null						8				13				20	22						28													
*68:13:01, 68:66, 68:134	A68, -			4			8								20							28													
*68:19	A68						8				14				20							28													
*68:25	A68						8								19	20						28													
*68:29	A68						8							18	20	21						28													
*68:45, 68:117	-					7	8								20							28													
*68:71	-						8				13				20							28													
*68:84	-						8		10						20							28													
*68:103:01-68:103:02	-						8								20							27	28												
*68:109	-						8								20		23																		
*68:130	-						8				12				20						26	28													
*68:136	-						8								20		23					28													
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*69:01, 69:03	A69, -						8															21													
*69:02	-						8					11										21													
*74:01:01-74:01:02, 74:01:04-74:03, 74:05, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:27	A74, Null, -																			19					25										
*74:01:03	-											12									19			24	25										
*74:04	A74																			18	19			25											
*74:06	A74																				19			25						29					
*74:07	A74																			17	19			25											
*74:13	-											13		16							19			25					28						
*74:19	-			4																	19			25											
*74:21	-																			18	19			25							29				
*74:23	-																				19			25				27							
*80:01:01:01-80:01:01:02, 80:03	A80, -						6																	23											
*80:02	-						6					11																							
B*07:81, B*08:52, B*18:67, B*38:41, B*51:185, B*53:05, B*53:16, B*53:33	B53, -								9																										

Negative Control

Negative Control

B*18:27	-					6																											
C*06:187	-																					23											
C*07:404	-											11																					
C*07:481, C*07:504	-																					?											
C*12:131	-				4																												
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32