

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*01:01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:03:02, 01:09:01:01:01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67-01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303-01:306, 01:308N-01:313, 01:315-01:331N	A1, Null, -																																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314	A1, Null, -	1			4																							26						
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307	-				4																							26						
*01:01:56	-	1			4							12																26				30		
*01:04:01:01N-01:04:01:02N	Null	1			4																							26				30	31	
*01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1			4																													
*01:07, 01:95, 01:289	A1, -	1			4																17							26						
*01:12, 01:19, 01:200, 01:244	A1, -	1																																
*01:13, 01:176	A1, -	1			4							11		13	15													26			28			
*01:14	A1	1																																
*01:15N	Null	1			w																													
*01:17, 01:302	A1, -	1			4																							26				28		
*01:21, 01:126	A1, -	1			4																							26		27				
*01:26, 01:192	A1, -	1			4							11																26						
*01:28, 01:299	-	1			4							11			15													26						
*01:43	-	1			4							10																26						
*01:51	-	1			4						8		10															26						
*01:121	-	1			4																	19						26						
*01:134	-	1			4																							26				29		
*01:136	-	1			4																													
*01:194	-	1			4																							26				28		
*01:229	-	1			4							11			13	15												26				29		
*01:298	-	1			4									9								17						26						
*01:301Q	-	1			4	5		7																			w							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control

*74:37	-														13			19						25															
*80:01:01-80:01:02, 80:03, 80:05-80:06	A80, -				6																			23															
*80:02	-				6						11																												
*80:04	-				6												17							23															
<i>B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60</i>	B53, -										9																												
B*18:27	-				6																																		
B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569	-																							22															
C*03:365, C*12:131	-			4																																			
C*06:187	-																								23														
C*06:247	-																																					29	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32						