

		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32							
*02:35:02	A2													13							20	22											28							
*02:46, 02:70	A2		2																		20	22																		
*02:48, 02:129, 02:571	A2, -																				20	22																		
*02:55	A28, A2		2						8													21	23																	
*02:65, 02:407, 02:449, 02:932	A19, A2, -																			19																				
*02:78	-		w	4																	20	22												28						
*02:81, 02:124	A2		2						9																															
*02:135	-		2											13										24										30						
*02:149:01-02:149:02, 02:842, 02:947	-		2																			21																		
*02:156, 02:338, 02:952	-		2																																27					
*02:169	-		2	4																																				
*02:237, 02:809, 02:927	-		2											14	16								23																	
*02:243:01-02:243:03	-		2																		18	21																		
*02:309	-		2											12	14							21																		
*02:454	-		2											12	14								24																	
*02:527, 02:582	-		2						8																															
*02:576, 02:682	-		2																				22																	
*02:644	-								8	10												21																		
*02:651	-		2																			20																		
*02:671	-		2																																					
*02:741	-		2						8					13								20	21	22											31					
*02:804	-		2						7																															
*02:815	-		2						8														21																	
*02:829	-				4					9																														
*02:929	-		2																																					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32							
*03:01:01-03:01:05, 03:01:07-03:01:13, 03:01:15-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:29, 03:01:31-03:01:34, 03:01:36-03:01:37, 03:01:39-03:01:46, 03:01:48, 03:01:51, 03:01:53-03:01:82, 03:01:84-03:01:89, 03:01:91-03:01:93, 03:01:95-03:01:105, 03:01:107-03:04:03, 03:06-03:07:02, 03:11N, 03:13-03:14, 03:16-03:17:02, 03:20, 03:22:01-03:23:02, 03:25-03:29, 03:31, 03:33, 03:35, 03:37-03:39, 03:44:01-03:49, 03:51:01-03:53, 03:55-03:56, 03:58, 03:60-03:61, 03:65-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:97, 03:99-03:103:01, 03:104, 03:106, 03:109-03:110, 03:112-03:121, 03:123:01, 03:124-03:126, 03:128-03:134, 03:136-03:139, 03:141, 03:143-03:151, 03:154:01-03:164, 03:166, 03:168N-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197:02N, 03:201-03:202, 03:206-03:207, 03:209-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:232-03:233, 03:235-03:245, 03:247, 03:250-03:251, 03:253-03:259, 03:261-03:262N, 03:264-03:266N, 03:268-03:272, 03:274-03:281, 03:283N-03:287, 03:289:01-03:321, 03:323N-03:325, 03:327-03:329N, 03:331, 03:333-03:342N, 03:344-03:346, 03:348-03:376, 03:378-03:381N, 03:383-03:394, 03:396-03:397, 03:399-03:412, 03:415, 03:417-03:427, 03:430-03:433	A3, Null, -				3									13																						27	28			
*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:127, 03:140, 03:153, 03:165, 03:231:02, 03:234Q, 03:249, 03:288, 03:322, 03:416	A3, -				3									13																							28			
*03:01:19, 03:103:02	-				3								12	13										24												27	28			
*03:01:23, 03:01:83, 03:01:94, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211, 03:252, 03:267	Null, -				3																																27	28		
*03:01:30, 03:01:35, 03:01:50, 03:01:90, 03:01:106, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208, 03:225, 03:326, 03:332	A3, -													13																							27	28		
*03:01:38, 03:123:02, 03:171, 03:428	-				3									13	14																					27	28			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32							
*03:01:49, 03:204, 03:215, 03:414	-													13																							28			
*03:08:01:01-03:08:01:02, 03:36N, 03:89:01, 03:107, 03:176, 03:398, 03:413	A3, Null, -				3																																27			
*03:09	A3				3									13																							27	28	29	
*03:12	A3				3	4																															27	28		
*03:15, 03:19, 03:30	A3				3				5					13																							27	28		
*03:18, 03:135	-				4									13																							26	28		
*03:21N	Null				3									13																							27	28	30	31
*03:24	A3				3					8				13																								27	28	
*03:32	-				3																																	27		
*03:34, 03:62:01:01-03:62:01:02, 03:263	-				3									13																								27		
*03:41, 03:382	-				3																																	23	27	

Negativ

Negative Control

*26:78	-						8			11											23						29						
*26:92	-									11												24						29	30				
*26:120	-	1					8			10			13															30					
*26:122	-						8			10			13								20							30					
*26:152	-						8			10	11		13															30					
*26:177	-										11		13				18				21	23						29	30				
*26:193	-						8			10	11		13															29	30				
*26:198	-						8			10			13	14														30					
*26:204:01:01-26:204:01:02	-			4			8			10																							
*26:207	-									10			13								18							30					
*26:217	-						8			10		12	13															30					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*29:01:01:01-29:01:14, 29:02:01:01-29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40-29:47, 29:50-29:65, 29:68-29:73, 29:75-29:79, 29:81-29:97, 29:99-29:104, 29:107-29:118, 29:120-29:140, 29:142-29:154, 29:157-29:158, 29:160-29:163, 29:165	A29, Null, -														14											25			28				
*29:01:15, 29:119	-														14														28				
*29:07, 29:49	A29, -					5	6								14											25		28					
*29:13, 29:39	-, -														14				17							25		28					
*29:14	-														14			16								25		28					
*29:19	-														14											25			29				
*29:20, 29:34, 29:66, 29:74, 29:106	-														14											25							
*29:28	-							8							14											25		28					
*29:30, 29:141	-														14											25			29				
*29:48	-														14				18							25			29				
*29:67, 29:156	-				4										14											25		28					
*29:80	-														14							21				25		28					
*29:98	-														14				17							25							
*29:105, 29:164	-									10					14				18							25		28					
*29:155	-														14							21				25			28				
*29:159	-														14											25		28	29				
*30:01:01:01-30:01:07, 30:01:09-30:01:19, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48-30:49, 30:52-30:54, 30:56, 30:58-30:59N, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:88, 30:91-30:98, 30:102, 30:104, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:126, 30:128-30:131, 30:134-30:138, 30:141-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:164-30:165, 30:167-30:168, 30:170-30:173, 30:176, 30:178N-30:181, 30:184Q-30:185, 30:187, 30:190-30:192, 30:194-30:197N, 30:199	A30, Null, -														15										25			28					
*30:01:08, 30:18, 30:41, 30:60	A30, -														15														28				
*30:02:01:01-30:03, 30:07, 30:10, 30:22, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85, 30:100-30:101Q, 30:107-30:108, 30:119, 30:121N, 30:124, 30:127, 30:133, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156-30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198	A30, Null, -														15											25							
*30:04:01:01-30:04:03, 30:06, 30:09:01:01-30:09:01:02, 30:29, 30:77, 30:90, 30:99, 30:105, 30:117, 30:139, 30:155, 30:160	A30, -														15											25		27					
*30:08:01-30:08:02	A30, -				4										15											25		28					
*30:12:01-30:12:02, 30:31, 30:152, 30:166, B*07:260, C*12:328 ⁴	A30, -														15																		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*30:13, 30:44	A30, -															15										25			29				
*30:16	A30															15										25		28	29				
*30:17, 30:183	A30, -															15										25		27	28				
*30:46	-															15										25		27	29				
*30:55	-															13	15												28				
*30:89	-			3												15			17							25		27	28				
*30:103	-															15												27					
*30:125, 30:140	-											11				15										25							
*30:132N	Null															15										25		28		30	31		
*30:150	-				4											15										25		27					
*30:163	-								7							15										25		28					
*30:177	-															15			17							25							
*30:186	-													13	15											25		28					

Negative Control

