

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*01:01:01:01-01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:03:02, 01:09:01:01-01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303:01-01:306, 01:308N-01:313, 01:315-01:341, 01:343, 01:345-01:359, 01:362-01:375, 01:377-	A1, Null, -																																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:396Q, 01:403	A1, Null, -	1			4																						26							
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307, 01:342, 01:401	-				4																						26							
*01:01:56	-	1		4							12																26					30		
*01:04:01:01N-01:04:01:02N	Null	1		4																							26					30	31	
*01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1		4																														
*01:07, 01:95, 01:289	A1, -	1		4														17									26							
*01:12, 01:19, 01:200, 01:244	A1, -	1																										27						
*01:13, 01:176	A1, -	1		4							11		13	15													26	28						
*01:14, 01:391	A1, -	1																																
*01:15N	Null	1		w																														
*01:17, 01:302	A1, -	1		4																							26	28						
*01:21, 01:126	A1, -	1		4																							26	27						
*01:26, 01:192	A1, -	1		4							11																26							
*01:28, 01:299	-	1		4							11			15													26							
*01:43	-	1		4							10																26							
*01:51	-	1		4				8		10																	26							
*01:121	-	1		4															19								26							
*01:134	-	1		4																							26			29				
*01:136, 01:380:01-01-01:380:01:02	-	1		4							11																							
*01:194	-	1		4									13	15													26	28						
*01:229	-	1		4							11			15													26			29				
*01:298	-	1		4					9									17									26							
*01:301Q	-	1		4	5	7																					w							
*01:344	-	1		4				8																										
*01:360	-	1		4																									27					
*01:361N	Null	1		4							12																26							
*01:381	-	1		4				8																			26							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*02:01:01:01-02:01:07, 02:01:09-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:155, 02:157:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N,	A2, Low A2, A203, A210, Null, -																																	
*02:17:02:01-02:17:04	A2, -	2				w																												
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907	A2, -	2												13						20		22						28						
*02:35:02	A2													13						20		22						28						
*02:46, 02:70, 02:1050	A2, -	2																		20		22												
*02:48, 02:129, 02:571	A2, -																			20		22												
*02:55	A2, A28	2						8														21		23										

Negative Control

Negative Control

[illegible]

*03:246, 03:282	-			3									13				19										27	28					
*03:248	-			3									13														27	28			31		
*03:330N	Null			3																							27	28	29				
*03:347	-					5		7					13											23			28						
*03:377	-			3									13											23			28						
*03:395	-		1	3									13	15										23				27	28				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*11:01:01:01-11:02:07, 11:04-11:05, 11:07:01, 11:09, 11:12:02-11:16, 11:19, 11:22-11:24:02, 11:26-11:27, 11:29-11:30, 11:32:01-11:34, 11:36-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208:02N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q-11:281, 11:283-11:289, 11:291-11:305, 11:307-11:311, 11:313Q-11:324, 11:327-11:347N, 11:349-11:365N,	A11, Null, -																																
*11:03, 11:08, 11:20, 11:25:01-11:25:02, 11:31, 11:35, 11:60, 11:158, 11:175, 11:209, 11:348, 11:366, 11:381	A11, -				4							11															27						
*11:06, 11:18, 11:312	A11, -				4							11																	29				
*11:07:02, 11:372	-				4							11							19														
*11:10, 11:269	A11, -				4				8			11																					
*11:11	-				4							11	12														26						
*11:12:01	A11				4							11																28					
*11:17, 11:40, 11:223	-				4						10	11																					
*11:21N	Null				4							11																		30	31		
*11:94, 11:112, 11:211, 11:290	-				4							11															26						
*11:113, 11:162	-				4							11				15																	
*11:116, 11:140, 11:199:01-11:199:02, 11:222	-				4							11		13																			
*11:130	-			3	4							11			14										23								
*11:139	-				4	5		7				11																					
*11:155	-				4							11												22				28					
*11:166	-				4			6				11																					
*11:183	-				4				8			11																27					
*11:191	-				4				8			11																			31		
*11:199:03	-				4							11		13							20		22				26	28					
*11:226	-				4							11											22				26	28					
*11:271	-				4						10																						
*11:282	-		1		4							11			15																		
*11:306	-				4							11			16											25							
*11:325, 24:555 ⁴	-				4	5						11																					
*11:326	-				4							11															26	28					
*11:404	-				4							11																			31		
*23:01:01:01-23:06, 23:08N-23:10:02, 23:12, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:98, 23:100-23:103N, 23:106N-23:112, 23:114-23:122	A23, Null, -						5	6																									
*23:07N	Null						5	6																						30	31		
*23:11N	Null						5	6							16																		
*23:13	A23						5	6																					28				
*23:14:01-23:14:02, 23:104-23:105, 24:71, 24:315, 24:392, 24:527 ⁴	A23, A24, -						5	6	7																								
*23:57	-						5																21										
*23:64	-						5	6								17																	
*23:66, 23:99, 24:14:01:01-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, 24:291, 24:296, 24:304, 24:316, 24:324, 24:412, 24:481, 24:515. C*04:01:03 ⁴	A24, Null, Cw4, -																																
*23:113N	Null						5	7								16																	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*24:02:01:01-24:06, 24:08, 24:10:01:01-24:10:02, 24:13:01-24:13:02, 24:17:01:01-24:17:01:02, 24:20:01:01-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56:02, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:103, 24:105-24:107, 24:109-24:111, 24:113, 24:115-24:130, 24:132N-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:287, 24:289, 24:292-24:293, 24:295, 24:297-24:303N, 24:305-24:314, 24:317-24:323N, 24:325-24:338, 24:340-24:346, 24:348-24:386, 24:388N-24:391, 24:393-24:405, 24:407-24:411, 24:413-24:452, 24:454-24:456N, 24:459-24:470, 24:472-24:476, 24:478-24:480, 24:482-24:488, 24:490-24:509, 24:511-24:514N, 24:516-24:526, 24:529N-24:540, 24:542-24:543,	A24, Low A24, A2403, A9, Null, -						5		7																								

Negative Control

[illegible]

[illegible]

[illegible]

