

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*01:01:01:01-01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:59, 01:01:61-01:01:64, 01:01:67-01:01:78, 01:01:80-01:03:03, 01:09:01:01-01:11N, 01:16N, 01:18N, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:52:02N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170, 01:172-01:175, 01:177, 01:179N-01:187, 01:189-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:219, 01:221, 01:223-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:254, 01:256-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:303:01-01:306, 01:308N-01:313, 01:315-01:341, 01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:400N,	A1, Null, -	1			4																						26							
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:396Q, 01:403, 01:418N	A1, Null, -	1																									26							
*01:01:32, 01:01:60, 01:69:01-01:69:03, 01:143, 01:171, 01:188, 01:220, 01:307, 01:342, 01:401	-				4																						26							
*01:01:56	-	1			4							12															26					30		
*01:04:01:01N-01:04:01:02N	Null	1			4																						26					30	31	
*01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130, 01:222, 01:255	A1, Null, -	1			4																													
*01:07, 01:95, 01:289	A1, -	1			4													17									26							
*01:12, 01:19, 01:200, 01:244	A1, -	1																										27						
*01:13, 01:176	A1, -	1			4							11		13		15											26		28					
*01:14, 01:391	A1, -	1																																
*01:15N	Null	1			w																													
*01:17, 01:302	A1, -	1			4																						26		28					
*01:21, 01:126	A1, -	1			4																						26	27						
*01:26, 01:192	A1, -	1			4								11														26							
*01:28, 01:299	-	1			4								11			15											26							
*01:43	-	1			4							10															26							
*01:51	-	1			4				8			10															26							
*01:121	-	1			4															19							26							
*01:134	-	1			4																						26				29			
*01:136, 01:380:01:01-01:380:01:02	-	1			4								11														26							
*01:194	-	1			4									13		15											26		28					
*01:229	-	1			4								11			15											26				29			
*01:298	-	1			4					9								17									26							
*01:301Q	-	1			4	5		7																			w							
*01:344	-	1			4				8																									
*01:360	-	1			4																								27					
*01:361N	Null	1			4								12														26							
*01:381	-	1			4				8																		26							
*01:427, 36:02 ⁴	A36, -	1		3											14								22	23				26						
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	

Negative Control

[illegible]

*03:01:06, 03:01:14, 03:01:47, 03:01:52, 03:54, 03:64, 03:69N, 03:127, 03:140, 03:153, 03:165, 03:231:02, 03:234Q, 03:249, 03:288, 03:322, 03:416	A3, -		3									13																28			Negative Control			
*03:01:19, 03:103:02	-		3								12	13															24		27	28				
*03:01:23, 03:01:83, 03:01:94, 03:57, 03:59, 03:72, 03:111, 03:142, 03:173, 03:178N, 03:199, 03:203, 03:205, 03:211, 03:252, 03:267	Null, -		3																										27	28				
*03:01:30, 03:01:35, 03:01:50, 03:01:90, 03:01:106, 03:05:01-03:05:02, 03:10, 03:40, 03:42, 03:98, 03:105, 03:122, 03:167, 03:208, 03:225, 03:326, 03:332, 03:452	A3, -											13																	27	28				
*03:01:38, 03:123:02, 03:171, 03:428	-		3									13	14															27	28					
*03:01:49, 03:204, 03:215, 03:414	-											13																28						
*03:08:01:01-03:08:01:02, 03:36N, 03:107, 03:176, 03:398, 03:413	A3, Null, -		3																										27					
*03:09	A3		3									13																27	28	29				
*03:12	A3		3	4																								27	28					
*03:15, 03:19, 03:30	A3		3		5							13																27	28					
*03:18, 03:135	-			4								13															26	28						
*03:21N	Null		3									13																27	28		30			
*03:24	A3		3				8					13																27	28					
*03:32	-		3													17												27						
*03:34, 03:62:01:01-03:62:01:02, 03:263, 03:447, 03:451	-		3									13																27						
*03:41, 03:382	-		3																								23		27					
*03:43, 03:82, 03:186	-		3									13	15	17														27	28					
*03:50	-		3				8					13																	28					
*03:63	-		3							11		13															23		27					
*03:75, 03:177	-											13															23		28					
*03:88	-			3	4						11																23		27					
*03:89:01	-	2	3																									27						
*03:89:02, 03:108, 03:172, 03:198	-		3																									27		29				
*03:95, 03:343	-											13	14	16													23		28					
*03:152, 03:273	-		3		5										17														27					
*03:187	-											13															22		28					
*03:219, 03:429	-			3												17													27	28				
*03:231:01	-			3								13	14																28					
*03:246, 03:282	-			3								13					19												27	28				
*03:248	-			3								13																	27	28		31		
*03:330N	Null			3																									27	28	29			
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³						5		7						13													23		28				
*03:347	-					5		7							13													23		28				
*03:377	-			3											13													23		28				
*03:395	-	1		3											13	15														27	28			
*11:01:01:01-11:02:07, 11:04-11:05, 11:07:01, 11:09, 11:12:02-11:16, 11:19, 11:22-11:24:02, 11:26-11:27, 11:29-11:30, 11:32:01-11:34, 11:36-11:39, 11:41-11:52Q, 11:54:01-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208:02N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:268, 11:270, 11:272Q-11:281, 11:283-11:289, 11:291-11:305, 11:307-11:311, 11:313Q-11:324, 11:327-11:347N, 11:349-11:365N, 11:367-11:371, 11:373-11:380, 11:382N-11:403N,	A11, Null, -			4								11																						
*11:03:01:01-11:03:01:02, 11:08, 11:20, 11:25:01-11:25:03, 11:31, 11:35, 11:60, 11:158, 11:175, 11:209, 11:348, 11:366, 11:381	A11, -			4								11																		27				
*11:06, 11:18, 11:312	A11, -			4								11																			29			
*11:07:02, 11:372	-			4								11							19															
*11:10, 11:269	A11, -			4				8				11																						
*11:11	-			4								11	12															26						
*11:12:01	A11			4								11																		28				
*11:17, 11:40, 11:223	-			4						10	11																							
*11:21N	Null			4								11																			30	31		
*11:94, 11:112, 11:211, 11:290	-			4								11															26							
*11:113, 11:162	-			4								11																						
*11:116, 11:140, 11:199:01-11:199:02, 11:222	-			4								11			13																			
*11:130	-		3	4								11			14										23									
*11:139	-			4	5		7					11																						
*11:155	-			4								11																		28				
*11:183	-			4				8				11																		27				
*11:191	-			4				8				11																				31		
*11:199:03	-			4								11			13					20				22										
*11:226	-			4								11												22										
*11:271	-			4						10																				26	28			

Negative Control

Negative Control

Negative Control

*32:26:01-32:26:02, 32:152	-								9				13			17										25							
*32:42	-								9				14			17										25							
*32:64	-			4					9							17									25								
*32:72	-					6			9							17									25								
*32:93	-								9							17	18								25								
*32:128	-								9							16	17								25								
*32:156	-			1					9							17									25								
*32:170	-					5	6		9							17									25								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																
*33:01:01:01-33:01:18, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:162, 33:164-33:183, 33:186, 33:188-33:189, 33:191-33:234	A33, Null, -																		18							25							
*33:13, 33:48	-									10								18								25							
*33:19	-					5	7											18							25								
*33:22	-																	18			21				25								
*33:24	-																	18					23		25				29				
*33:49, 33:144	-																	18							25			27					
*33:51	-							8										18							25								
*33:61, 33:190	-												13					18							25								
*33:119	-						7											18							25								
*33:125, 33:131	-															16		18							25			28					
*33:163, 33:185	-																	18							25								
*33:184	-									11								18					23		25				29				
*33:187	-				5	6												18							25								
*34:01:01:01-34:01:07, 34:05, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 34:27	A34, -							8			11	12																					31
*34:02:01:01-34:02:07, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24-34:26N, 34:28-34:30N	A34, Null, -							8			11	12															27						31
*34:03	A34							8			11	12														27							
*34:04	A34							8			11	12	14													27							31
*34:06, 66:40 ⁴	A10, -							8			11	12																					
*34:08	A34							8			11	12	13													27							31
*34:09	-							8			12													24		27	28						31
*34:19	-			4							11	12																					
*34:22	-							8			11	12										22				27							
*36:01:01:01-36:01:02, 36:03, 36:05-36:13	A36, -	1																					22										
*36:04	A36	1		4																			22										
*43:01-43:03	A43, Null, -									10	12	13														26							30
*66:01:01:01-66:01:01:06, 66:01:03-66:01:07, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41, 66:44-66:47	A66, A26, Null, -							8			11	13																					30
*66:01:02	-							8			11	13																					
*66:02:01:01-66:02:01:03, 66:16, 66:21, 66:25-66:26Q, 66:34, 66:43	A66, -							8			12													24			28						
*66:03:01:01-66:03:01:03, 66:28N, 66:39N	A10, Null, -							8			12													24									
*66:06	A66							8			11	13										21											30
*66:09	-							8			11	13	14									21											30
*66:10	-							8			11	w																					30
*66:12	-							8				13												24			28						30
*66:19	-							8			11	12	13																				30
*66:23	-			4							11	13																					30
*66:42	-										11	13												24									30
*68:01:01:01-68:01:17, 68:01:19-68:01:56, 68:01:58-68:02:19, 68:02:21-68:02:22, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30, 68:32-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:73-68:83, 68:85-68:89, 68:91-68:92, 68:94N-68:96, 68:98-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179, 68:181N-68:205N, 68:207-68:211, 68:213N-68:217, 68:219-68:226, 68:228-68:229, 68:231-68:248, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:280, 68:282-68:285, 68:287-68:290, 68:292-68:295, 68:297-68:300, 68:302-68:305, 68:307-68:310, 68:312-68:315, 68:317-68:320, 68:322-68:325, 68:327-68:330, 68:332-68:335, 68:337-68:340, 68:342-68:345, 68:347-68:350, 68:352-68:355, 68:357-68:360, 68:362-68:365, 68:367-68:370, 68:372-68:375, 68:377-68:380, 68:382-68:385, 68:387-68:390, 68:392-68:395, 68:397-68:400, 68:402-68:405, 68:407-68:410, 68:412-68:415, 68:417-68:420, 68:422-68:425, 68:427-68:430, 68:432-68:435, 68:437-68:440, 68:442-68:445, 68:447-68:450, 68:452-68:455, 68:457-68:460, 68:462-68:465, 68:467-68:470, 68:472-68:475, 68:477-68:480, 68:482-68:485, 68:487-68:490, 68:492-68:495, 68:497-68:500, 68:502-68:505, 68:507-68:510, 68:512-68:515, 68:517-68:520, 68:522-68:525, 68:527-68:530, 68:532-68:535, 68:537-68:540, 68:542-68:545, 68:547-68:550, 68:552-68:555, 68:557-68:560, 68:562-68:565, 68:567-68:570, 68:572-68:575, 68:577-68:580, 68:582-68:585, 68:587-68:590, 68:592-68:595, 68:597-68:600, 68:602-68:605, 68:607-68:610, 68:612-68:615, 68:617-68:620, 68:622-68:625, 68:627-68:630, 68:632-68:635, 68:637-68:640, 68:642-68:645, 68:647-68:650, 68:652-68:655, 68:657-68:660, 68:662-68:665, 68:667-68:670, 68:672-68:675, 68:677-68:680, 68:682-68:685, 68:687-68:690, 68:692-68:695, 68:697-68:700, 68:702-68:705, 68:707-68:710, 68:712-68:715, 68:717-68:720, 68:722-68:725, 68:727-68:730, 68:732-68:735, 68:737-68:740, 68:742-68:745, 68:747-68:750, 68:752-68:755, 68:757-68:760, 68:762-68:765, 68:767-68:770, 68:772-68:775, 68:777-68:780, 68:782-68:785, 68:787-68:790, 68:792-68:795, 68:797-68:800, 68:802-68:805, 68:807-68:810, 68:812-68:815, 68:817-68:820, 68:822-68:825, 68:827-68:830, 68:832-68:835, 68:837-68:840, 68:842-68:845, 68:847-68:850, 68:852-68:855, 68:857-68:860, 68:862-68:865, 68:867-68:870, 68:872-68:875, 68:877-68:880, 68:882-68:885, 68:887-68:890, 68:892-68:895, 68:897-68:900, 68:902-68:905, 68:907-68:910, 68:912-68:915, 68:917-68:920, 68:922-68:925, 68:927-68:930, 68:932-68:935, 68:937-68:940, 68:942-68:945, 68:947-68:950, 68:952-68:955, 68:957-68:960, 68:962-68:965, 68:967-68:970, 68:972-68:975, 68:977-68:980, 68:982-68:985, 68:987-68:990, 68:992-68:995, 68:997-68:1000	A68, A28, Null, -							8													20											28	
*68:01:18, 68:01:57, 68:02:20, 68:02:23-68:04:02, 68:31, 68:72, 68:90, 68:93, 68:97, 68:180, 68:206, 68:212	A68, -							8													20												
*68:05, 68:15, 68:20, 68:255	A68, -							8													20			23									29
*68:08:02, 68:105:01:01-68:105:01:02, 68:157, 68:276Q	A68, -							8													20	21											
*68:11N, 68:41, 68:230	Null, -							8													20			22									28
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -			4				8																									

Negative Control