

*26:207	-									10		13				18						25					30							
*26:217, 26:231	-								8	10	12	13															30							
*26:227	-								8			13										23					29 30							
*26:229	-								8		11 12	13										20		23			29 30							
*29:01:01:01-29:01:14, 29:02:01:01-29:06, 29:08N-29:12, 29:15-29:18, 29:21-29:27, 29:29, 29:31-29:33, 29:35-29:38, 29:40-29:47, 29:50-29:65, 29:67:02-29:73, 29:75-29:79, 29:81-29:97, 29:99-29:104, 29:107-29:118, 29:120-29:140, 29:142-29:154, 29:157-29:158, 29:160-29:163, 29:165-29:171, 29:173, 29:175Q, 29:177-29:179	A29, Null, -												14											25		28								
*29:01:15, 29:119	-												14														28							
*29:07, 29:49	A29, -					5	6						14											25		28								
*29:13, 29:39	-												14			17								25		28								
*29:14, 29:172	-												14		16									25		28								
*29:19	-												14											25			29							
*29:20, 29:34, 29:66, 29:74, 29:106	-												14											25										
*29:28, 29:176	-								8				14											25		28								
*29:30, 29:141	-												14																					
*29:48	-												14			18								25		29								
*29:67:01, 29:156	-				4								14											25		28								
*29:80	-												14									21		25		28								
*29:98	-												14			17								25										
*29:105, 29:164	-										10		14			18								25		28								
*29:155	-												14										21			28								
*29:159	-												14											25		28 29								
*29:174	-												14										22		25		28							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-A allele ^{1,2}	ser ³																																	
*30:01:01:01-30:01:07, 30:01:09-30:01:24, 30:11:01-30:11:02, 30:14L-30:15, 30:19-30:20, 30:23-30:26, 30:30, 30:35-30:40, 30:42-30:43, 30:48-30:49, 30:52-30:54, 30:56, 30:58-30:59N, 30:62-30:63, 30:65, 30:71-30:75, 30:78N-30:79, 30:81-30:83, 30:86-30:88, 30:91-30:98, 30:102, 30:104, 30:106, 30:109-30:116, 30:118, 30:120, 30:122-30:123N, 30:126, 30:128-30:131, 30:134-30:138, 30:141-30:143, 30:145N, 30:147-30:148, 30:154, 30:159, 30:161, 30:164-30:165, 30:167-30:168, 30:170-30:173, 30:176, 30:178N-30:181, 30:184Q-30:185, 30:187, 30:190-30:192, 30:194-30:197N, 30:199, 30:201-30:203, 30:205, 30:207, 30:209N-30:211	A30, Null, -														15										25		28							
*30:01:08, 30:18:01-30:18:02, 30:41, 30:60	A30, -															15																		28
*30:02:01:01-30:03, 30:07, 30:10, 30:27N-30:28, 30:32-30:34, 30:45, 30:47, 30:50-30:51, 30:57, 30:61, 30:64, 30:66-30:70N, 30:76N, 30:80, 30:84-30:85, 30:100-30:101Q, 30:107-30:108, 30:119, 30:121N, 30:124, 30:127, 30:133, 30:144, 30:146, 30:149, 30:151, 30:153, 30:156-30:158N, 30:162, 30:169, 30:174-30:175, 30:182, 30:188-30:189, 30:193, 30:198, 30:200, 30:204N, 30:206, 30:208N, 30:213	A30, Null, -														15										25									
*30:04:01:01-30:04:04, 30:06, 30:09:01:01-30:09:01:02, 30:29, 30:77, 30:90, 30:99, 30:105, 30:117, 30:139, 30:155, 30:160, 30:212	A30, -															15										25		27						
*30:08:01-30:08:02	A30, -				4											15									25		28							
*30:12:01-30:12:02, 30:31, 30:152, 30:166, B*07:260, C*12:328 ⁴	A30, -															15																		
*30:13, 30:44	A30, -			2												15									25							29		
*30:16	A30			2												15									25		28	29						
*30:17, 30:183	A30, -															15									25		27 28							
*30:22	A30			2												15									25									
*30:46	-			2												15									25		27	29						
*30:55	-													13		15																	28	
*30:89	-				3											15			17						25		27 28							
*30:103	-															15												27						
*30:125, 30:140	-											11				15									25									
*30:132N	Null															15									25		28				30 31			
*30:150	-				4											15									25		27							
*30:163	-								7							15									25		28							
*30:177	-															15			17						25									
*30:186	-													13		15									25		28							

Negative Control

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-A allele ^{1,2}	ser ³																																		
<i>B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60</i>	B53, -									9																									
<i>B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569, B*07:990, B*07:994</i>	-																						22												
<i>B*15:642</i>	-																	17																	
<i>B*18:27</i>	-						6																												
<i>B*35:01:60, B*58:01:24</i>	-		w																																
<i>C*03:365, C*12:131, B*12:388</i>	-				4																														
<i>C*06:187</i>	-																																		
<i>C*06:247</i>	-																																		29
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		

Negative Control